

SIZE SELECTION CHART FOR YOKE SLINGS

This document is intended as a guide only. Correct selection for an individual patient remains the responsibility of the healthcare professional or case manager.

It is critical to select the correct size of sling to match the individual patient. Typically there is a degree of overlap between each


sling size within a particular model. If the patient falls between two sizes, it is generally better to select the smaller size of sling, unless you are aware the patient is likely to grow.

Please note, however, that selecting a sling involves more than simply measuring a patient's size.

To be confident a sling meets individual requirements and is the most appropriate sling available, the following basic factors should be taken into consideration:

- patient information
- lifting task and environment
- carer information
- other equipment information

For more details, please refer to our ProSling Range Guide available to download from www.novis.com.au

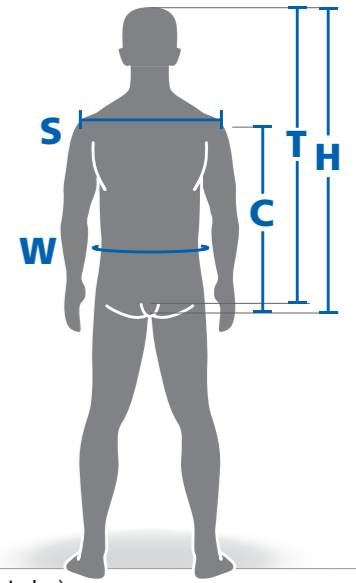

Yoke General Purpose	S Shoulder width mm																				C Coccyx to Shoulders mm																																				
	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1100-1120	1120-1140	1140-1160	1160-1180	1180-1200	1200-1220	1220-1240	1240-1260	1260-1280	1280-1300	440-460	460-480	480-500	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780
SL147181	Small															Small																																									
SL147281						Medium															Medium																																				
SL147381						Large															Large																																				

Yoke General Purpose with Head Support	S Shoulder width mm																				H Coccyx to top of head mm (Sitting Height)																																																
	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1100-1120	1120-1140	1140-1160	1160-1180	1180-1200	1200-1220	1220-1240	1240-1260	1260-1280	1280-1300	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1100-1120	1120-1140	1140-1160	1160-1180	1180-1200	1220-1240
SL146181	Small															Small																																																					
SL146281						Medium															Medium																																																
SL146381						Large															Large																																																
SL146481																Extra Large					Extra Large																																																
SL146581																					Extra Extra Large																																																

Yoke Hygiene with Head Support	S Shoulder width mm																				T Sacrum to top of head mm (Sitting Height)																																						
	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080
SL246181						Small															Small																																						
SL246281						Medium															Medium																																						
SL246381						Large															Large																																						
SL246481																Extra Large					Extra Large																																						
SL246581																					Extra Extra Large																																						

Yoke Hygiene	S Shoulder width mm																				W Waist circumference mm																																								
	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1120-1140	1140-1160	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1120-1140
SL267181	Small																				Small																																								
SL267281						Medium															Medium																																								
SL267381						Large															Large																																								

Disposable General Purpose with Head Support	S Shoulder width mm																				H Coccyx to top of head mm (Sitting Height)																																								
	500-520	520-540	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1100-1120	1120-1140	540-560	560-580	580-600	600-620	620-640	640-660	660-680	680-700	700-720	720-740	740-760	760-780	780-800	800-820	820-840	840-860	860-880	880-900	900-920	920-940	940-960	960-980	980-1000	1000-1020	1020-1040	1040-1060	1060-1080	1080-1100	1100-1120
SL156281						Medium															Medium																																								
SL156381						Large															Large																																								

SIZE SELECTION CHART FOR PIVOT SLINGS

Pivot Hygiene	S Shoulder width mm																																						
	500 - 520	520 - 540	540 - 560	560 - 580	580 - 600	600 - 620	620 - 640	640 - 660	660 - 680	680 - 700	700 - 720	720 - 740	740 - 760	760 - 780	780 - 800	800 - 820	820 - 840	840 - 860	860 - 880	880 - 900	900 - 920	920 - 940	940 - 960	960 - 980	980 - 1000	1000 - 1020	1020 - 1040	1040 - 1060	1060 - 1080	1080 - 1100	1100 - 1120	1120 - 1140	1140 - 1160	1160 - 1180	1180 - 1200	1200 - 1220	1220 - 1240	1240 - 1260	1260 - 1280
SL247191	Small																																						
SL247291	Medium																																						
SL247391	Large																																						

Pivot General Purpose with Head Support	S Shoulder width mm																																						
	500 - 520	520 - 540	540 - 560	560 - 580	580 - 600	600 - 620	620 - 640	640 - 660	660 - 680	680 - 700	700 - 720	720 - 740	740 - 760	760 - 780	780 - 800	800 - 820	820 - 840	840 - 860	860 - 880	880 - 900	900 - 920	920 - 940	940 - 960	960 - 980	980 - 1000	1000 - 1020	1020 - 1040	1040 - 1060	1060 - 1080	1080 - 1100	1100 - 1120	1120 - 1140	1140 - 1160	1160 - 1180	1180 - 1200	1200 - 1220	1220 - 1240	1240 - 1260	1260 - 1280
SL146191	Small																																						
SL146291	Medium																																						
SL146391	Large																																						

This document is intended as a guide only. Correct selection for an individual patient remains the responsibility of the healthcare professional or case manager.


It is critical to select the correct size of sling to match the individual patient. Typically there is a degree of overlap between each

sling size within a particular model. If the patient falls between two sizes, it is generally better to select the smaller size of sling, unless you are aware the patient is likely to grow.

To be confident a sling meets individual requirements and is the most appropriate sling available, the following basic factors should be taken into consideration:

- patient information
- lifting task and environment
- carer information
- other equipment information

For more details, please refer to our ProSling Range Guide available to download from www.novis.com.au

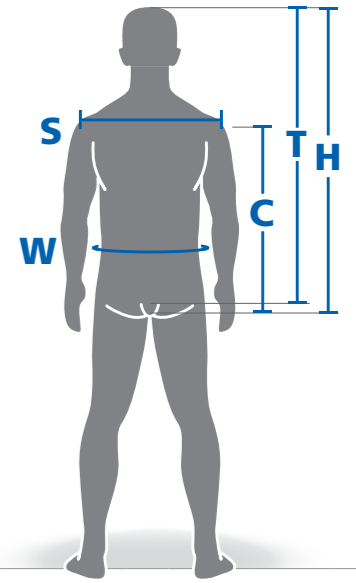

Pivot General Purpose with Head Support	H Coccyx to top of head mm (Sitting Height)																															
	540 - 560	560 - 580	580 - 600	600 - 620	620 - 640	640 - 660	660 - 680	680 - 700	700 - 720	720 - 740	740 - 760	760 - 780	780 - 800	800 - 820	820 - 840	840 - 860	860 - 880	880 - 900	900 - 920	920 - 940	940 - 960	960 - 980	980 - 1000	1000 - 1020	1020 - 1040	1040 - 1060	1060 - 1080	1080 - 1100	1100 - 1120	1120 - 1140		
SL146191	Small																															
SL146291	Medium																															
SL146391	Large																															

SIZE SELECTION CHART FOR GAIT TRAINING VEST

Gait Training Vest	W Waist circumference mm																																						
	520 - 540	540 - 560	560 - 580	580 - 600	600 - 620	620 - 640	640 - 660	660 - 680	680 - 700	700 - 720	720 - 740	740 - 760	760 - 780	780 - 800	800 - 820	820 - 840	840 - 860	860 - 880	880 - 900	900 - 920	920 - 940	940 - 960	960 - 980	980 - 1000	1000 - 1020	1020 - 1040	1040 - 1060	1060 - 1080	1080 - 1100	1100 - 1120	1120 - 1140	1140 - 1160	1160 - 1180	1180 - 1200	1200 - 1220	1220 - 1240	1240 - 1260	1260 - 1280	1280 - 1300
SL346191	Small																																						
SL346291	Medium																																						
SL346391	Large																																						

This document is intended as a guide only. Correct selection for an individual patient remains the responsibility of the healthcare professional or case manager.

It is critical to select the correct size of sling to match the individual patient. Typically there is a degree of overlap between each

sling size within a particular model. If the patient falls between two sizes, it is generally better to select the smaller size of sling, unless you are aware the patient is likely to grow.

Please note, however, that selecting a sling involves more than simply measuring a patient's size.

To be confident a sling meets individual requirements and is the most appropriate sling available, the following basic factors should be taken into consideration:

- patient information
- lifting task and environment
- carer information
- other equipment information

For more details, please refer to our ProSling Range Guide available to download from www.novis.com.au

