



RM200S

RM X Fold Rollator

Assembly / Adjustment

Your new Aluminium Rollator is suitable for everyday use. It can be used for both indoor and outdoor purposes. The lightweight folding design offers easy transportation and the sturdy aluminium frame gives stability and support when in use.

Please follow these simple steps to assemble your rollator -

1. Place your rollator on a level surface



2. Unfold the two front wheels until they lock into position, then insert the star knob as indicated and turn clockwise until fully secured.



3. Remove the locking pin, unfold the two rear wheels until the holes on the frame and the leg line up, replace the pin and lock in with the attached bracket ensuring it is secure.



4. Now your rollator is ready to use. Always make sure you check the rollator has been assembled correctly before use.

Maximum User Weight

<u>Product Code</u>	<u>Description</u>	<u>Weight Limit</u>
RM200S	RM X Fold Rollator	130kg

It is important to never exceed this limit and always ensure that the product weight capacity is sufficient before using the product. Failure to do so could result in injury or damage the product.

Care & Maintenance

All of the product components are manufactured from high quality materials. However, it is very important that you carry out regular checks: all nuts, bolts, knobs and handles should remain secure. Wheels should move freely and brakes engage when applied. To ensure the best from your rollator it should be kept well maintained.

In the event of identifying any irregularities you should contact your local distributor for advice.

NB: Your Rollator should only be used for its intended purpose and for highest safety you should not push anyone seated.

Warranty

All products carry a 12 month warranty from the date of purchase against manufacturing defects.

Misuse or Damage is not covered.

This warranty does not affect your statutory rights.
