

Torbay Bariatric Perching Stools



VG868W



VG869W



VG870W



VG871W

Fixing and Maintenance Instructions



INTRODUCTION

Thank you for deciding to purchase the Torbay Perching Stool from Aidapt. The Torbay Perching Stool range is manufactured from the finest material and components available, when installed and used correctly it is designed to give many years of reliable trouble-free service. The Torbay Perching Stool is available with optional arms, back or padded back.

N.B. This equipment must be installed by a competent person and a risk assessment may be needed as to the suitability of the chair for the particular user.

254 kg (40 stone) weight limit.

Do not exceed the weight limit stated – doing so could damage the equipment or put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

USE OF THE PERCHING STOOL

When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.



Do not allow children or unauthorised persons to play with or use the stool.



Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.



Do not use the perching outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury.



Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.


HEIGHT ADJUSTMENT

Remove the 'E' clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. (see fig. 1)

Once you have achieved the desired height; reinstall the 'E' clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg. (see fig. 2)



NB. The perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.

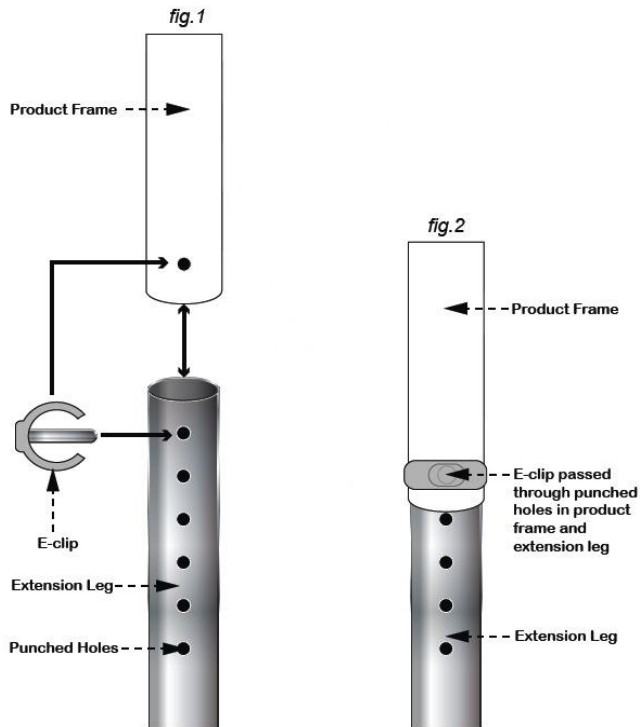

COMPOSITION

The perching stools are made from mild steel PPA/ epoxy coated with upholstered CMHR foam and fire retardant vinyl seats or contoured polyurethane seats.

CLEANING

Prior to cleaning remove all pin clips and the four extension legs with the ferrules fitted.

Clean these items separately to the main frame.

Clean the upholstery using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads will seriously damage the upholstery beyond repair.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/bolts/castors and other screw in/bolt in/push in components.
- Also check all upholstery (if applicable) for security, splits, exposed staples, etc.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

When reissuing, a risk assessment may be required as to the suitability of the chair for a particular user.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.