# PALDA®

## Paladon® 65

### Paladon® 65

- DE** Gebrauchsanweisung
- GB** Instructions for use
- FR** Mode d'emploi
- ES** Instrucciones de uso
- IT** Istruzioni per l'uso
- NL** Gebruiksaanwijzing
- DK** Brugervejledning
- GR** Οδηγίες χρήσεως
- CZ** Návod k použití
- HR** Upute za uporabu

**KULZER**  
MITSUBISHI CHEMICALS GROUP

**Manufacturer:**  
Kulzer GmbH  
Leipzig Straße 2  
63450 Hanau (Germany)

+49 800 4372522

Made in Romania

Distributed in USA/Canada exclusively by:  
Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785

Caution: Federal law restricts this device to sale by or on the order of a dental professional.

660873/39/01

## **GB** Instructions for use Paladon® 65

**Medical Device – for use by dental health care professionals only.**

### Intended purpose

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and oral aesthetics. They stabilize the retention of dentition and/or the alveolar ridge restoratively or prosthodontically.

### Product description

Denture acrylic for full dentures, heat-curing acrylic.  
Paladon 65 consists of a powder component in a range of colours (pink, pink veined, R50 veined, clear) and liquid.  
For use in pack and press technique, and also injection technique for fixed and removable dentures.

### Composition

#### Main powder component:

Poly methyl methacrylate  
**Main liquid component:**  
Methyl methacrylate, dimethacrylate  
Paladon 65 is cadmium-free.  
Contains benzoyl peroxide.

### Process and indications

#### Pack and press technique:

- partial dentures
- full dentures

#### Contraindications

Direct lining with Paladon 65 is contraindicated. The use of this product is contraindicated in case of known or suspected allergies against components of this product. Do not use in case of known or suspected allergy against (meth)acrylate compounds or benzoyl peroxide.

#### Side effects

This product or one of its components may in particular cases cause hypersensitive reactions. If suspected, information on the ingredients can be obtained from the manufacturer. Hypersensitivity reactions (allergies), local sensory disturbances, taste disturbances and irritation of the oral mucosa have been described in isolated cases. After processing, all MMA/PMMA-based denture materials always contain a permitted, small proportion of residual monomer (< 4.5 %). For further risk reduction of intolerance reactions, the denture has to be stored in tepid water for at least 12 hours before placement.

#### Handling

A hard stone (type 3) is sufficient for investment of the prosthesis set up in wax. After wax investment with boiling water, the plaster is thinly isolated with Aislar® twice (see Aislar Instructions for Use). To improve the bond between Paladon 65 and the prefabricated teeth, the basal surfaces are roughened with a coarse diamond bur (remove grinding dust) and coated with Palabond®. To prevent wax using a brush (without metal ferrule) and allow to set for 30 seconds each time. After the second application; the adhesive remains active for 10 minutes.

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© = marca registrada de la empresa Kulzer GmbH

### General information

Please, carefully observe the following safety instructions and all processing notes in other sections.  
This product is intended to be used only in accordance with this Instructions for Use. Any use of this product inconsistent with these instructions is at the discretion and sole responsibility of the dental health care professional.

#### Warnings and safety instructions

May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water and soap. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If symptoms persist, consult an eye specialist. Highly flammable liquid and vapour. Flash point: 10°C. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources – no smoking. May cause respiratory irritation. Avoid breathing dust/fume/gas/mist/vapours/spray. Toxic to aquatic life with long lasting effects.

#### Application notes

Examine the integrity of the primary packaging before first use. Damaged products must not be used. When positioning teeth in wax, do not use alcohol based separating agents (separation of gypsum and wax), as this may cause white discoloration on the basal surface of the prosthesis.

When giving feedback on the product, please always state batch designation and article number.

According to EU Medical Device Regulation users' patients are obliged to report serious incidents with a medical device to the manufacturer and to the competent authority of the country, the incident occurred.

#### Disposal information

Recommendation: Dispose of in accordance with official regulations. Do not dispose of contents or partially emptied packaging in the household waste or allow them to enter the sewage system. European Waste Catalogue: 180106 Chemicals consisting of or containing hazardous substances.

Safety data sheets and more information are available at our website [www.kulzer.com](#)

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\* = registered trademark of Kulzer GmbH