


artyst

NR.1

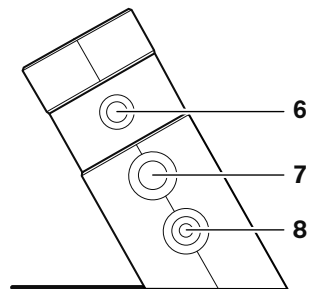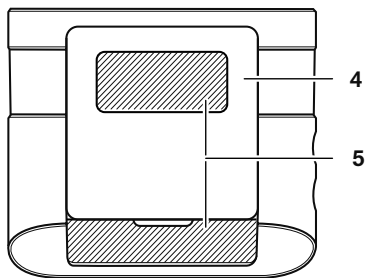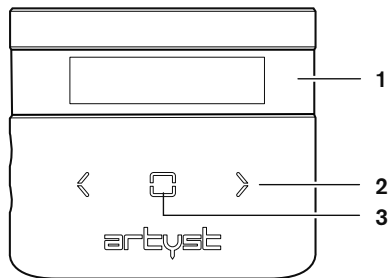Gebrauchsanweisung
Instruction for use
Manuel d'utilisation
Istruzioni per l'uso
Manual de instruções
Instrucciones de uso
инструкции

artyst NR.1

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Original

Congratulations on your purchase of this micropigmentation device which, as with all our micropigmentation devices, was produced by ISO 13485-certified manufacturing plants located in Germany.

artyst
NR.1

Instruction for use

Table of Contents

1	About this instruction for use	25	6	Using the device	36
1.1	Warning symbols and statements used in this instruction for use.....	25	6.1	Switching the control unit into operating mode or standby mode	36
2	Important safety notices	26	6.2	Setting the Piercing Speed	37
2.1	General safety information.....	26	6.3	Switching the handpiece on and off.....	37
2.2	Product-dependent safety information.....	26	7	Cleaning and maintenance	38
2.3	Important hygiene and safety regulations	26	7.1	Routine maintenance.....	38
2.4	Contraindications and side effects	28	7.2	Material compatibility.....	38
2.5	Purpose, application area and intended use.....	30	7.3	Disinfecting surfaces	39
2.6	Symbols on the product.....	30	7.4	Cleaning surfaces	39
3	Scope of delivery	31	8	Transport and storage conditions	39
4	Product information.....	31	9	Troubleshooting.....	39
4.1	Technical data.....	31	9.1	Remedying problems when operating the handpiece	39
4.2	Operating conditions	31	9.2	Eliminating malfunctions of the device.....	40
4.3	Parts and accessories.....	32	10	Manufacturer declarations	40
5	Commissioning.....	32	10.1	Warranty	40
5.1	Setting up or adhering the device.....	33	10.2	Declaration of Conformity	41
5.2	Connecting the power supply.....	34			
5.3	Connecting the foot switch (optional).....	35			
5.4	Connecting the handpiece	35			
5.5	Cleaning and disinfecting the equipment.	35			
5.6	Checking the equipment.....	36			

1 About this instruction for use

This operating manual applies to the Permanent Make-Up (PMU) Control unit artist NO.1 and to its accessories. It contains important information about how to commission, operate, and maintain the device safely and in the intended manner.

This operating manual does not contain all of the information needed for the safe operation of the artist PMU handpieces and their accessories. Therefore, observe the following additional documents:

- Operating manual for the artist PMU handpieces
- Information about PMU needle cartridges and PMU inks
- Occupational safety regulations and statutory provisions for micropigmentation in your country

1.1 Warning symbols and statements used in this instruction for use

Warning notices draw attention to the risk of personal injury and damage to materials and are structured as follows:

 SIGNAL WORD	
Type of hazard	
Consequences	
▶ Remedy	
Element	Meaning
	Indicates a hazard of injury
Signal word	Specifies the severity of the hazard (see table below)
Type of hazard	Gives the type and source of the hazard
Consequences	Describes potential consequences of non-compliance
Remedy	Indicates how to avoid a hazard

Signal word	Meaning
Hazard	Indicates a hazard that will definitely lead to death or to serious injury if the hazard is not avoided
Warning	Indicates a hazard that may lead to death or to serious injury if the hazard is not avoided
Caution	Indicates a hazard that can lead to minor to relatively serious injury if the hazard is not avoided
Attention	Indicates potential risks to the environment, property or equipment if this hazard is not avoided

Symbols used in this operating manual

Symbol	Meaning
▶	Call for action
•	List point
-	List sub-point

2 Important safety notices

2.1 General safety information

- ▶ Carefully read this entire operating manual.
- ▶ Keep this operating manual in a readily accessible location for everyone who uses, cleans, disinfects, stores or transports the device.
- ▶ Always include this operating manual when passing the device on to other persons.
- ▶ Follow the safety regulations applicable to micropigmentation in your country. Keep your workstation in a hygienic, clean condition and provide sufficient lighting.
- ▶ Use the device, its accessories and all handpiece cables only in perfect technical condition.
- ▶ Only use original needle cartridges, accessories and spare parts from MT.DERM GmbH.

2.2 Product-dependent safety information

- ▶ Never modify the device, the power supply or other accessories.
- ▶ Prevent fluids from entering the device and the power supply plug.
- ▶ Protect the device with a plastic sheet during use.
- ▶ Disconnect the power supply when the device is not in use.
- ▶ Route all cables so that they cannot kink.
- ▶ Observe the technical data (see chapter 4.1 on page 31) specified in this operating manual and observe the operating conditions (see chapter 4.2 on page 31) and transport and storage conditions (see chapter 8 on page 39).
- ▶ Hand over the device to a specialist retailer for inspection if it exhibits any external signs of damage, does not function as usual or fluids have run into the device or handpiece.

- ▶ It is recommended that the device be handed over to a specialist retailer for inspection at regular intervals of 24 months. Observe chapter 7.1 on page 38 for this.

2.3 Important hygiene and safety regulations

Please observe the following instructions to prevent contamination or infectious diseases from being transmitted to the client or user during treatment:

- ▶ Before treatment, follow all the steps required to disinfect the equipment (see chapter 5.5 on page 35).
- ▶ During treatment, wear disposable nitrile or latex gloves and disinfect them before use. When selecting an appropriate disinfectant, take note of the applicable regulations in your country.
- ▶ Before treatment, clean the affected areas of the client's skin with a mild cleanser and disinfectant. When selecting an appropriate disinfectant, take note of the applicable regulations in your country.

- ▶ Only use PMU inks, which have been tested and approved for penetrating human skin, for the treatment. Observe the applicable regulations in your country when making the selection. If PMU inks are introduced into the skin that are not intended for permanent make-up treatment or whose sterility is not guaranteed, infections or possible side effects may occur.
 - ▶ Only use new, sterile-packaged safety cartridges for each client for each new application. Before each usage, ensure that the packaging is undamaged and that the use-by date has not passed. Note the lot number of the needle cartridges used in the corresponding customer file in order to be able to inform the manufacturer in the event of problems.
 - ▶ needle cartridges are sterile, disposable products (consumable) and may always only be used once!
 - ▶ Dispose of used or defective needle cartridges and needle cartridges, whose packaging is damaged, in a puncture-proof container (so-called sharps container) according to the regulations in your country.
 - ▶ Check that the needles are completely retracted into the needle cartridge before the treatment and with the handpiece switched off. If this is not the case, dispose of the needle cartridge immediately.
 - ▶ Prevent the needle cartridges from coming into contact with contaminated objects, e.g. clothing. Contaminated needle cartridges may not be used, rather must be disposed immediately.
 - ▶ During use, the handpiece, handpiece cable and device including integrated handpiece tray, must be covered with a plastic sheet. The handpiece is covered with the plastic sheet before the needle cartridge is attached (see chapter Pulling the hygiene sleeve on).
 - ▶ Check on a regular basis if the handpiece has become visibly contaminated by the return flow of bodily fluids (e.g. blood) or micropigmentation inks used. In this case, in addition to the regular disinfection, observe chapter 7.4 on page 39 and 7.2 on page 38 as well as chapter 2.2 on page 26.
 - ▶ If a local anaesthetic is used, it must be carefully removed before treatment.
 - ▶ Always hold the handpiece securely before switching it on, or place it on the handpiece tray. If an unsecured handpiece is switched on, it can move due to vibrations and injure or prick people (see chapter 6.3 on page 37).
- Injuries caused by contaminated needles or needle cartridges can lead to the transmission of diseases. Seek medical attention immediately if injured by a contaminated needle
- to prevent the intensity of treatment from exceeding the desired extent:
- ▶ Avoid risk of injury from excessive needle protrusion depth.
 - ▶ Avoid risk of injury due to too high of a piercing frequency (see chapter Setting the piercing frequency).

Please advise your clients to avoid contact with recently treated skin as much as possible. Recently treated areas of skin should also be

- protected against contamination and
- UV radiation and sunlight

and additional irritants, such as

- swimming pool or sauna visits,
- abrasive or chemical peels,
- hair removal at the treated areas or
- self-tanning lotions

should be avoided in the first two to three days after the treatment of the client.

2.4 Contraindications and side effects

The following contraindications and side effects are the result of careful analysis of clinical literature on micropigmentation. If the person responsible for treatment should have even the slightest doubt that the safety of the client to be treated cannot be guaranteed, for example due to accompanying illnesses, the treatment must be refrained from or stopped immediately.

2.4.1 Contraindications

Permanent make-up treatment is **not** permitted in case of the following conditions:

- Hemophilia or other blood coagulation disorders
- Current consumption of blood thinners (e.g. acetylsalicylic acid, heparin, aspirin, warfarin)
- Uncontrolled diabetes mellitus
- Any form of active acne in the treatment area
- Dermatoses (e.g. skin tumors, keloids or extreme inclination for keloid formation, solar keratosis, warts and/or moles) in the treatment area
- Open wounds and/or eczema and/or skin rashes in the treatment area
- Scars in the treatment area
- Systemic infections and infective diseases (e.g. hepatitis A, B, C, D E or F; HIV infection) or acute local skin infections (e.g. herpes, rosacea)

- During chemotherapy, radiotherapy or high-dosage corticosteroid therapy (recommendation: less than 4 weeks before the beginning until 4 weeks after the end of the therapy)
- Up to 12 months after plastic surgery in the treatment area
- Up to 6 months after filler injections in the treatment area
- Allergy to topical anesthetics (local anesthesia)
- Under the influence of alcohol and/or drugs
- Pregnancy and lactation

Treatment of mucous membranes and eyeballs is strictly prohibited.

The treatment must be terminated immediately in the event of:

- Excessive sensations of pain
- Fainting / dizziness

In rare cases, the following problems can be caused by the introduction of micropigmentation ink:

- Color differences
- Pigment loss

- Allergic reactions to components of the aseptic pigment color

2.4.2 Risks

In some cases, minor side effects can arise during and after the micropigmentation treatment. These side effects are listed below.

Frequent:

- Local bleeding on the surface of the freshly treated skin
- Pain and discomfort on the first day after treatment
- Temporary inflammatory reactions, erythema and/or edema up to 6 days after treatment
- Skin irritations (e.g. itching or heating), which normally wear off during the first 12 to 72 hours after treatment
- Scabbing, which normally recedes within the first 5 days
- Temporary flaking of the skin, which normally wears off within 8 days

Rare:

- Formation of blisters of the herpes-simplex virus type I (HSV-I)
- Formation of small pimples or skin eruptions if skin is not cleaned carefully before the treatment
- Hyper-pigmentation with body's own pigments, in particular on dark skin types, but which disappear completely within a few weeks
- Inflammatory reactions, haematomas, erythema and edema
- Retinoid reaction (slight irritation up to peeling of skin)
- Possible warming of the pigmented area under PET and MRT radiation

Basically, recently treated areas of skin should be protected from UV radiation and sunlight.

Furthermore, the following problems can occur with the micropigmentation introduced:

- Color differences
- Pigment loss
- Allergic reactions to components of the aseptic PMU ink

2.4.3 Required qualifications (user requirements)

The device, including its accessories, may not be used by persons with reduced physical, sensory or mental capabilities, or by children. The device, including its accessories, may not be used by persons who lack experience and knowledge, unless they are being supervised or instructed (training).

The device may only be used by people who have acquired the following knowledge:

- Qualification training for micropigmentation (permanent makeup)
- Basic knowledge of micropigmentation treatment (see chapter Setting the puncture frequency and 6.3 on page 37)
- Knowledge of the hygiene and safety regulations (see chapter 2.3 on page 26)
- Knowledge of the risks and side effects (see chapter 2.4.1 on page 28 and 2.4.2 on page 29).

2.5 Purpose, application area and intended use

2.5.1 Purpose - Indications for use

The purpose of the device is the minimally invasive penetration of sterile or aseptic PMU inks of mineral or organic origin into the dermis by means of micropigmentation (permanent make-up).

Due to the minimally invasive puncturing of the epidermis (top layer of skin) and dermis (middle layer of skin), the skin does not bear any open wounds. Therefore, the skin needs only a very short regeneration phase, in which the epithelial function is quickly restored.

2.5.2 Application area

Application examples for micropigmentation (permanent makeup) are:

- Pigmenting eyebrows
- Pigmenting eye-liners
- Pigmenting lip-liners or filling in the lip color
- Cosmetic pigmentation of the body

2.5.3 Intended use - requirements for safe use

The treatment must take place in a dry, clean and smoke-free environment, and under hygienic conditions. The device must be prepared, used and maintained in the manner described in this operating manual. In particular, the applicable provisions for workplace layout must be observed, and the materials to be used must be sterile.

Intended use also includes thoroughly reading and understanding this operating manual, in particular the chapter Important safety notices.

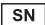
The intended use exclusively includes the following consumer groups: Adult men and women who are in good health.

Unintended use is defined as using the device or its accessories in a way other than as described in this operating manual, or failing to comply with its operating conditions. In particular, treatment of contraindications is prohibited (see chapter 2.4.1 on page 28).

2.6 Symbols on the product

The symbols described below can be found in this operating manual, on the device, its accessories or packaging:

Symbol	Meaning
	Satisfies the requirements of directives 2014/35/EC (Low-Voltage Directive) and 2014/30/EU (EMC Directive)
	Attention!
	Standby/Operation
	Handpiece
	Foot switch
	DC connection / positive inner pin
IP20	Housing provides protection against coarse dust
	Refer to operating manual!
	Protection with double or reinforced insulation according to Protection Class II
	Manufacturer

	Date of manufacture
	Catalog number, order number
	Serial number
	Lot number
	Sterilized with ethylene oxide
	Use-by date
	Temperature limit
	Humidity limit
	Protect from wetness
	Only use indoors
	Fragile
	Cannot be used if packaging is damaged
	Not suitable for re-use
	Dispose of properly as used electronic equipment

3 Scope of delivery

- 1 control device with metal plate
- 1 Power supply plug
- 1 country-specific adapter (EU)
- 1 country-specific adapter (UK)
- 1 Country-specific adapter (US)
- 1 Instruction for use

4 Product information

The Control unit artist NO.1 supplies power to the associated artist PMU handpieces and allows you to easily control of the piercing speed during use.

4.1 Technical data

Control unit artist NO.1

Piercing speed	SPEED LEVEL 1 (approx. 70 Hz)* up to SPEED LEVEL 9 (approx. 150 Hz)*
Operating mode	Continuous operation
Dimensions (W x H x D)	87 mm x 76 mm x 28 mm

Weight	Approx. 150 g
Input voltage	15 V DC
Power intake	max. 10 W
Protection class	II
IP protection class	IP20

Power supply E1165

Voltage supply	100 to 240 V AC 50 to 60 Hz
----------------	--------------------------------

* actual frequency value depending on the artist NO.1 handpiece used.

4.2 Operating conditions

Ambient temperature	+15°C to +25°C +50°F to +95°F
Relative humidity	30% to 75%
Air pressure	700 hPa to 1070 hPa

4.3 Parts and accessories

The accessories and spare parts listed below can be purchased from your local specialist retailer.

Accessories	Article description	Pieces/ packaging unit
artyst handpiece	CL100601X3 – CL100699X3, CB510601 – CB510699	1
artyst foot switch	E100601	1
artyst needle cartridge	For article descriptions of the needle cartridges, see operating manual of the artyst handpiece used	
Spare part	Article description	Pieces/ packaging unit
Control unit artyst NO.1	DE10601	1

Spare part	Article description	Pieces/ packaging unit
artyst NO.1 operating instructions	7EDE10601	1
artyst NO.1 power supply	E116503	1
artyst NO.1 counter-specific adapter	EU: E-1154-W UK: E-1155-W USA: E-1153-W	1

5 Commissioning

CAUTION

Cables constitute a trip hazard

People can stumble over inappropriately routed cables and get injured.

- ▶ Route all cables so that no-one can stumble over them or pull them accidentally.

CAUTION

Risk of injury and danger of malfunction of the device

Unsuitable accessories and spare parts can impair the function and safety of the device. The device may be damaged, fail or have faulty functions, which could endanger persons.

- ▶ Only use the needle cartridges accessories and spare parts listed in chapter 4.3 on page 32.

CAUTION

Risk of short-circuit

If there is visible damage to cables or cable connections, there is a risk of electric shock and damage to the electronics.

- ▶ Perform a visual inspection of the device and the cables for damage, e.g. a defective cable connection.
- ▶ Never bend the product cable.

NOTICE

Damage caused by condensation

If the device is exposed to significant variation in temperature, e.g. during transport, condensate may form on the inside and cause damage to electronic components.

- ▶ Ensure that the device has reached ambient temperature before using it. If it was exposed to high temperature fluctuations, wait for at least 3 hours for every 10 °C of temperature difference before putting it into operation.
- ▶ Only use the device at ambient temperatures of +15°C to +25°C (+50°F to +95°F).

5.1 Setting up or adhering the device

⚠ CAUTION

Risk of short-circuit

If there is visible damage to cables or cable connections, there is a risk of electric shock and damage to the electronics.

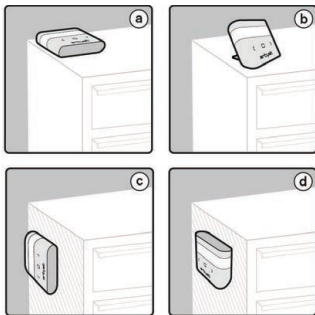
- ▶ Perform a visual inspection of the device and the cables for damage, e.g. a defective cable connection.
- ▶ Never bend the product cable.

NOTICE

Device damage due to little stability

If the device is not placed securely, safe operation is not guaranteed. The device can fall during operation and be damaged.

- ▶ Position the device on a clean, stable and level surface.
- ▶ Keep the buttons and display of the device freely accessible during operation.
- ▶ Never place the device on or under other devices.

The device can be positioned on a level surface with the supplied metal plate (b). The magnetic surface on the rear side of

the device can be used to adhere it to metallic surface (a and d), e.g. on the leg of a table or cart. The device can be adhered in a vertical arrangement or rotated 90° (c).

Positioning the device on the metal plate:

NOTICE

Device damage due to little stability

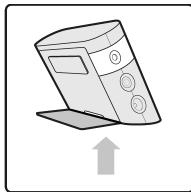
If the device is not placed securely, safe operation is not guaranteed. The device can fall during operation and be damaged.

- ▶ Position the device on a clean, stable and level surface.
- ▶ Keep the buttons and display of the device freely accessible during operation.
- ▶ Never place the device on or under other devices.

- ▶ Reach into the recess on the underside of the device and pull the metal plate off of the rear side of the device.


- ▶ Fasten the metal plate on the magnetic surface on the underside of the device. In the process, ensure that the edges of the metal plate align with the shape of the recess.

- ▶ Position the device with the metal plate on a clean, stable and level surface.

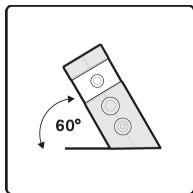The device also has a sturdy base for operation with the tilt angle.

Adhering the device with the magnetic surface:

The device can be adhered in a vertical arrangement or rotated 90°. Consequently, connected cables face downwards and can be routed more easily.

NOTICE

Damage to cables and jacks

If cables are already connected to the device, they can be kinked or damaged during assembly.

- ▶ Never kink the cable when adhering the device.
- ▶ Ensure that all jacks are freely accessible.
- ▶ Ensure that the device is disconnected from the voltage supply.
- ▶ Ensure that the metal plate is fastened on the rear side of the device.
- ▶ Adhere the device to a metallic surface with the magnetic surface.

5.2 Connecting the power supply

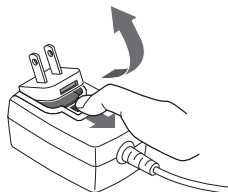
The device may only be operated with the power supply designated in the operating manual of the device. The mains

voltage must correspond to the device voltage designated on the label of the power supply.

- ▶ If a suitable country-specific adapter is not included in the scope of delivery of the device, contact your local specialist retailer (also see chapter 9 on page 39).

To remove a previous country-specific adapter, if necessary:


- ▶ Pull and hold the holder for the adapter back, while simultaneously lifting the country-specific adapter out of the recess of the power supply.

To insert the desired country-specific adapter:

- ▶ Insert the country-specific adapter in the recess of the power supply as shown below.

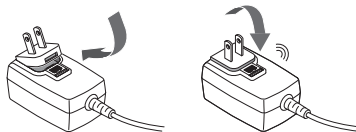- ▶ Press the adapter onto the holder until it audibly engages.

In order to establish the power supply:

- ▶ Plug the DC plug of the power supply into the jack for the power supply (⊙⊙⊙).
- ▶ Plug the power supply into the power outlet.

The device starts and the display shows the logo "artyst" for 3 seconds.

The switch to standby mode is indicated by rectangles running from left to right.

5.3 Connecting the foot switch (optional)

Only the foot switch cited in section 4.3 on page 32 will ensure safe operation.

- ▶ Plug the cinch plug of the foot switch into the jack for the foot switch (Ⓜ_{footswitch}).

5.4 Connecting the handpiece

- ▶ Insert the plug of the handpiece cable completely into the output socket on the control unit.

5.5 Cleaning and disinfecting the equipment

NOTICE

Damage caused by fluid

If cleansers or disinfectants get inside the handpiece, the control unit or the power supply, a short-circuit can occur. The electrical and mechanical components can also corrode.

- ▶ Never immerse the handpiece, control unit or power supply in cleansers or disinfectants.
- ▶ Never clean the handpiece, control unit or power supply in autoclaves or ultrasonic baths.
- ▶ Pay attention to compatibility of materials when selecting cleansers and disinfectants (see chapter 7.2 on page 38).

NOTICE

Device damage caused by non-approved cleansers or disinfectants

Cleansers or disinfectants that are not compatible with the materials of the device can damage its surface.

- ▶ Always use cleaning agents and disinfectants that are approved, i.e. that comply with stipulations in your country.
- ▶ Pay attention to compatibility of materials when selecting cleansers and disinfectants (see chapter 7.2 on page 38).
- ▶ Disconnect the control unit from the power supply.
- ▶ Wipe the control unit, handpiece cable, handpiece and handpiece tray with a soft cloth moistened with a cleanser.
- ▶ Wipe the handpiece, control unit, handpiece cable and handpiece tray with a soft cloth moistened with disinfectant.

5.6 Checking the equipment

- ▶ Conduct a visual inspection of the equipment:
 - Are there any visible signs of external damage (e.g. bends in the connecting cable, a loosened cap near the cable connection)?
 - Are there any exposed cables?
 - Are the needle cartridge and the needles correctly aligned? Are all needles completely retracted into the needle cartridge?
- ▶ Switch on the handpiece and conduct a noise test: Can you hear any unusual operating noises, or is the unit running loudly?
- ▶ If you detect anything unusual, refer to chapters 9 on page 39 and 10.1 on page 40.
- ▶ If the device is no longer operating safely, take it out of service and consult your local specialist retailer.

6 Using the device

6.1 Switching the control unit into operating mode or standby mode

To switch the control unit into operating mode:

- ▶ Ensure that the power supply is established (see chapter 5.2 on page 34).
The control unit is in standby mode.
- ▶ Switch on the control unit by briefly tapping one of the three operating buttons.
You will hear a short signal tone.
The preset piercing speed appears on the display (SPEED LEVEL 6).
The piercing speed pulses (light/dark) as long as the handpiece is switched off.

To switch the control unit into standby mode:

- ▶ Press and hold the **RUN/STOP** button for more than 1.5 seconds until a short signal tone sounds twice and the display shows the rectangles running from left to right.

If the control unit is not used for more than 30 minutes in operating mode, the control unit automatically switches to standby mode.

6.2 Setting the Piercing Speed

⚠ CAUTION

Risk of Injury due to High Piercing Speed

If the piercing speed is too high, the skin can be cut.

- ▶ Select the piercing speed depending on the skin condition of the client and suitable for the working speed.

The piercing speed is preset to **SPEED LEVEL 6**. The piercing speed can be adjusted in 9 increments from **SPEED LEVEL 1** (lowest piercing speed) to **SPEED LEVEL 9** (highest piercing speed).

To set an appropriate piercing speed:

- ▶ Ensure that control unit is switched on (see 6.1 on page 36).
- ▶ Press the **UP** button (↗) to decrease the piercing speed.
- ▶ Press the **down** button (↘) to decrease the piercing speed.

6.3 Switching the handpiece on and off

⚠ CAUTION

Risk of injury when operating the handpiece

If an unsecured handpiece is switched on, uncontrolled movement can occur due to vibrations, which can cause personal injury.

- ▶ Hold the handpiece securely before switching it on, or place it on the handpiece tray.

To switch on the handpiece:

- ▶ Ensure that the power supply is established (see chapter 5.2 on page 34).
- ▶ Ensure that control unit is switched on (see chapter 6.1 on page 36).
- ▶ Ensure that the handpiece is connected correctly (see chapter 5.4 on page 35).

- ▶ Press the **RUN/STOP** button (☐) on the control unit.

When the handpiece is on, the set piercing speed stops pulsing and the piercing speed is continuously displayed until the handpiece is switched off.

If the connected handpiece does not start:

- ▶ Observe chapter.

To switch off the handpiece:

- ▶ Press the RUN/STOP button again.

7 Cleaning and maintenance

NOTICE

Risk of short-circuit and risk of mild electric shocks

If components of the device are dismantled and cleaned when voltage is present, there is a risk of damage to the electronics.

- ▶ Disconnect the control unit from the power supply before starting all cleaning and maintenance work.

NOTICE

Damage caused by fluid

If cleansers or disinfectants get inside the handpiece, the control unit or the power supply, a short-circuit can occur. The electrical and mechanical components can also corrode.

- ▶ Never immerse the handpiece, control unit or power supply in cleansers or disinfectants.
- ▶ Never clean the handpiece, control unit or power supply in autoclaves or ultrasonic baths.
- ▶ Pay attention to compatibility of materials when selecting cleansers and disinfectants (see chapter 7.2 on page 38).

7.1 Routine maintenance

The device has **no** self-serviceable parts.

As part of the maintenance of the device, a regular inspection (biennial service) is required at intervals of 24 months. The inspection may **only** be carried out by authorized local distributors in accordance with the manufacturer's instruc-

tions. All steps required for the inspection of the device can be found on the inspection sheet included in delivery.

- ▶ Send the device at regular intervals of 24 months to your local distributor to have it inspected in accordance with the manufacturer's instructions (biennial service).

7.2 Material compatibility

- ▶ To clean the device, use a 50% aqueous 1-propanol solution or a mild soap solution, which is approved in your country.
- ▶ To disinfect the device, use one of the following disinfectants:

Manufacturer	Product	Exposure time
Antiseptics	Big Spray 'new'	1 to 5 mins
Bode Chemie	Bacillol	30 s to 1 min
Ecolab	Incidin Foam	1 to 2 mins
Schülke & Mayr	Mikrozid Liquid	1 to 2 mins

Based on the material compatibility, you can also assess which other cleansers and disinfectants are suitable.

The surfaces of the device are **resistant** to:

- Weak acids (e.g. boric acid $\leq 10\%$, acetic acid $\leq 10\%$, citric acid $\leq 10\%$)
- Aliphatic fluorocarbons (e.g. pentane, hexane)
- Ethanol
- Most inorganic salts and their aqueous solutions (e.g. sodium chloride, calcium chloride, magnesium sulfate)

The surfaces of the device are **not resistant** to:

- Strong acids (e.g. hydrochloric acid $\geq 20\%$, sulfuric acid $\geq 50\%$, nitric acid $\geq 15\%$)
- Oxidizing acids (e.g. peracetic acid)
- Lyes (e.g. caustic soda, ammonia and all substances with a pH value > 7)
- Aromatic/halogenized fluorocarbons (e.g. phenol, chloroform)
- Acetone and gasoline

7.3 Disinfecting surfaces

- ▶ Disinfect the control unit, metal plate, power supply and handpiece cable as described in chapter 5.5 on page 35.

7.4 Cleaning surfaces

In the event of external contamination:

- ▶ Wipe the device and its accessories with a soft cloth moistened with a cleanser or disinfectant.

8 Transport and storage conditions

NOTICE

Device damage caused by dropping it

If the device is dropped, it can be damaged.

- ▶ If the device falls, perform a visual inspection of the components.
- ▶ Hand over the device to a specialist retailer for inspection if it exhibits any external signs of damage or if it does not function normally.
- ▶ Always transport the device and its accessories in the original packaging.
- ▶ Fasten the metal plate of the device for transport on the rear side.
- ▶ Always store the device and accessories under the following conditions:

Ambient temperature	-10°C to +40°C +14°F to +104°F
Relative humidity	10% to 90%
Air pressure	540hPa to 1070hPa

9 Troubleshooting

9.1 Remediating problems when operating the handpiece

If a longer signal tone sounds when the handpiece is started and the message CONNECT HANDPIECE or SYSTEM OVERLOAD appears on the display:

- ▶ Ensure that a handpiece is connected to the device (see chapter 5.4 on page 35).
- ▶ Ensure that you only use artist handpieces. Otherwise, the device can be overloaded.

If a longer signal tone sounds during ongoing operation of the handpiece and the message CONNECT HANDPIECE or SYSTEM OVERLOAD appears on the display:

- ▶ Ensure that the handpiece cable for the handpiece is connected correctly.

If three consecutive longer signal tones sound and the message SYSTEM ERROR appears on the display:

- ▶ Ensure that you only use the power supply included in the scope of delivery.
- ▶ Disconnect the device from the voltage supply by pulling the power supply plug out of the socket.
- ▶ Re-connect the voltage supply.
- ▶ If this fault occurs frequently, perform all steps in chapter 9.2 on page 40.

9.2 Eliminating malfunctions of the device

- ▶ First, disconnect the device from the power supply, from the optional foot switch and from the handpiece.
- ▶ Check all connections and re-connect the components.

- ▶ Check the functions of the device once again.
- ▶ If functional problems persist, and if you have any questions or complaints, please contact the specialist retailer responsible.

10 Manufacturer declarations

10.1 Warranty

With this device, you have purchased a high-quality branded product. The reliability of this device is assured through application of the latest inspection technologies and certification. MT.DERM GmbH is certified in acc. with:

- DIN EN ISO 13485:2016 (quality management system for medical products)

For this product, the statutory warranty period applies to radio interference problems resulting from defects in material and workmanship.

We guarantee sterility for the needle cartridges with delivery of sealed and undamaged packaging. For complaints relating to needle cartridges, please advise us of the lot number printed on the label.

The warranty does not apply to the following damage:

- Accident, misuse, abuse or alteration
- Servicing by unauthorized persons
- Connection to inappropriate power sources
- Use with unauthorized accessories
- Use for anything other than indicated in this instruction for use
- Damage arising from the ingress of fluids or dirt into the inside of the device or power supply.

10.2 Declaration of Conformity

The manufacturer

MT.DERM GmbH
Gustav-Krone-Str. 3
14167 Berlin, Germany

Product:

Control unit
Product name: artyst no.1
Article description: DE10601

provisions of the following directives:

EMC directive:

2014/30/EU

Low-voltage Directive:

2014/35/EU

RoHS Directive:

2011/65/EU

The following harmonized standards were applied:

DIN EN 60335-1:2012-10

Household and similar electrical appliances – Safety – Part 1: General requirements

DIN EN 61000-6-1:2007-10

Electromagnetic compatibility (EMC) Part 6-1: Generic standards - Immunity for residential, commercial and light-industrial environment

DIN EN 61000-6-3:2011-09

Electromagnetic compatibility (EMC) Part 6-3: Generic standards - Immunity for residential, commercial and light-industrial environment

DIN EN 1037:2008-11

Safety of machinery - Prevention of unexpected start-up

DIN EN ISO 12100:2011-03

Safety of machinery - General principles for design - Risk assessment and risk reduction

DIN EN 82079-1:2013-06

Preparation of instructions for use - Structuring, content and presentation - Part 1: General principles and detailed requirements

Authorized representative for compiling the relevant technical documentation:

Dr. Andreas Pachten, MT.DERM GmbH, Documentation Officer

This declaration is issued on behalf of the manufacturer by:

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MT.DERM GmbH | Gustav-Krone-Str. 3 | 14167 Berlin