

Use as a remote control

StreamLine Mic can also be used as a remote control for your hearing instruments. You can:

- Adjust the volume by pressing **[+]** or **[-]**
- Mute the hearing instrument microphones by holding down the **[-]** button for 3 seconds to mute **OR** unmute the hearing instrument*
- Change the next hearing program by holding down the **[+]** button for approximately 3 seconds*


*This function is not available while StreamLine Mic is used as a remote microphone.

Maximum distances:

- Hearing instruments – StreamLine Mic: 6.5 ft

StreamLine Mic controls

- 1 Microphones
- 2 Power indicator
- 3 Status indicator
- 4 Multi-function button
- 5 Volume buttons **[+]** and **[-]**
- 6 USB port for charging

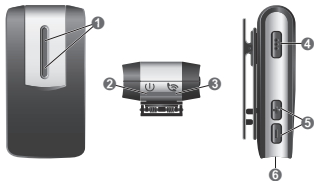

5

StreamLine Mic status and power indicator overview

Status indicator

| | Description |
|--|-----------------------------------------------------------------------------------------------------------------|
| | Flashing yellow Searching for hearing instruments. |
| | Yellow Idle mode: Connected to hearing instruments. Not connected to a smartphone or other device. |
| | Flashing blue StreamLine Mic is visible to a smartphone for pairing purpose. |
| | Blue Phone mode: Connected to a smartphone. |
| | Flashing green Incoming call. |
| | Green In a call. |
| | Orange Streaming music. |
| | Magenta Remote microphone mode. |
| | Flashing magenta Reconnecting remote microphone. |

6

Power indicator

| | Description |
|--|---------------------------------------------------------------------------|
| | Red Battery is low. |
| | Flashing green Charging while turned on. |
| | Green fades off Charging is completed and StreamLine Mic is turned on. |
| | Green Turned on, not charging. |
| | Flashing orange Charging while turned off. |
| | Orange Charging is completed and StreamLine Mic is turned off. |

Manufactured by

Sivantos, Inc.
10 Constitution Avenue
Piscataway, NJ 08854
Phone: (800) 766-4500
Fax: (732) 562-6696

Information in this brochure is subject to change without notice. Copyright © 2018 Sivantos, Inc. All rights reserved. Sivantos, Inc. is a Trademark Licensee of Siemens AG.

The *Bluetooth*® word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by Sivantos is under license. Other trademarks and trade names are those of their respective owners.

Part #10240258 2/18 1.0 22B711 8K SI/18302-18

signiausa.com

StreamLine Mic









Quick start guide



7





signiausa.com

Pairing StreamLine Mic with hearing aids



1. Ensure that StreamLine Mic is off: Hold down the  button until the power indicator  is off. This takes approximately 6 seconds.
2. Turn the hearing instruments off and on again:
Open and close the battery compartments. If your hearing instruments do not have a battery compartment, turn them off and on as described in the hearing instrument's user guide.
3. The hearing instruments are now ready for pairing. You have 3 minutes to complete the pairing.
4. Turn on StreamLine Mic: Hold down the  button until the power indicator  is green . This takes approximately 3 seconds.
5. The status indicator  flashes yellow .
6. Place both hearing instruments close to StreamLine Mic (maximum distance 8 inches).
7. The pairing process starts automatically
Pairing is finished when the status indicator is yellow .



Pairing StreamLine Mic with smart phones

1. Hold down the  and the  button simultaneously until the status indicator  flashes blue . This takes approximately 6 seconds.
StreamLine Mic is now ready for pairing. You have 3 minutes to complete the pairing.
2. Set your smartphone or music player to "search for other Bluetooth" devices". Refer to the device's manual.

1

3. On your smartphone or music player, select "StreamLine Mic" from the list of detected devices.
4. If requested, enter the PIN "0000".
Pairing is finished when the status indicator  is blue .

Making phone calls









StreamLine Mic should be worn as illustrated in the pictures for best possible performance. It is important that the microphone of StreamLine Mic is pointing towards your mouth, and that StreamLine Mic is not covered by clothes etc.

Maximum distance:

- Smartphone – StreamLine Mic: 33 ft
- Hearing instruments – StreamLine Mic: 6.5 ft



Receiving a call

You will hear a ringtone in your hearing instruments. The status indicator  flashes green .

1. To accept/end the call, press  or accept/end the call with your smartphone.
While you are in a call, the status indicator  is green .
2. To reject a call, hold down the  button for approximately 3 seconds.


2

Making a call

When you call someone with your connected smartphone, the call is automatically streamed to your hearing instruments. While you are in a call, the status indicator  is green .

This function is not available while StreamLine Mic is used as a remote microphone.

Further information

Voice dialing: When connected to a smartphone, double-press  to activate voice dialing (if supported by the smartphone).

Listening to music


If StreamLine Mic is connected to a music player through *Bluetooth*, the music will automatically be played in your hearing instruments. You do not need to operate anything on StreamLine Mic.

While music is streamed, the status indicator  is orange .

Maximum distances:


- Music player – StreamLine Mic: 33 ft
- Hearing instruments – StreamLine Mic: 6.5 ft

Starting/stopping music

- Use your smartphone or music player to start/stop the music.
To stop the music, you can also shortly press  on StreamLine Mic.

3

Use as a remote microphone

The remote microphone function does not start automatically. You have to toggle to the remote microphone mode via the  button.





It is important that the microphone is pointing towards the speaker's mouth, and that StreamLine Mic is not covered by clothes etc.

Maximum distance:

- Hearing instruments – StreamLine Mic: 66 ft

Turning remote microphone on

- Press  for approximately 3 seconds.

The status indicator  is magenta . A confirmation tone is played in your hearing instruments.

The remote microphones are turned on. Speech from the person wearing StreamLine Mic is streamed directly to your hearing instruments.

Turning remote microphone off

- Hold down the  button for approximately 3 seconds.

The status indicator  is blue  (connected to a smartphone) or yellow  (not connected to a smartphone).

4