

### Quickstart Guide

1) Select a location for your Yates Protect Y-Q5 contactless temperature scanning kiosk

A good location will be near an entrance and an electrical outlet, with enough space for a socially-distanced line to form. Please avoid direct sunlight on the display.

2) Attach Base to Pedestal

Use appropriate Allen wrench and install the ½ inch screws from the bottom of the base into the pedestal through the holes in the base (see A on right for location)

3) Install Display


Thread Display cords through square Pedestal cover (B) and washer (C) as pictured. Connect ethernet and power cables coming out of the bottom of Display (D) with ethernet and power cables in Pedestal (E)

Use included Allen wrench to secure Display to Pedestal with ½ inch screws

4) Plug included electrical cord at the bottom of the Pedestal (D) into an outlet and wait one minute

Your Yates Protect Y-Q5 is ready to start

### Included Components

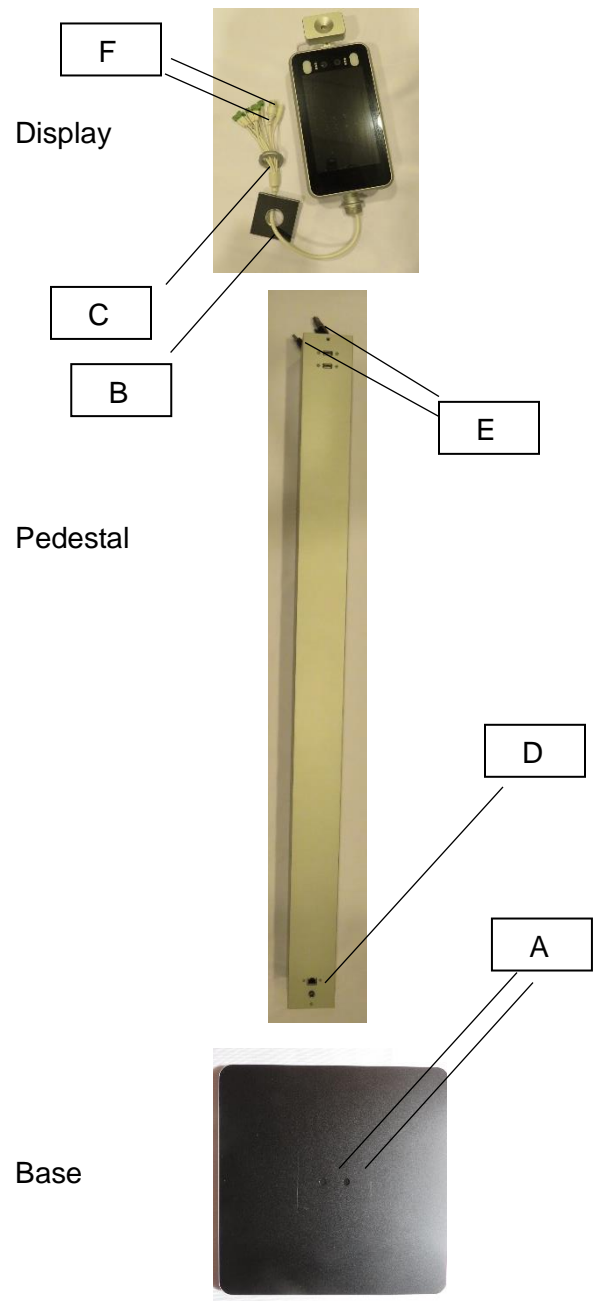

### All Other Included Equipment

Allen wrench (2)



½ inch screws



¼ inch screws



Electrical Cord

