P E R I F I T

## User Guide

support@perifit.co

- 1 English
- 22 Français
- 45 Deutsch
- 68 Italiano
- 90 Español
- 112 Nederlands

Perifit - X6 Innovations

128 rue la Boetie, 75008 Paris, France

TF-IFU-PER003-v20231108

# PERIFIT

## Perifit Pump Wearable Powered Breast Pump

### READ ALL INSTRUCTIONS BEFORE USE.

Failure to follow the instructions and safety information can lead to injury or damage to the device.

## 1. Overview

### 1.1 Indications for use and population

The Perifit breast pump is intended to express milk from lactating women in order to collect milk from their breasts. The device is intended for a single user.

### 1.2 Product Parts

#### Washable components:


1. Diaphragm (anti-backflow)
2. Connector
3. Duckbill valve
4. Flange
5. Bottle
6. Bottle cap
7. O-Ring (+1 extra in box)

#### Non-washable components:

8. Hub

#### Accessories (not pictured):

9. Bra extender strap
10. USB charging cable

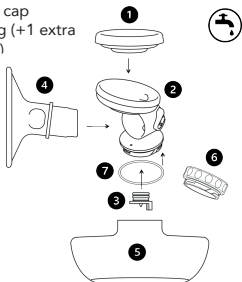## 2. Safety

### 2.1 Warnings

- If the packaging appears tampered with or damaged when you receive the breast pump, do not use and please contact Customer Support.
- If you are a mother who is infected with Hepatitis B, Hepatitis C, or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through breast milk.
- The pump should never be rented or shared. Use by more than one person may represent a health risk.
- Pregnant women should not use a breast pump, as pumping can induce contractions or premature labor.
- Do not operate or plug in the Perifit Pump near flammable materials, where aerosol products are being sprayed, or where oxygen is being administered.
- During normal use, the Perifit Pump may become warm. If the Hub or other components become unusually hot to the touch, if you notice smoke or smell something burning, stop using immediately, unplug the hub, and turn the power off.
- To protect against electric shock, injury, or fire, do not immerse the hub or USB charging cable in water. Do not operate or charge the Hub while it is wet.
- Always store milk safely immediately after pumping. See section 5, "Storing Milk Safely", for instructions.
- The device is provided with a USB charging cable that must be used with an appropriate adapter. Read the information on your charging adapter to ensure it is rated for 5V DC – 2A max.

### 2.2 Precautions

- If discomfort is felt during pumping, suction can be broken by inserting a finger between the breast and the flange. Do not attempt to pull the pump off of the nipple while suction is present.
- If irritation or discomfort occurs, or if you are not able to express milk, discontinue use and consult a medical professional.
- Before each use, visually inspect the individual components for cracks, chips, tears, mold, discoloration, or deterioration. In the event that damage to the device is observed, discontinue use until the parts have been replaced.
- Do not attempt to open the Hub or perform unauthorized repairs to the hub.
- This product contains small parts and a charging cable that may present a choking or strangulation hazard. Keep out of reach of children.
- Do not use the Perifit pump while driving, operating machinery, when sleeping or drowsy, or while lying down.

- Only use Perifit Pump parts, including the charging cable. Do not substitute or modify pump parts.
- The Perifit Pump uses Bluetooth technology. Portable or wireless equipment may affect breast pump function.
- The Perifit Pump is not intended for use in an aircraft, as the pressurized environment may impact pump performance.
- If the power cable casing or wiring becomes loose, separated, or frayed, stop use of the power cable immediately and contact the device manufacturer.

### 2.3 Contraindications

There are no known contraindications.

## 3. Cleaning and disinfection

The Perifit Pump is not provided sterile. It must be cleaned and disinfected prior to first use. Make sure all pump parts are dry before reassembly. Wet parts may affect pump performance or cause damage to the Hub.

#### Non-Washable Parts:

The Hub (8) contains electronics such as the motor and battery and is not washable. The Hub can be wiped down with a disinfectant wipe (such as isopropyl alcohol wipes). If these wipes come in contact with the breast pump parts that come into contact with breast milk, these parts should be cleaned according to the instructions below before being used for pumping. The Hub should never be put into water or other liquids for cleaning. It should also never be cleaned using a microwave sterilizer.

#### Washable Parts:

The Diaphragm (1), Flange (4), Duckbill valve (3) and O-Ring (7) are made of food-grade silicone. The Bottle (5), Bottle cap (6) and Connector (2) are made of food- safe plastic. All of these parts may be hand-washed or placed in the dishwasher, and must be washed before first use and after each use according to the instructions below. All of these parts are safe to disinfect in boiling water according to the instructions below, must be disinfected before first use and may be disinfected regularly according to the instructions below.

### 3.1 Cleaning

#### Before Use and Between Uses: Cleaning

1. Take apart and inspect pump kit. Take apart and separate all parts that come in contact with breast/breast milk (flange, duckbill valve, diaphragm with O-ring,

connector, milk collection bottle and bottle cap).

2. Rinse pump kit. Rinse all breast pump parts that come into contact with breast/breast milk under running water to remove remaining milk.

3. Clean pump kit. As soon as possible after pumping, clean pump parts that come into contact with breast/breast milk in one of the following ways.

#### To clean by hand:

1. Use a wash basin. Place pump parts in a clean wash basin used only for washing infant feeding equipment. Do not place pump parts directly in the sink, because germs in sinks or drains could contaminate the pump.
2. Add soap and water. Fill wash basin with hot water and add soap.
3. Scrub. Scrub items with hot, soapy water. If using a brush, use a clean one that is used only to clean infant feeding items.
4. Rinse. Rinse by holding items under running water, or by submerging in fresh water in a separate basin that is used only for cleaning infant feeding items.
5. Dry. Allow to air-dry thoroughly. Place pump parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.
6. Clean wash basin and bottle brush. Rinse the wash basin and brush well and allow them to air-dry after each use. Wash them every few days, either in a dishwasher with hot water and a heated drying cycle or by hand with soap and warm water. If your baby is less than 2 months old, was born prematurely, or has a weakened immune system due to illness (such as HIV) or medical treatment (such as chemotherapy for cancer), wash basin and bottle brush after every use.

#### To clean in a dishwasher:

1. Place disassembled pump parts in dishwasher. Be sure to place small items into a closed-top basket or mesh laundry bag so they don't end up in the dishwasher filter. If possible, run the dishwasher using hot water and a heated drying cycle (or disinfection setting); this can help kill more germs.
2. Remove from dishwasher. Wash your hands with soap and water before removing and storing cleaned items. If items are not completely dry, place items on a clean, unused dish towel or paper towel to air-dry thoroughly before storing. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.
3. Clean wash basin and bottle brush. If you use a wash basin or bottle brush when cleaning your pump parts, rinse them well and allow them to air-dry after each use. Consider washing them every few days, either in a dishwasher with hot water and a heated drying cycle or by hand with soap and warm water.

### Why should I air-dry infant feeding items on a clean towel instead of using a drying rack?

Air-drying infant feeding items on a clean dish towel or paper towel is probably more hygienic than using a drying rack. Drying racks may trap moisture, allow mold and germs to grow, and be difficult to clean. If you prefer to use a drying rack, use it to dry only your infant's feeding items. Every few days (or at least daily if your baby is less than 2 months old, was born prematurely, or has a weakened immune system), be sure to wash it, disinfect it, and allow it to dry thoroughly to reduce contamination

## **3.2 Disinfection**

### **Before First Use and Regularly between uses**

#### Non-Washable Parts:

The Hub cannot be disinfected.

#### Washable Parts:

For extra germ removal, disinfect washable pump parts at least once daily. Disinfection is especially important if your baby is less than 2 months old, was born prematurely, or has a weakened immune system due to illness or medical treatment (such as chemotherapy for cancer). Daily disinfection of pump parts may not be necessary for older, healthy babies, if the parts are cleaned carefully after each use. Disinfect all items (even the bottle brush and wash basin) by using one of the following options.

Note: If you use a dishwasher with hot water and a heating drying cycle (or disinfection setting) to clean breast pump parts, a separate disinfection step is not necessary.

1. Clean first. Pump parts, bottle brushes, and wash basins should be disinfected only after they have been cleaned.
2. Disinfect. Disinfect the pump kit, bottle brushes, and wash basins in boiling water as described below:
  - a. Put the pot with enough water to cover all pump parts over heat and bring to a boil.
  - b. Use tongs to place disassembled items into the pot.
  - c. Boil for 5 minutes.
  - d. Remove items with clean tongs.

Allow to air-dry thoroughly. Place disinfected pump parts, wash basin, and bottle brush on a clean, unused dish towel or paper towel in an area protected from

dirt and dust. Do not use a dish towel to rub or pat items dry because doing so may transfer germs to the items.

We do not recommend using a microwave or plug-in steam systems as extremely high temperatures may damage breast pump parts. Parts that have been placed in a steam system and subsequently damaged will not be covered under warranty.

### 3.3 Storing Pump Parts between Uses


Allow the clean pump parts, bottle brushes, and wash basins to air-dry thoroughly before storing to help prevent germs and mold from growing. Once the items are completely dry, put them back together and store them in a clean, protected area to prevent contamination.

1. Wash hands. Wash hands well with soap and water.
2. Put back together. Put together the clean, dry bottle parts.
3. Store safely. Place reassembled bottles and other feeding items, wash basin, and dry bottle brush in a clean, protected area such as inside a closed kitchen cabinet that is used only to store clean dishes.

## 4. Using your Perifit Pump


### 4.1 Assembling the breast pump

1. Wash your hands well with soap and water for 20 seconds.
2. Inspect whether any of the pump parts have become moldy or soiled during storage. If any of the pumps parts are moldy, discard and replace immediately.
3. Check that the duckbill valve is not sealed shut by pressing on the sides with your fingers. It's completely normal for it to look 'closed' at rest, but if it is sealed shut even when you press on the sides as shown, contact us or try the spare that is in your Perifit Pump accessories. Likewise, if the duckbill valve is no longer 'closed' at rest, it may need to be replaced as this will affect pump suction. Replacement parts can be ordered from the Perifit Pump website ([www.perifit.co](http://www.perifit.co)).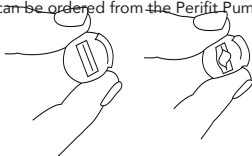


4. Fit the silicone diaphragm onto the top of the connector.


5. Insert the duckbill valve into the hole at the bottom of the connector. Make sure it is fully inserted.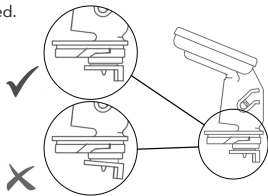


6. Insert the flange into the connector. Make sure to insert it fully (the thicker ring on the flange should be pressing against the connector without leaving any gap). Check the alignment of the flange in the connector. The small hole should face towards the top of the connector (towards the diaphragm), and the large hole should face the bottom of the connector (towards where the milk bottle will screw on), to allow for milk to flow.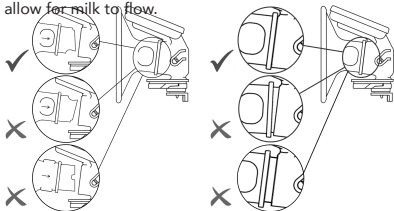


7. Make sure the O-ring is positioned around the base of the connector and that it is securely in place. If you lose it, there is a spare in the accessory bag.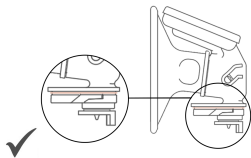


8. Screw the bottle onto the bottom of the connector.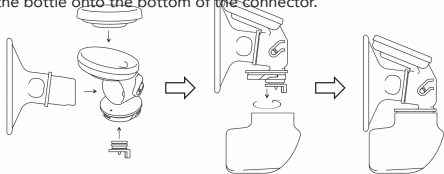

9. Slide your connector assembly into the shell. Start by lining up the circular part at the top of the connector, with the ring inside the shell. Rotate the bottom of the connector slightly towards the front of the hub. You will hear a small click!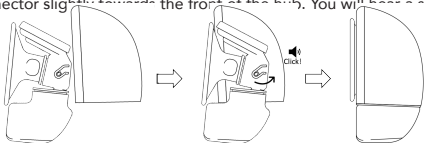

10. Check that the diaphragm is tightly pressed against the shell.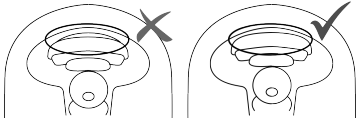

## 4.2 Taking apart the Perifit pump

1. Rotate and remove the entire Milk Assembly (Diaphragm, Connector, Duckbill valve, Flange, and Bottle) from the Hub.
2. Unscrew the Bottle from the Connector.
3. Disassemble all remaining Milk Assembly elements for cleaning. Follow cleaning instructions in this User Guide.


## 4.3 Using your Perifit

### Pump User Controls

#### Power button

Long press:  
power ON/OFF  
Short press:  
start / stop  
pumping

#### Adjust suction intensity

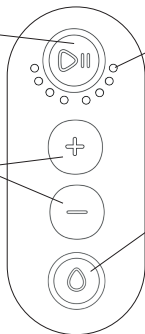#### Pumping mode and intensity level indicator

Blinking lights:  
stimulation mode  
Solid lights:  
expression mode

#### Switch between stimulation and expression modes

Perifit Pump will automatically switch into expression mode after 2 minutes of stimulation

Power button: Long press for Power On / Power Off; short press for Play / Pause. A green light indicates that the device is ready to operate. A pale orange light indicates less than 20% battery remaining. A dark orange light indicates less than 10% battery remaining. Light will blink while charging.

Level and Mode indicator LEDs: Blinking light indicates stimulation mode, solid light indicates expression mode.

Intensity level control: Press "+" to increase suction intensity.

Intensity level control: Press "-" to decrease suction intensity.

Mode button: Press to switch between Stimulation and Expression modes. The Perifit pump will automatically switch into Expression mode after two (2) minutes

of Stimulation.

### **How to Use**

1. Make sure your hands and pump parts are clean and dry before pumping.
2. Turn on pump: Press Power button (long press) to switch pump on.
3. Position the Perifit pump in your nursing bra so that your nipple is centered in the flange tunnel.

4. Use the included bra extender strap to adjust the fit of the Perifit Pump in your nursing bra. The breast pump should be pressed firmly against the breast with no gap.
5. Press Power button again (short press) to start pumping. You can switch between Stimulation and Expression modes at any time by pressing the 'Mode' button. For more information on pump cycle speeds and suction strength, see Product Specifications section. Once powered on, the Perifit pump can be controlled using the buttons on the pump, or through the Perifit Pump app.
6. Adjust intensity level to what feels comfortable for you. You may have to play with this over time to see which settings work best for you. When you are pumping, you should feel suction but not pain.
7. When you are finished pumping, short-press the Power button again to stop the pump motor.
8. Carefully remove the Perifit pump from your nursing bra. Unclip your bra and remove the Perifit pump. Long-press the power button to turn the pump off.
9. Store milk immediately according to instructions for safe milk storage (Storing and Handling Milk section).
10. Clean all milk assembly components after each use as described in the Cleaning and disinfection section.

## 4.4 Choosing your flange size

Flange size is important for comfort and milk output. Generally, the right size flange for you is the one that fits the most closely around your nipple (before pumping), without your nipple touching the sides - usually this means choosing a flange 3-4mm larger than your nipple diameter.

Flange sizing is part science, part 'art': if your nipples are very elastic you may benefit from sizing down, while some people find that they fall between sizes and prefer one size up from their measured size. You may need to experiment with different flange sizes to see what works best for you, and it is also possible to have 2 different nipple sizes.

Our flange size guide may help. Visit [www.perifit.co](http://www.perifit.co) to view and download the guide.

## 5. Storing Breast Milk Safely

Following recommended storage and preparation techniques can maintain the safety and quality of expressed breast milk for the baby's health.

### 5.1 Safe Storage of Expressed Breast Milk

These are general guidelines for storing human milk at different temperatures. Various factors affect how long human milk can be stored safely. Such factors include milk volume, room temperature when milk is expressed, temperature fluctuations in the refrigerator and freezer, and cleanliness of the environment.

Type of Breast Milk	Storage Location and Temperatures		
	Room temperature (25C / 77F) or colder	Refrigerator (4C/40F)	Freezer (-18C/0F or colder)
Freshly Expressed or Pumped	Up to 4 hours	Up to 4 days	Within 6 months is best, up to 12 months is acceptable
Thawed, Previously Frozen	1 to 2 hours	Up to 1 day (24 hours)	NEVER refreeze human milk after it has been thawed
Leftover from a Feeding (baby did not finish the bottle)	Use within 2 hours after baby is finished feeding		

### 5.2 Before expressing or handling breast milk

- Wash your hands well with soap and water. If soap and water are not available, use an alcohol-based hand sanitizer that contains at least 60% alcohol.
- Inspect the pump kit to make sure it is clean. Discard and replace moldy pump

parts immediately.

### **5.3 Storing breast milk after expressing**

- Use breast milk storage bags or clean, food- grade containers to store expressed breast milk. Make sure the containers are made of glass or plastic and have tight fitting lids.
  - Avoid bottles with the recycle symbol number 7, which indicates that the container may be made of a BPA-containing plastic.
  - Never store breast milk in disposable bottle liners or plastic bags that are not intended for storing breast milk.
  - Freshly expressed or pumped milk can be stored:
    - At room temperature (77°F or colder) for up to 4 hours.
    - In the refrigerator for up to 4 days.
    - In the freezer for about 6 months is best; up to 12 months is acceptable.
- Although freezing keeps food safe almost indefinitely, recommended storage times are important to follow for best quality.

### **5.4 Storage tips**

- Clearly label the breast milk with the date it was expressed.
- Do not store breast milk in the door of the refrigerator or freezer. This will help protect the breast milk from temperature changes from the door opening and closing.
- If you don't think you will use freshly expressed breast milk within 4 days, freeze it right away. This will help to protect the quality of the breast milk.
- When freezing breast milk:
  - Store small amounts to avoid wasting milk that might not be finished. Store in 2 to 4 ounces or the amount offered at one feeding.
  - Leave about one inch of space at the top of the container because breast milk expands as it freezes.
- If you deliver breast milk to a child care provider, clearly label the container with the child's name. Talk to your child care provider about any other requirements for labeling and storing breast milk.
- Breast milk can be stored in an insulated cooler with frozen ice packs for up to 24 hours when you are traveling. At your destination, use the milk right away, store it in the refrigerator, or freeze it.

### **5.5 Safe Thawing of Breast Milk**

- Always thaw the oldest breast milk first. Remember first in, first out. Over time, the quality of breast milk can decrease.
- There are several ways to thaw your breast milk:
  - In the refrigerator overnight.
  - Set in a container of warm or lukewarm water.

- Under lukewarm running water.
- Never thaw or heat breast milk in a microwave. Microwaving can destroy nutrients in breast milk and create hot spots, which can burn a baby's mouth.
- If you thaw breast milk in the refrigerator, use it within 24 hours. Start counting the 24 hours when the breast milk is completely thawed, not from the time when you took it out of the freezer.
- Once breast milk is brought to room temperature or warmed, use it within 2 hours.
- Never refreeze breast milk after it has thawed.

## 5.6 Feeding Expressed Breast Milk

- Breast milk does not need to be warmed. It can be served room temperature or cold.
- If you decide to warm the breast milk, here are some tips:
  - Keep the container sealed.
  - Place the sealed container into a bowl of warm water or hold it under warm, but not hot, running water for a few minutes.
  - Test the milk's temperature before feeding it to your baby by putting a few drops on your wrist.
- Do not heat breast milk directly on the stove or in the microwave.
- Swirl the breast milk to mix the fat, which may have separated. Do not shake the milk. This can make some of the milk's valuable parts break down.
- If your baby did not finish the bottle, use the leftover milk within 2 hours after the baby is finished feeding. After 2 hours, leftover breast milk should be discarded.

## 6. Battery charging and Using the App

### 6.1 Charging

The colored LEDs on the hub indicate battery life:

- A green light indicates that the device is ready to operate
- A pale orange light indicates less than 20% battery remaining
- A dark orange light indicates less than 10% battery remaining

To charge, plug USB charging cable (included) into the Pump, then into a 5V, 2A (max) USB charger (not included). The LED will blink to indicate charging.

Note that the pump cannot be used while the battery is charging. To use the pump, you must unplug it from its charger.

## 6.2 Downloading the Perifit Pump App

Search for 'Perifit Pump' in the App Store (iOS) or Google Play Store (Android).

The app is not mandatory but offers you an additional way to control the Perifit Pump, plus provides you with the ability to save your pumping history, access help content, and access additional features. The app also indicates remaining battery life and allows you to periodically update the device software, as indicated in Section 6.4 below.

To keep your data safe, we recommend you choose a secure password, keep your phone's operating system up to date, and secure your mobile device with proper controls such as anti-virus software or firewalls.

The Perifit Pump is equipped to detect various cybersecurity events, including unauthorized access attempts, data breach attempts, and firmware/software tampering.

In the event of any such incident, users will be promptly informed through the mobile app with real-time alerts. These alerts will provide clear instructions on recommended actions to ensure the security of your device and data.

Cybersecurity support for this device will continue as long as X6 Innovations is in operation. If X6 Innovations ceases its operations, cybersecurity support for the device will end. Please be aware that any continued use of the device beyond this point will be at the sole risk of the customer.

If you suspect or identify a cybersecurity issue with your Perifit Pump, please follow these steps:

- Immediately stop using the Perifit Pump and disconnect it from any network connections, including Wi-Fi or cellular data, to isolate it from potential external threats.
- Contact our support team at [support@Perifit.co](mailto:support@Perifit.co), providing detailed information about the issue you encountered. Please include any error messages or unusual behavior you observed.
- Our support team will guide you through the necessary steps to address the issue. This may include providing software updates, patches, or further instructions for remediation.

## 6.3 Connecting the breast pump to the app

1. On your smartphone, make sure that Bluetooth is enabled but do not use your phone's Bluetooth pairing function to connect to the device.
2. Launch the Perifit Pump app and follow in-app instructions.
3. Power on the breast pump. The app will automatically detect the Perifit Pump

and pair the device to the app.

4. If you are using two breast pumps, power on the second breast pump. It will also be automatically detected by the app.

You can now control device features (start or stop pumping, change modes or intensity levels) either on the device controls or through the app interface. In the event of failure to pair your Perifit Pump with the app on your smartphone, you may continue to use the Perifit Pump using the user controls on the device hub.

In the event that the app disconnects from the device during use or if you can no longer control the breast pump device with the app, you may continue to control the breast pump with the user controls on the hub, including stopping the breast pump if you feel irritation or discomfort. You may then resume controlling the pump manually.

#### **6.4 Updating the breast pump device**

Occasionally, updates will be proposed for the software that controls the breast pump ("Firmware"). When a Firmware update is available, you will receive a pop-up notification when you connect the device to the app. To update the device Firmware, make sure that the device is powered on and connected to the app. Under the "Settings" tab, if an update is available, click on "Breast pump update". The device LED will change colors to indicate that the update is in process. Do not turn off the breast pump or disconnect from the app during the update. At the end of the update, the breast pump will power off. You will then be able to power the device back on and resume normal use of the device.

#### **6.5 Wireless technology information**

The Perifit Pump connects to the smartphone app using BLE v4.2 (Bluetooth Low Energy) with an operating range of 10m. The device is configured to be able to connect to the smartphone app even in the event of low Quality of Service (latency of 1 second, data rate of 1 kbit/second), and will automatically attempt to reconnect to the app in the case of disconnection. The device has been tested for electromagnetic compatibility including immunity testing at 2.450 Mhz with no disruption to the device's operation.

No specific steps need to be taken to configure your smartphone, apart from downloading the app and enabling your bluetooth connection. Further specifications regarding electromagnetic compatibility can be found in Section 7.6.

## 7. Product Specifications

### 7.1 Technical Specifications

Vacuum range	45 - 295 mmHg
Cycle speed	Stimulation Mode: 68-85 cycles per minute Expression Mode : 28-50 cycles per minute
Power	Input: 5v DC - 2A max
Battery capacity and type	1500 mAh Lithium ion
Dimensions	138 x 120 x 74 mm
Weight	350g
Operating temperature	+5°C to 30°C (+41 to 86°F)
Bluetooth information	Bluetooth V4.2 (Bluetooth Low Energy) Range up to 10m
Essential Performance Characteristics	Max vacuum $\leq$ 300mmHg Anti-backflow diaphragm separates breast pump electronics from any contact with breast milk

The Perifit pump complies with all requirements of the relevant standards and regulations with regard to electromagnetic emission and immunity to interference. The Perifit breast pump should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the Perifit breast pump should be observed to verify normal operation in the configuration in which it will be used.

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30cm (12 inches) to any part of the Perifit Pump, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the breast pump and should be kept at a distance from the device.

#### FCC Warning

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

**NOTE:** This equipment has been tested and found to comply with the limits for a

Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

### Radiation Exposure Statement

The device has been evaluated to meet general RF exposure requirement. The device can be used in portable exposure condition without restriction.

## **7.2 Transportation, storage, and disposal**

### Transportation and Storage

Always make sure pump parts are clean and dry before transportation and storage. Charge the Perifit pump battery prior to long-term storage.

Temperature: -20 to +60oC (-4 to +140oF)

Relative humidity: 15% to 90%

Ambient pressure: 80 to 106kPa

### Disposal

If you wish to dispose of the device, log in to your device account and initiate the account deletion process. Deleting your account will permanently remove all stored data associated with it. The Perifit pump contains a rechargeable lithium ion battery. Dispose of the Hub and USB charging cable according to your local return and collection regulations for electronic equipment. All other components can be disposed of according to your local waste management requirements.

## **7.3 Use life**

The operating life of the Hub is approximately 365 hours. Inspect and change the milk-contacting parts regularly. The use life for the duckbill valve and diaphragm depend on how often the breast pump is used and is approximately 1 to 3 months. The use life of the flange, connector and bottle are approximately

6 months. Failing to change the duckbill valve and diaphragm may impact suction performance. Failing to change the diaphragm may impact the anti-backflow mechanism and damage the pump motor.

## 7.4 Troubleshooting

More topics are covered in the Perifit Pump app Help Center. affect the breast pump and should be kept at a distance from the device

Problem	Possible causes	Solutions
I am not getting enough suction	Pump parts are not fitted tightly Parts are wet Missing duckbill valve Duckbill valve needs replacing Diaphragm needs replacing	Check assembly and make sure diaphragm is tightly seated in ring on back of hub Make sure parts are dry Replace duckbill valve Replace diaphragm
My pump will not turn on	Battery is dead Pump needs to be reset	Charge pump Perform a pump reset: press and hold the Power button for 15 seconds. The hub should reset.
Too much of my nipple is getting pulled into the flange	Elastic nipple tissue Flange size too large	Try a different size flange
I am not getting enough milk output	Incorrect flange size Poor suction (see above) Body needs to adjust to new pump	Try a different size flange Check assembly (see above) Consult the FAQs in the app to access many solutions to adjust your body to a new pump
I broke one of my pump parts		Replacement parts are available on the website.
My hub is making an unusual noise	Pump parts are not fitted tightly Problem with hub	Check assembly and suction Contact customer support

## 7.5 Smartphone Requirement

The Perifit Pump currently works only with Smartphones running the Android operating system (version 6 and later) or iOS operating system (version 13 and later). Your phone should be able to tell you what version of the operating system it is running. It is also able to be used with Android tablets or iPads with the previously mentioned operating systems. Consult the instruction manual that accompanied the phone or tablet if you are unsure how to find this information. If your Smartphone was purchased after 2010, it should be compatible with the Perifit Pump system. The phone must be Bluetooth communication capable and run version 6 of the Android operating system (or later) or for iPhone, 6S or later.

The Perifit Pump utilizes BLE (Bluetooth Low Energy) for wireless communication with the associated mobile application.

If you cannot connect to the internet or in rare instances where our cloud service may be temporarily unavailable, your device will store the data locally until a stable connection is available. Once reconnected, the data will be automatically

uploaded to our secure cloud storage.

Bluetooth communication has a limited range. For best performance, place the Smartphone you are using within two meters of the Perifit Pump. For a comprehensive list of software components used in this device, you can contact our customer support team to request the full Software Bill of Materials (SBOM).

## 7.6 Electromagnetic compatibility (EMC)

The Perifit Pump complies with relevant standards for electromagnetic compatibility. However, electronic and mobile communications equipment may transmit electromagnetic energy through the air and there is no guarantee that interference will not occur in a particular usage or environment. If this occurs, it may be beneficial to temporarily turn off the Wi-Fi feature on your Smartphone while using Perifit. Interferences might create delays with the Bluetooth communication and the connection with the device. Environments for use that are suitable for the use of the device include: the home and indoor environment.

Manufacturer's declaration – Electromagnetic Emissions				
Emission test	Standard		Compliance	Electromagnetic environment -guidance
Radiated disturbance	CISPR 11:2016		Class B (30MHz to 100MHz)	The device is suitable for use in domestic, home, office, or public space environment
Manufacturer's declaration – Electromagnetic Immunity				
Immunity test	Standard	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD)	EN 61000-4- 2:2009 IEC 61000-4- 2:2008	± 2, ± 4, ± 8, ± 15 kV (air)	± 2, ± 4, ± 8, ± 15 kV	The device is suitable for use in domestic, home, office, or public space environment
Electrostatic discharge (ESD)	EN 61000-4- 2:2009 IEC 61000-4- 2:2008	± 8 kV (contact)	± 8 kV	The device is suitable for use in domestic, home, office, or public space environment
Radio-frequency, Continuous radiated disturbance	EN 61000-4- 3:2006+A1:2010 IEC 61000-4- 3:2010	10 V/m 80 MHz to 2.7 GHz	10 V/m 80 MHz to 2.7 GHz	The device is suitable for use in domestic, home, office, or public space environment
Power frequency magnetic field	EN 61000-4-8:2010 IEC 61000- 4-8:2009	30 A/m	30 A/m	The device is suitable for use in domestic, home, office, or public space environment

**7.7 Symbol Legend** Manufacturer has taken all necessary measures to ensure that the product complies with applicable safety legislation for distribution within the European Community.

Type BF applied part - Electrical classification of the device part that contacts the patient.

This device should be kept between 5°C and 30°C (41°F and 102°F).

This device should be kept between 15% and 90% humidity.

Indicates the range of atmospheric pressure to which the medical device can be safely exposed (between 80 kPa and 106 kPa).

The Perifit breast pump is an electronic product. Please refer to your country and state law for the disposal of the Perifit breast pump.

This device receives and transmits RF electromagnetic energy for the purpose of its operation.

**RoHS**

This device is compliant with the RoHS directive about hazardous substances.

**IP22**

Protected from water spray less than 15 degrees from vertical.

Indicates the Manufacturer.

Indicates the date of Manufacture.

**LOT**

Lot number.

**MD**

Indicates that this item is a medical device in certain jurisdictions.

**UDI**

Indicates the Unique Device Identifier (UDI)

**SN**

Indicates the device serial number

Consult user guide for important safety information.

**UK  
CA**

UK Conformity Assessed

## 8. Customer support

Customer support can be reached by clicking on "? " directly in the app or via email at the address [support@perifit.co](mailto:support@perifit.co). If you have any technical issue, please first refer to the Help articles of the App. If you notice any change in device performance or any malfunction, please turn off your Perifit Pump and contact us. Any serious incident that has occurred in relation to should be reported to us and your local authority having jurisdiction. If you have any complaint, please contact us. Any feedback is welcome.

### DISCLAIMER:

Neither X6 Innovations nor its retailers assume any responsibility or liability associated with or for the use of this product. X6 Innovations reserves the right to revise and/or update this manual without being obligated to notify any users. The most current version of the user guide may be downloaded from our website ([www.perifit.co](http://www.perifit.co))

## 9. Warranty

The Perifit Pump is backed by a limited manufacturer's warranty against any defects in materials and/or workmanship: the Hub for twelve (12) months from the date of Pump purchase. The duration of this Warranty may be extended as a function of applicable local law.

Washable parts (bottle, connector, duckbill valve, diaphragm, flange) are covered against any defects in materials and/or workmanship for ninety (90) days from date of purchase. Warranty on washable parts will be voided if they have been exposed to a steam sterilization system.

This warranty does not cover cosmetic deterioration nor damage caused by accident, misuse, normal wear and tear, improper use, unauthorized maintenance or repair, or failure to follow written instructions for use.

This warranty terminates if you sell or otherwise transfer this product to another person. The warranty does not cover purchases from non-authorized resellers.

Defects discovered during the warranty period should be reported to X6 Innovations, SAS. Any warranty claims must be supported by proof of date of purchase. In the event that X6 Innovations provides you with a replacement product, this does not extend the duration of the warranty.

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**Modo de uso**

1. Make sure your hands and pump parts are clean and dry before pumping.
2. Turn on pump: Press Power button (long press) to switch pump on.
3. Position the Perifit pump in your nursing bra so that your nipple is centered in the flange tunnel.
4. Use the included bra extender strap to adjust the fit of the Perifit Pump in your nursing bra. The breast pump should be pressed firmly against the breast with no gap.
5. Press Power button: again (short press) to start pumping. You can switch between Stimulation and Expression modes at any time by pressing the 'Mode' button. For more information on pump cycle speeds and suction strength, see Product Specifications section. Once powered on, the Perifit pump can be controlled using the buttons on the pump, or through the Perifit Pump app.

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