

RemovableTM Veneers USA

www.RemovableVeneersUSA.com

STEP 1

Watch the demonstration video here: <https://bit.ly/3p6tQW5>

You may scan the QR code to the right with your camera on your phone as well. After watching the video, start with brushing your teeth thoroughly and rinsing your mouth. With a tissue wipe your mouth completely dry.



STEP 2

Fit the different impression trays on your teeth, until you decide which size to use (Small #5 or Medium #6).

The tray should overlap all your teeth and surrounding gums. Make sure the walls of the tray are not hitting the side of your teeth or gums at any point, or on the other hand being too wide.



STEP 3

Get the impression material out of container 1 and container 2. Mix them quickly by kneading and folding for 30-40 seconds or until it becomes a solid one color without any marbling.




STEP 4

Use the palm of your hands to roll the material and make it in the shape of a sausage link about as tall as your finger, then load the tray evenly and fully with it. The rolling and loading shouldn't take more than 50 seconds.



STEP 5

Before you insert the tray into your mouth try to swallow all of the saliva to make sure your mouth is as dry as possible. Wipe your mouth completely dry with a tissue one more time.

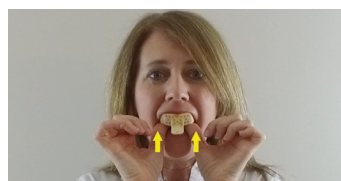

STEP 6

You're going to be taking an impression of your top teeth first. Insert the tray inside your mouth. Place it fully inside your mouth before touching your teeth. Try to aim your teeth into the middle of the material. In other words, not to close to the front of the tray because you don't want your teeth to scrape the inside of the tray. ****Tip - The goal is to make sure your teeth are not touching any part of the inside walls of the tray.**

STEP 7

Push the tray all the way against your teeth in a straight line without rocking or wiggling. **Push the tray so hard, all the way, until you can't push anymore.** The tray should be fully surrounding your teeth and gums.

STEP 8



Use your fingers to gently press around the borders of the tray only for a few seconds to make a perfect detailed impression. *Tip – this helps to create a gum line inside of the impression. Keep the impression perfectly still using your thumbs without moving or rocking for 4 minutes. We suggest sitting at a table and resting your elbows on the table while still applying pressure with your thumbs. After 4 minutes has elapsed, take the tray out by pulling down in a straight line. You will have to apply a little pressure for this step. Congratulations, you just made your first impression.

STEP 9

Once you complete a top impression, at that point please text your smile consultant several pictures at different angles so that we can review the impression for you. You will stop here and not move forward with any additional impressions until you hear back from your Smile Consultant. If we approve the top impressions, you will proceed to the bottom impression and repeat the process. If we don't approve the impressions, don't worry, we've sent you enough material and supplies to do another set if needed. We supply the most material in your initial Impression kit in the snap on smile industry. 90% of our customers complete impressions with the initial kit as long as they have followed all written directions along with specific detailed instructions from their smile consultant. If you are unable to achieve approved impressions with all the material in the initial kit that has been sent to you, you will be required to purchase a 2nd impression kit from our website. In case you need to purchase a second impression kit, visit the link: <https://removableveneersusa.com/products/impression-kit>. Please note and remember that Perfect Impressions = Perfect Fitting Veneers. Your veneers are only as good as your impressions.

STEP 10

Once you have your assigned smile consultant's impression approval, place the impressions in the zip lock bag and seal. There is also a customer information card, and terms & conditions contract that will need to be completed, signed, and placed back in the box along with the impressions. You will also want to ship back any materials or trays that you did not use. Place the box in the return shipping bag, seal the bag up and drop off at any local US Post office. The package will be postage paid.

Below are images of accepted and rejected impressions. These images will show you what we are looking for with a final impression. Rejected impressions are impressions that have failed inspection and would need to be re-taken.



Good Impressions

Well demarcated teeth borders.
Good separation lines between teeth.

Rejected Impressions

Distorted impressions showing walls of the tray. Showing dragging lines because the material was too soft when pulled from the mouth. Too shallow of impression because the material was too hard before inserting into mouth.

In most cases, it's very important to have photos of the customers mouth and teeth to develop the most suitable and matching veneers. Please take a few minutes to take those photos and text the pictures to your smile consultant.

