


Quick guide

Vivitone

How To Put On




1. Clean the ear canal. Push the silicone ear dome into the canal.

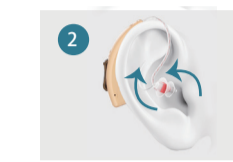


2. Gently turn the silicone ear dome left and right until it is stable and comfortable.

Wearing

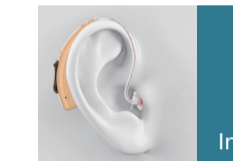


Incorrect

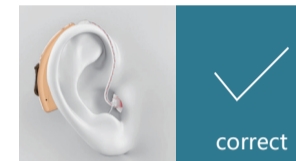

Correct

- Attach the hearing device behind your ear.
- Gently insert the canal portion of the hearing device into your ear.

Functions Overview

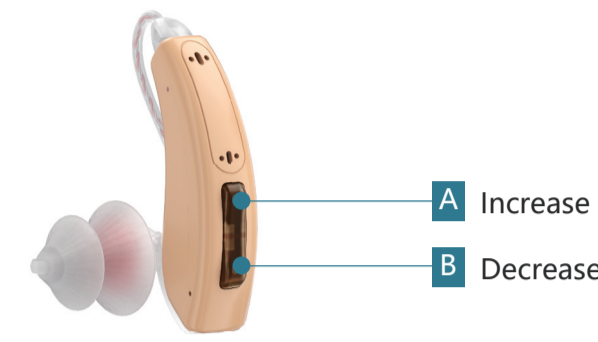

- A**
1. Hold down for 3 seconds to switch mode.
 2. Press long for 5 seconds to switch on and off.
 3. Quick press to increase sound volume, "Di Di Di"=maximum

- B**
1. Hold down for 3 seconds to switch mode.
 2. Press long for 5 seconds to switch on and off.
 3. Quick press to decrease sound volume. "Du Du Du"=minimum

Switch Mode



Press and hold for 3 seconds to switch mode.

You will hear 1 to 3 "Du" sounds to indicate which mode you are in (for example, "Du Du Du" means mode 3).

Mode 1:

"Du" sound, indicates Normal Listening Mode. For most daily communication and watching TV.

Mode 2:

"Du..Du" sound, indicates Noisy Environment Mode. For party, in plane or other noisy situations.

Mode 3:


"Du..Du..Du" sound, indicates Tinnitus Masking Mode. For tinnitus sufferers or those who want to reduce sensitivity to surrounding sounds.

On & Off

- **Turn on:** take the amplifier out of its case, or press and hold the button for 5 seconds.
- **Turn off:** put the amplifier back into its case, or press and hold the button for 5 seconds.

Charging

Amplifier charging indicating

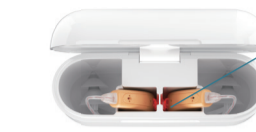

Red: charging
No light: fully charged

Charging

- **Charging time:** 2 hours for both charging case and devices
- **Working time:** 30 hours for continuous use
- **Charging case backup power:** 120 hours for a pair

Charging case power indicating

Only a single blinking white : low power
Solid white: normal power range




(A single white light indicates 25% power)

Cleaning & maintenance



1. Clean the silicone ear dome with the sound hole facing down.

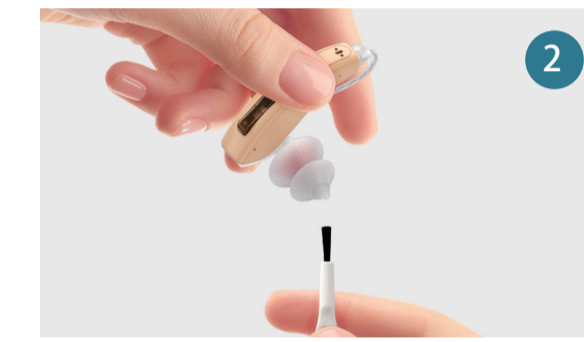

2. Clean the dustscreen with the cleaning brush. Remove the silicone ear dome if needed.

Cleaning & maintenance



3. Clean the dustscreens and frequently or replace it time by time.



4. Put the devices into the charging case.