
两折页:180*90mm 材质:铜板纸125克 工艺:双面印刷覆膜

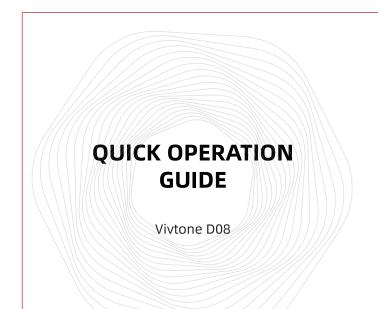

ON / OFF

- Power ON: Take the device out of the charging box.
- Power OFF: Put the device into the charging box.



VOLUME ADJUSTMENT

- Turn clockwise to increase the volume
- Turn counterclockwise to decrease the volume



HOW TO WEAR

Insert the hearing aid into the ear canal comfortably. Rotate the hearing aid to snap it into your ear and wear it comfortably and set the volume to an appropriate level.



ON / OFF

The charging box has a power storage function, and the hearing aid can be charged after being inserted;

- The digital indication represents the charging status of the charging case
- 2) The green light flashes when the hearing aid is charging, and turns off automatically when fully charged.



	Hearing Aid	Charging Case
Charging Time	≤2H	≤2H
Rated Working Time	≥30H	2 Times for Two Units