
Vivtone®

RIC02

Hearing Aid

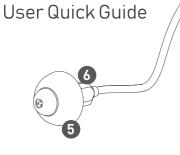

- Volume +
- 3 Volume -
- A LED Indicator Indicate power on/off and charging status
- Dome
- Receiver
- Receiver Tube

Programs (Modes)

- 1.TV watching
- 2. Noise reduction (Dual mic)
- 3.Normal
- 4. Outdoor (Dual mic)

Power On/Off

Press and hold for 3 seconds to turn on and off the hearing aid.

Program Switch

When it's on, press the button briefly to cycle between the 4 pre-set amplification programs. You will hear 1 to 4 beeps to indicate which program you are in (for example, 3 beeps mean program 3].



0

Turn on briefly green



Turn off briefly blue



Charging




Full charged within 1.5 hours for 20+ hours use

Assembly

Align dome opening and press over the grooves of sound tube, secure proper fitting.





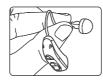



Round top dome (Power upgrade)
Effectively reduce the feedback/whistling sound



Magic dome

Effectively reduce occlusion effect and whistling sound

Wearing

- 1. Place the hearing device over the top of your ear.
- 2. Insert the dome with gentle pressure into your ear.

NOTE:

The different colors clothing on the receiver indicate different ear use. The Red color is for Right ear while the Blue color is for Left ear.



Trouble Shooting-Whistling

Whistling occurs when the amplified sound returns to themicrophone causing a feedback loop.

To stop whistling:

- 1. Simply reduce the volume. Volume level 3 is recommended for the first time users.
- 2. Choose the proper sized dome to cover the ear canal so the amplified sound won't come out. Use the semisphere dome for the best performance.

Trouble Shooting - Weak or no Sound

- 1. Increase the volume.
- 2. IMPORTANT! Check if the dome or the sound tube is congested by ear wax (very common]. Remove the debris from the dome and tube using the cleaning tool (comes within the accessories box].

Any question, please contact:



support@vivtone.com

www.vivtone.com