Lansinoh.

# Ihr Lansinoh® Pumpset

## Lansinoh® Pumpset

## Lansinoh® Pompa Seti

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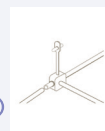
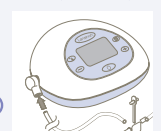
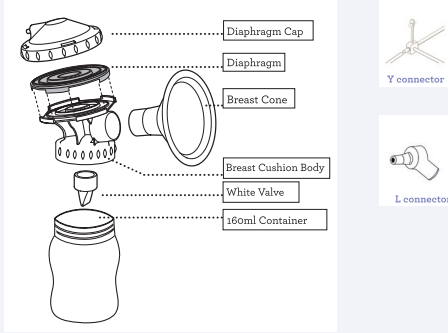
c)

**(GB) Your Lansinoh® Pumpset**

Please note these instructions will show you how to use this pumpset as a single pumpset for both the Lansinoh® Single Electric Breastpump as well as using it as part of a double pumpset for the Lansinoh® 2in1 Electric Breastpump.

- Apart from the tubing, Y connector and L connector, please take everything apart, and boil them for 10 minutes to sterilize.  
(Lansinoh pumps are a closed system design, therefore, the tubing must not be sterilized.)
- Always make sure all parts are clean and completely dry before assembly.
- Always wash your hands before assembly.
- All parts, apart from the tubing and connectors (Y & L) are dishwasher safe.  
*Please note: valve and diaphragm are made from silicone and thus need to be treated carefully.*  
*We recommend washing these parts by hand in warm soapy water.*

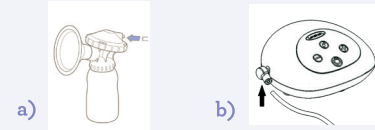
**1. Assembly**

(Valid for both the Lansinoh® Single Electric Breastpump as well as the Lansinoh® 2in1 Electric Breastpump.)

1. Place purple diaphragm on top of breast cushion body as shown in the diagram.
2. Then place and twist the diaphragm cap onto breast cushion body enclosing the purple diaphragm.
3. Push ComfortFit™ breast cone into breast cushion body.
4. Gently secure white valve into the bottom of breast cushion body.  
*Note: Use caution when handling and cleaning the small white valve.*
5. Screw collection container onto breast cushion body.
6. Insert tubing into back of diaphragm cap; ensure tight fit.
7. Insert tubing connector between the sets. Place tubing input into the pump.

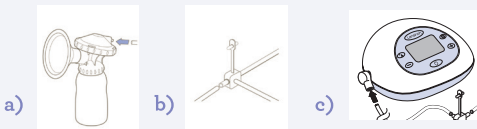
**2. Assembling the tubing for the Lansinoh® Single Electric Breastpump**

- a) Please insert the long tubing (600mm) into the rear of the diaphragm cap.
- b) Please insert the L-shaped pump connector into the pump motor and connect the other end of the long tube (600mm) to it.

**3. Assembling the tubing for the Lansinoh® 2in1 Electric Breastpump**

You will need two short tubes (300mm), one long tube (600mm), the L-shaped pump connector and the Y-connector

- a) Please place one short tube (300mm) into the each of the rear of the diaphragm cap.
- b) Connect both short tubes (300mm) and the long tube (600mm) using the Y-connector
- c) Please insert the L-shaped pump connector into the pump motor and connect the other end of the long tube (600mm) to it.

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