



koala
babycare


koala perfect head breastfeeding

instruction manual

1.



2-3.

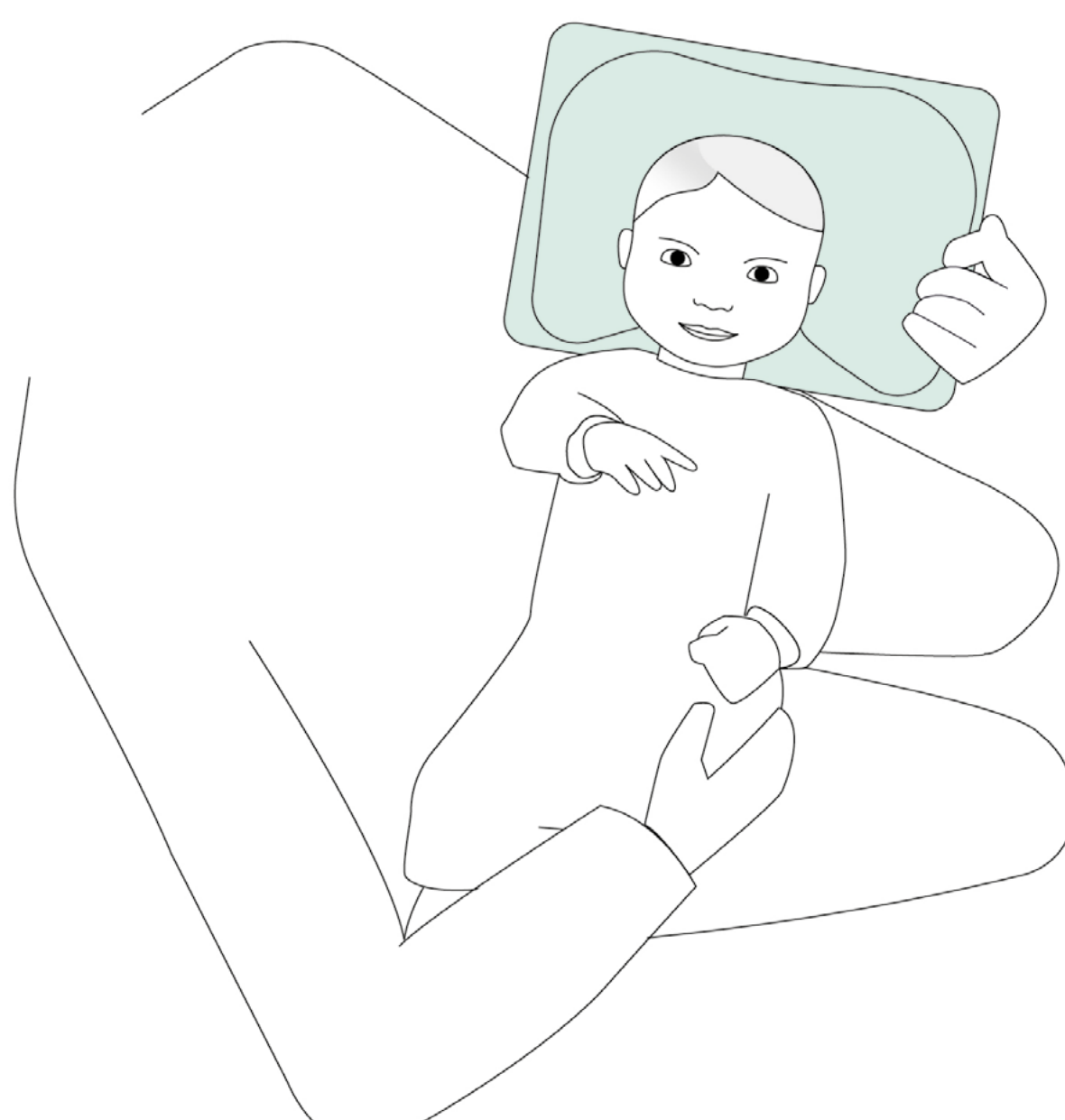

4.



5.



6.



7.



How to use Koala Perfect Head Breastfeeding ?

Koala Perfect Head Breastfeeding is a Class 1 medical device designed to protect your baby's head whilst breastfeeding or being held.

- 1. Insert your arm through the opening** of the pillow and slide it up to your elbow or to the position where you would normally rest your baby's head.
- 2. Place your baby's head in the central indent:** pressure will be distributed evenly over the whole skull area.
- 3. Baby's shoulders must remain off the pillow to help maintain** correct posture.
- 4.** Use the pillow each time you breastfeed, cradle, or hold your baby
- 5.** Alternate the arm you use to support your baby's head
- 6.** Along with using Koala Perfect Head Breastfeeding, vary your breastfeeding position. Help yourself with the Koala Hug Baby nursing pillow which allows you to comfortably assume the most suitable positions during breastfeeding.

Other tips to prevent Plagiocephaly whilst breastfeeding

- 7.** Choose positions where their head is not resting on a support

What can Koala Perfect Head Breastfeeding help you with ?

- Protecting your baby's head whilst breastfeeding or bottle-feeding.
- After feeding, it is only normal to cradle your baby or gently rock them to sleep in your arms. In such case, we tend to use the same arm (the one we prefer due to comfort or strength), and, consequently, we keep the baby in the same position for a long period of time. By using Koala Perfect Head Breastfeeding you can continue to hold your baby without the worry of your baby's head being affected.
- Koala Perfect Head Breastfeeding is particularly recommended for mums who have developed mastitis whilst breastfeeding and who, therefore, prefer to hold their baby in a supine position when breastfeeding.

How to wash Koala Perfect Head Breastfeeding ?

The outer pillowcase is removable and machine washable at 30°.

Pillowcase washing guidelines:

- If the pillowcase is particularly dirty, it can also be washed at 40°. However, washing at this temperature cannot guarantee the pillowcase the same durability as cotton.
- We recommend that you do not bleach, do not iron, do not dry clean and do not use stain removers with solvents.

Do not wash the memory foam inner cushion. If immersed in water, the memory foam will absorb the liquid, resulting in it losing its shape and becoming unusable.

What is Koala Perfect Head Breastfeeding made of ?

Pillowcase: 75% Cotton, 25% Polyester

Pillow: 100% Memory Foam



koalababycare.com

help@koalababycare.com