




koala
babycare


koala perfect head maxi

manuale d'uso


1.


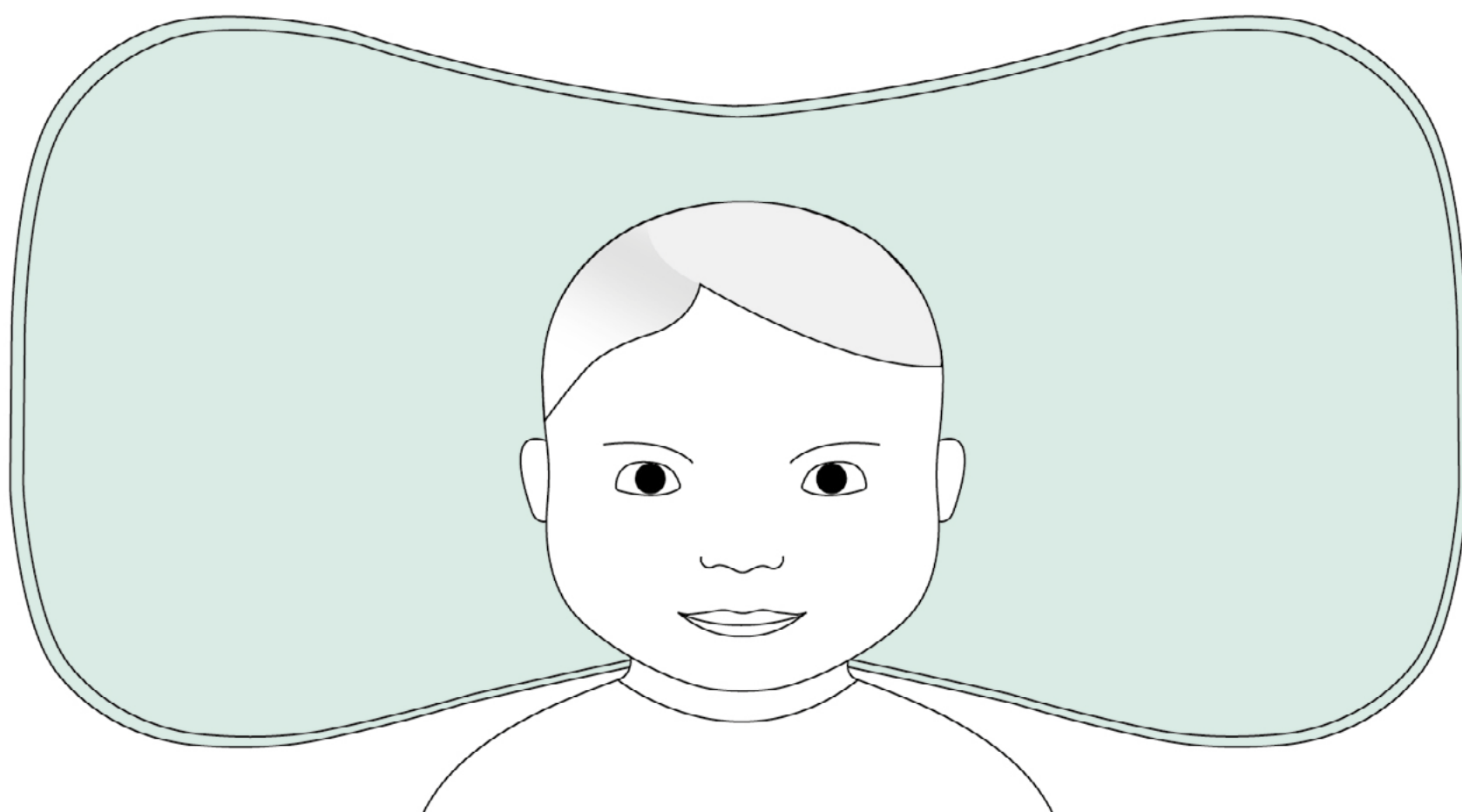


2.

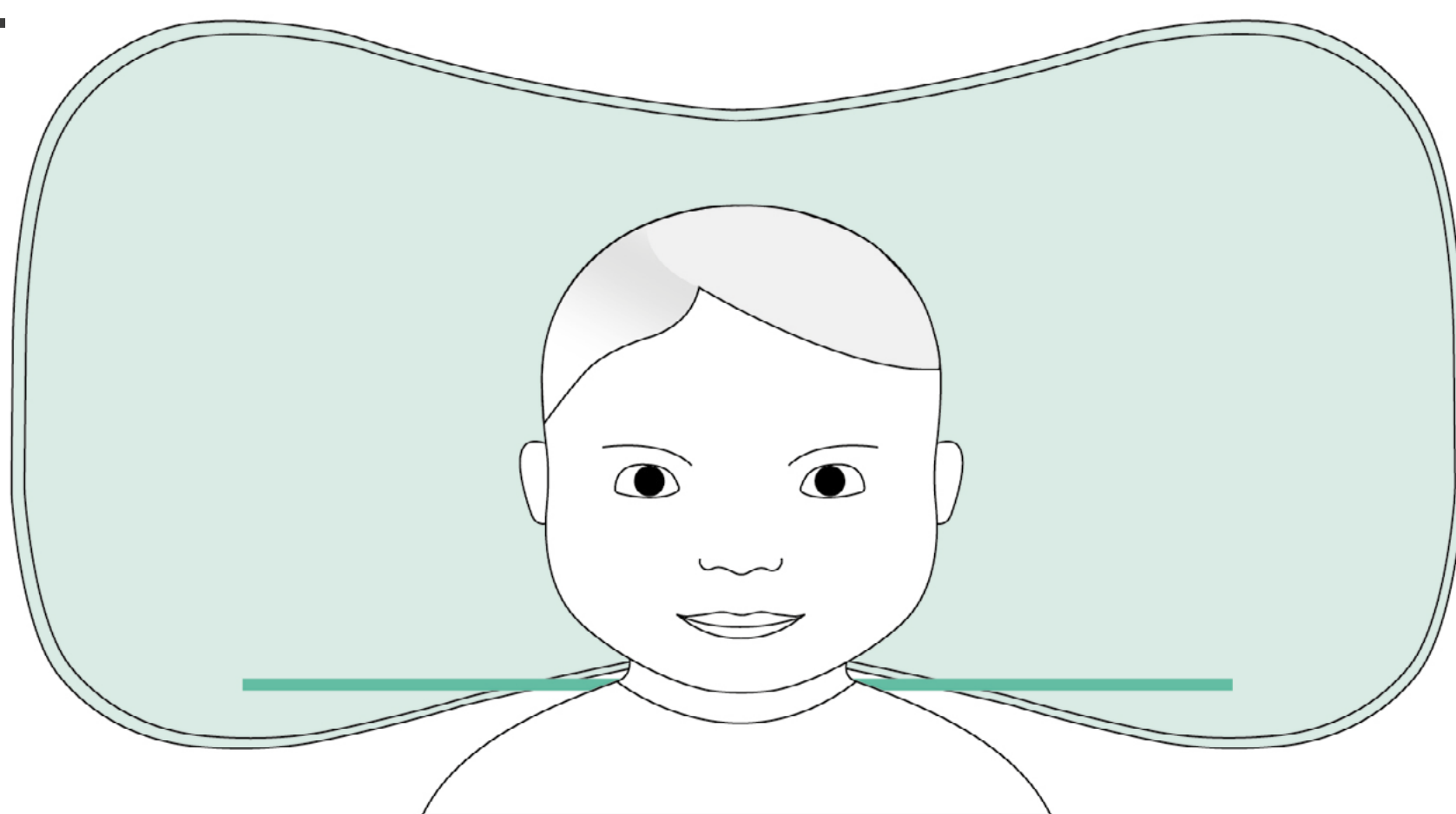

3.

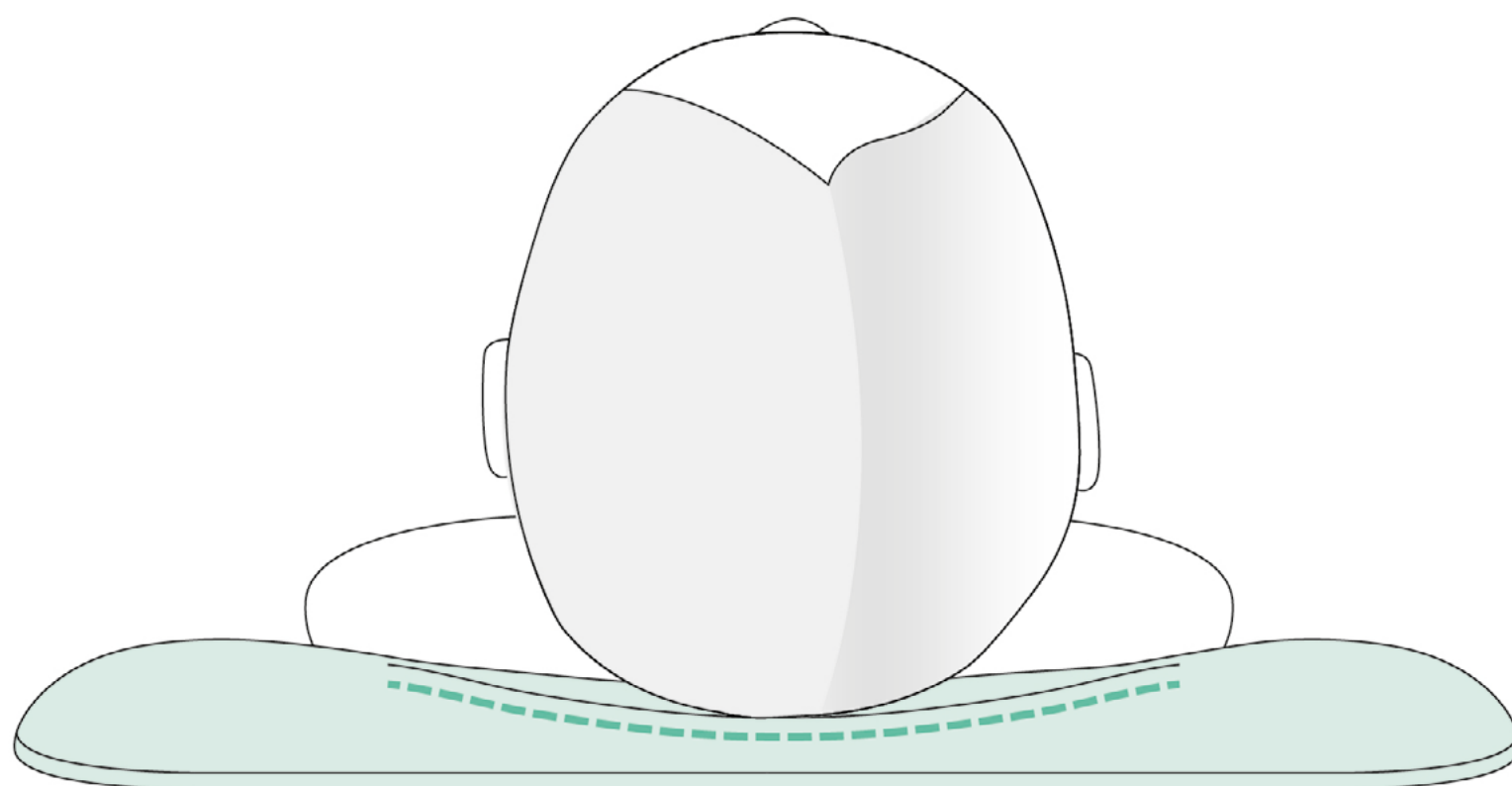

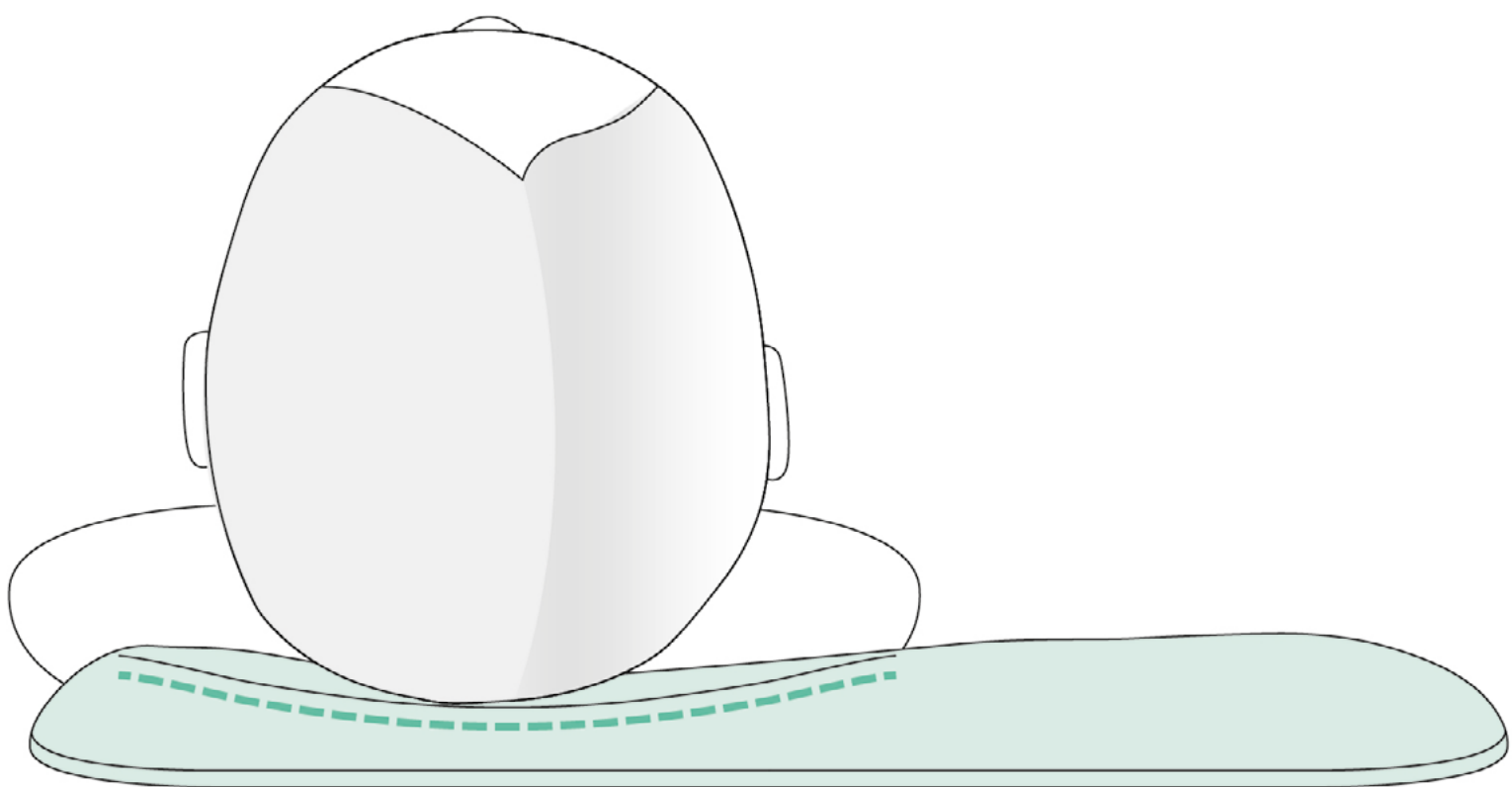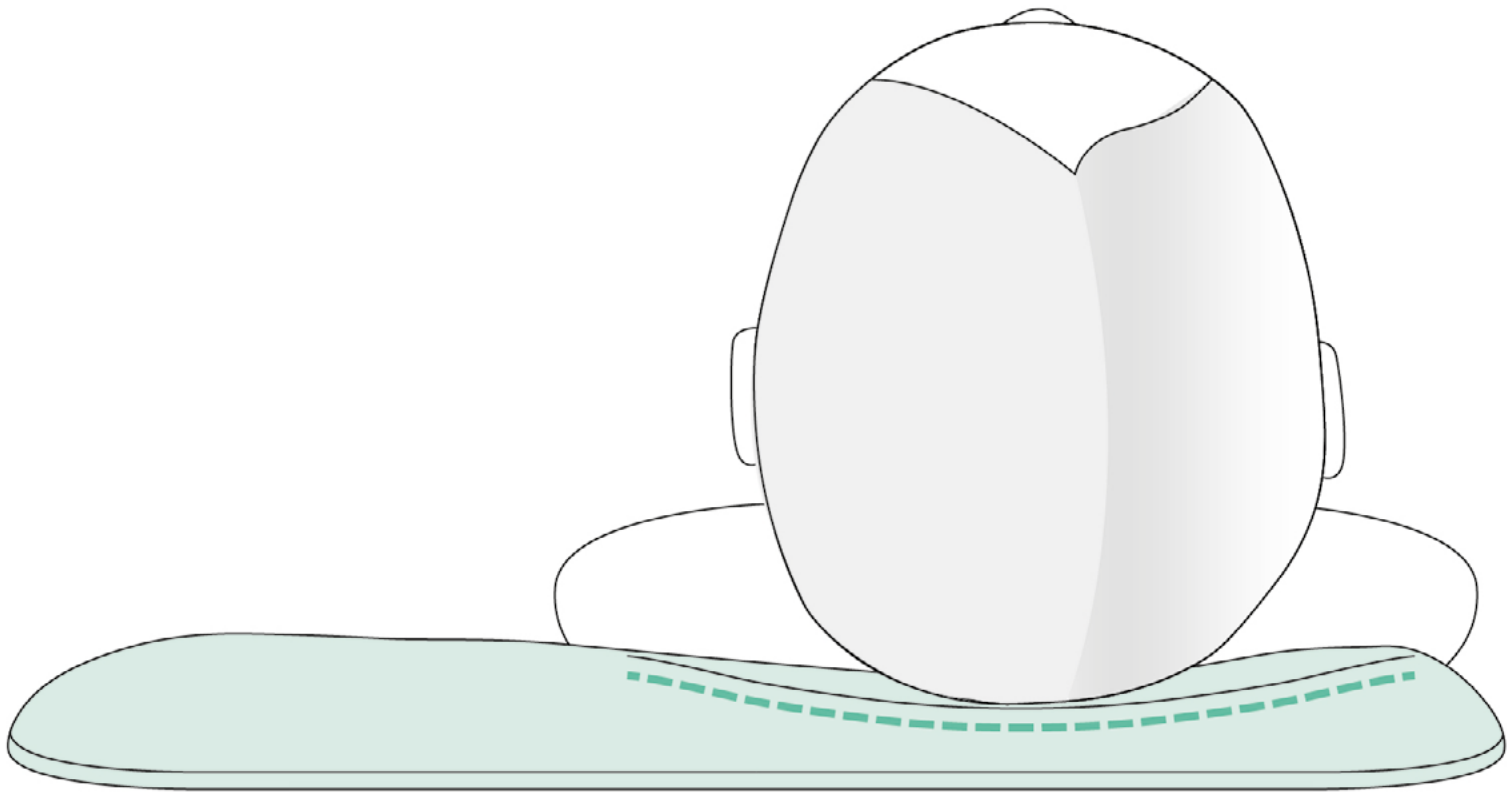

How to use Koala Perfect Head Maxi ?

Koala Perfect Head Maxi is not just a simple baby pillow, but a **Class 1 medical device for the prevention and treatment of Plagiocephaly (commonly known as Flat Head Syndrome) and Brachycephaly.**

Koala Perfect Head Maxi is specially designed for use in co-sleeping cribs, cots, and beds. It can also be used on play mats and baby gyms.

It can be used up to 36 months old. It is ideal both for newborns and as babies grow and start to move around more. In fact, its size (50 cm) allows babies to move freely sideways without their head coming off the pillow, combining prevention of plagiocephaly and brachycephaly with physiological psychomotor development

- 1. Always place your baby in supine position** (on their backs facing upwards) until they are 12 months old.
- 2. Position the back of your infant's head in the central indent:** the cushion is designed to ensure maximum comfort for babies, providing them with freedom to move around as they grow. At the same time, it also allows for optimal distribution of pressure across the skull even on the areas surrounding the central indent.
- 3. Make sure your baby's shoulders are not on the pillow:** this ensures the child maintains good back and neck posture and helps keep their head centrally positioned on the indent.

By following these three simple steps correctly, the memory foam pillow will distribute pressure from your baby's head over a large area, reducing localised pressure which is the cause of plagiocephaly (flat head syndrome).

The **pillow's slim design** is designed to ensure good head and back posture and prevent a baby's neck from tilting forward towards their chest, keeping their airways clear at all times.

We recommend that Koala Perfect Head Maxi is used under adult supervision until your baby is able to roll over on their own, in accordance with WHO guidelines.

How to wash Koala Perfect Head Maxi ?

The outer pillowcase is removable and machine washable at 30°.

Pillowcase washing guidelines:

- If the pillowcase is particularly dirty, it can also be washed at 40°. However, washing at this temperature cannot guarantee the pillowcase the same durability as cotton.
- We recommend that you do not bleach, do not iron, do not dry clean and do not use stain removers with solvents.

Do not wash the memory foam inner cushion. If immersed in water, the memory foam will absorb the liquid, resulting in it losing its shape and becoming unusable.

What is Koala Perfect Head Maxi made of ?

Pillowcase: 75% Cotton, 25% Polyester

Pillow: 100% Memory Foam



koalababycare.com

help@koalababycare.com