



koala  
babycare

# koala silver cups

instruction manual

## Koala Silver Cups

Breastfeeding is one of the most precious experiences a mother can have. Unfortunately, it can also bring with it a few challenges, as well as pain, which may spoil what should really be a wonderful time. One of the most common difficulties is sore, cracked nipples, caused by friction from the baby's palate and their tongue. Koala Silver Cups will help prevent and treat this condition, protecting the breasts from external agents and bacteria, and relieving pain within just 24/48 hours.

## Warnings and safety informations

### Warning

- If you are allergic to silver or if you experience any type of allergic reaction, discontinue use immediately.
- For use on the nipple area only.
- Follow the below guidelines on how to use, wash and store the product. Improper use or storage could impair the properties of the product.
- The product does not contain NICKEL.

## User Instructions

- Prior to use, wash the cups with cold water and neutral and odourless soap, rinsing and drying them with a clean cloth.
- Insert the cups between the breasts and the bra and place them directly over each nipple.
- When you are ready to breastfeed, gently remove the nipple cups.
- When breastfeeding has finished, apply a drop of milk or water inside the cups, this will help them to adhere well to each breast.

## **Koala Babycare recommendations**

- We recommend you start to use Koala Silver Cups 7-10 days before giving birth as a preventative measure.
- We always recommend drying them with a dry cloth to avoid oxidation of the silver (tarnishing).
- If breasts are leaking significant amounts of milk, breast pads may be used between Koala Silver Cups and the bra.
- If tarnishing of the silver occurs, wash the cups by immersing them in cold water and baking soda (50 g per litre) until the stain has gone. If staining persists, email us at [help@koalababycare.com](mailto:help@koalababycare.com) and we will find a solution together.

## **Benefits of using Koala Silver Cups**

### **Why should you use Koala Silver Cups?**

- Silver has been used in medicine for more than a century to treat various types of skin diseases, it has, in fact, been found to release substances which perform an antibacterial and healing action in the presence of moisture.
- Their shape allows them to fit perfectly on the breast and to protect the nipples from any friction from clothing or external agents.
- The obvious savings in both financial and environmental terms. Once opened, if stored properly, they can be reused for future pregnancies by the same mum, so you don't have to purchase other products, such as creams or single use disposable nipple shields.


## **Storage**

- After washing, place the product back inside the original packaging. Store the package in a cool, dry, and clean place, away from heat sources or direct sunlight.
- We recommend you keep the item inside the original packaging whilst in use. Alternatively, if the packaging has been lost or thrown out, you can keep your nipple shields wrapped inside a clean cloth to avoid contact with air and exposure to light which can cause oxidation.

# Expiry

If you follow these storage guidelines, the product will not have an expiry date.



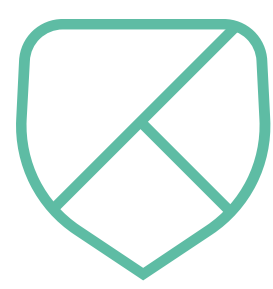

koala  
babycare

[koalababycare.com](https://koalababycare.com)

[help@koalababycare.com](mailto:help@koalababycare.com)