XCEL TROCARES

DESCRIPCION

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The following evaluations were performed:

Table 1. Criteria Evaluations

Function/Part	Criteria	Yes or No	Comments
Packaging / Package Protector	Packaging is sealed.	All Yes	None
	Sleeve and obturator remain assembled to package protector when transferred to mayo stand.	All Yes	None
Obturator Latches	Obturator latches to sleeve housing and remains latched until the buttons are depressed to unlatch.	All Yes	None
Trocar Insertion Performance	Trocar inserted in a comfortable/ predictable manner.	All Yes	None
Seal Function	Allowed for insertion of a gastric band without damage to the seal by the band or damage to the band by the seal.	All Yes	None
	Allowed for removal of a gastric band without damage to the seal by the band or damage to the band by the seal.	All Yes	On two occasions, the buckle of the bands caught on the edge of the cannula impeding withdraw of the band.
System Compatibility	The device accommodates commonly used laparoscopic instruments.	All Yes	None
Fixation (retention)	Sleeve stays where it is placed and does not slip in or out during use.	All Yes	None
Overall Strength/Integrity of Device	No component breaks or malfunctions during insertion/removal	All Yes	None

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