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INSTRUCTIONS FOR USE
Cooling care bandage

Aktimed HAUT is a cooling, moist care bandage from the group of zinc paste bandages. The formula contains, among other ingredients, a hydrocolloid gel in which calcium alginate and cardiospermum halicacabum 10⁶ are included. The bandage's main physical/mechanical effect is supported by this formula. The formula is free of fats such as vaseline and of paraffins and is hypoallergenic as well as latex-free.

INSTRUCTIONS FOR USE

- Open wounds/skin areas must first be covered with a sterile wound dressing.
- The bandage should be applied without, or with only slight tension. With approx. 3 to 4 layers on top of each other, the cooling effect lasts for approx. 3 to 4 hours.
- A protective bandage on top of the dressing protects against clothes getting soiled.
- The cooling effect can be extended by moistening the bandage. The bandage can absorb moisture (e.g. sweat) and release it through the surface. There is no need to fear maceration, as the gel, which contains calcium alginate, absorbs moisture.
- The bandage can be removed painlessly and easily, if necessary, by moistening the bandage or the compress.

SAFETY INFORMATION

- The bandage may only be used on open wounds in combination with, and over, a sterile wound dressing.
- No known interactions with ointments. However, it is not recommended to use ointments containing medicaments at the same time, as the bandage could lead to increased absorption.
- Do not use if you are allergic to any of the ingredients (e.g. phenoxyethanol).
- If itching, persistent redness of the skin or blistering occurs, treatment should be interrupted and the doctor in charge should be consulted.
- Keep out of reach of children.

STORAGE INSTRUCTIONS

The bandage should be stored at temperatures of up to max. 35° C. The cooling effect of the bandage can be improved when stored in a cool place.
The bandage should be stored horizontally.

COMPOSITION

Fabric: rayon, polyamide. / **Formula:** zinc oxide, water, glycerin, calcium alginate, hydrocolloid gel, cardiospermum at a dilution level of 10⁶, phenoxyethanol.

PRODUCT RANGE

AN 20145 - single-pack bandage (8 cm x 3 m)
incl. protective bandage (7,5 cm x 4,5 m)

CE: conforms to EU Directive 93/42/EEC for medical products

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Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Alemania
Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Germania