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INSTRUCTIONS FOR USE
Cooling compression bandage

Aktimed VENEN is a moist, cooling, semi-rigid compression bandage with good ply adhesion from the group of zinc paste bandages. The bandage remains fixed in place even after swelling of edema. The formula contains, among other ingredients, a hydrocolloid gel in which aesculus hippocastanum 10⁻⁸ is included. The bandage's main physical/mechanical effect is supported by this formula. The formula is free of fats such as vaseline and of paraffins and is hypoallergenic as well as latex-free.

INSTRUCTIONS FOR USE

- Open wounds/skin areas must first be covered with a sterile wound dressing.
- The bandage is wound circularly. The best stability is achieved with approx. 3 to 4 layers.
- A protective bandage on top of the dressing protects against clothes getting soiled.
- The dressing dries within approx. 4 to 5 hours, then forms a stable, compressive shell and can remain in place for up to 6 days.

SAFETY INFORMATION

- The bandage may only be used on open wounds in combination with, and over, a sterile wound dressing.
- In cases of arterial occlusive diseases, decompensated heart failure, septic phlebitis and phlegmasia caerulea dolens. The bandage should only be used after consultation with your doctor.
- Do not use if you are allergic to any of the ingredients (e.g. phenoxyethanol).
- If itching, persistent redness of the skin or blistering occurs, treatment should be interrupted and the doctor in charge should be consulted.
- Keep out of reach of children.

STORAGE INSTRUCTIONS

The bandage should be stored at temperatures of up to max. 35° C. The bandage should be stored horizontally.

COMPOSITION

Fabric: rayon, polyamide. / **Formula:** zinc oxide, water, glycerin, hydrocolloid gel, Aesculus hippocastanum at a dilution level of 10⁻⁸, phenoxyethanol, dextrin.

PRODUCT RANGE

AV 20133 - single-pack bandage (10 cm x 7 m)
incl. protective bandage (7,5 cm x 4,5 m)

CE: conforms to EU Directive 93/42/EEC for medical products
Production: Germany / **Last updated:** 09/2018
Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Germany
www.aktimed-tape.de

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Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Alemania
Aktimed GmbH, Theaterstrasse 18, 69117 Heidelberg, Germania