Acekool Earwax Removal EV1

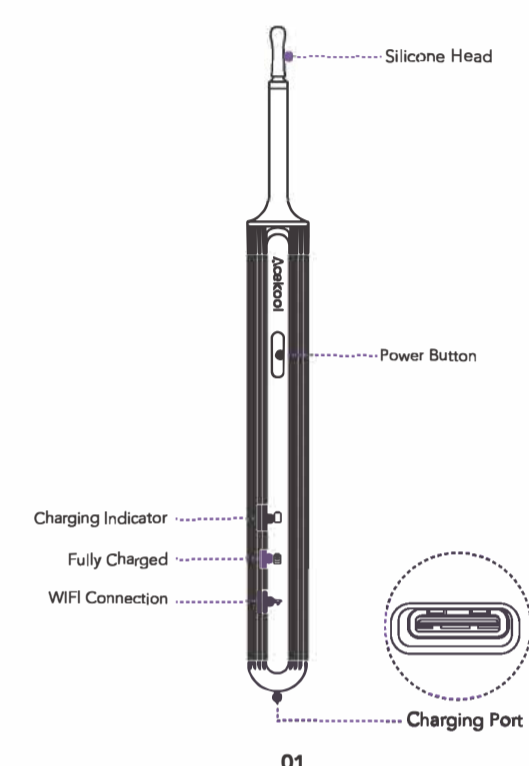
Acekool

Ear Wax Removal EV1

INSTRUCTION MANUAL

Product Illustration

01

Steps to Use on the Phone

Step 1: Download "Suesar" App

For IOS:
Download and install the App "Suesar" from App Store.

For Android:
Download and install the App "Suesar" from Google Play Store.

Optional:
Scan the QR Code below to download the App.

Compatible Devices:
Android 4.4, IOS 9.0 and higher version.
iPhone 13 / 13 Pro Max, iPhone 12, iPhone 11, iPhone X, iPhone 8, iPhone 8 Plus, iPhone 7, iPhone 7 Plus, iPhone 6S/5, iPhone 6 / 6S Plus, iPhone 5 / 5S/5C, etc.

Notice: This model can't work for computer system!

02

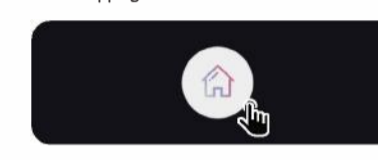
Step 2: Connection steps

1 Long press the switch key to open device, after 3 seconds, then step into next operation.

03


4 After the mobile phone displays "Suesar-xxxx" WIFI is connected, return to the App again and click the house icon to enter the device.

After connecting to the product WIFI connection, you will be asked some questions:


IOS:
"Suesar" would like to find and connect to devices on your local network.
Suggestion: Click "Keep".

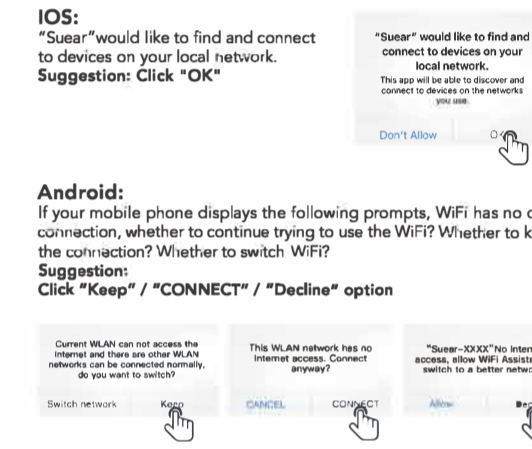Android:
If your mobile phone displays the following prompts, WIFI has no data connection, whether to continue trying to use the WIFI? Whether to keep the connection? Whether to switch the WIFI?
Suggestion: Click "Keep" / "CONNECT" / "Decline" option.


04

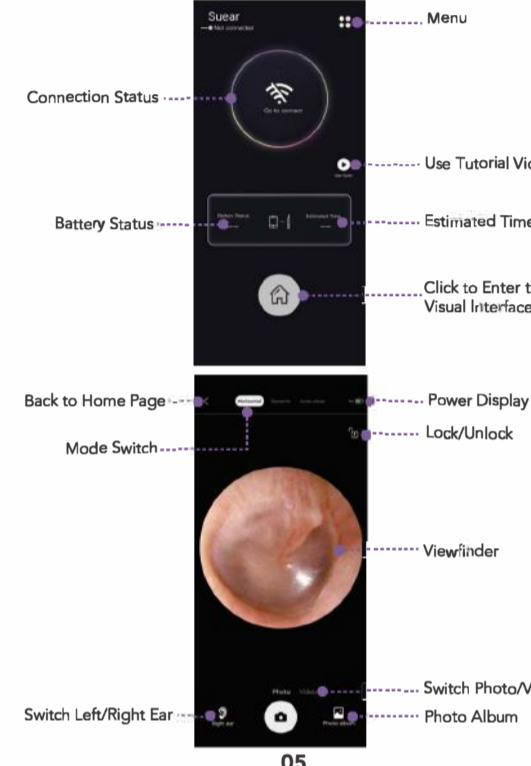Interface Introduction

05

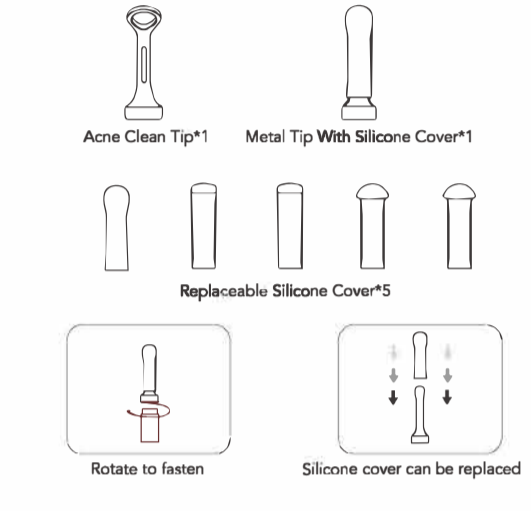Ear Pick Accessories

06

1 You can choose a tip which should be wipe by wet tissue or alcohol before use, if need, you can replace it with.

2 You should be careful or slowly go into your ear canal, then tracking it from APP to get the environment of ear canal.

Product Parameter

Product Name: Ear Wax Removal	Working Frequency: 2.4Ghz
Network Standard: IEEE 802.11b/g/n	Image Sensor: CMOS
Image Transmission Rate: 30fps	Battery Capacity: 350mAh
Environment Temperature: 10-50°C	Charging Time: 1.5h
Battery Life: About 90 minutes	Working Voltage: 3.7V4.2V
Input Parameter 5V=0.4A	Lens Diameter: 3.5mm
Rate Power 0.75W	Pixel: 5MP
Best Focal Length: 1.5-2cm	Charging port: USB2.0
Gravity Sensor: 6 Axis	

07

Troubleshooting

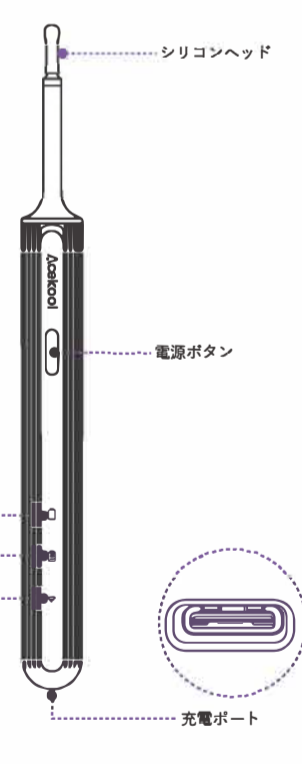
Problems	Suggestions
Cannot save pictures or videos to your phone	Allow Suesar to access photos, media, and files on your device
Camera is no response, camera doesn't light up	Please charge the device
APP crashed	Please uninstall APP and scan the QR code to install 1. Make sure your phone is connect to the "Suesar-XXXX" WIFI. 2. The question: The WLAN network has no Internet access. Connect anyway? Please be sure to choose the "Connect" if you choose "CAN-CEL" at the first time, you need to forget network of the "Suesar-XXXX" WIFI on your phone then connect with the WIFI again, continue to open the app, and choose "CONNECT" of this question so that it can work properly. 3. Enter your mobile device setting and allow the "Suesar" app to access "WLAN" or "WLAN & Cellular Data". 4. Close the App and open it again. 5. Turn the mobile phone data on then off again.
Live image does not show after launching the app	
Ear endoscope's red light on	Please charge the product
WIFI connection break	Please restart ear endoscope, open the app, connect to WIFI

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