MICRONEEDLING / MICROCHANNELING / NANONINFUSION HYPERPIGMENTATION REDUCING METHODS - ALL SKIN TONES

Age and sun spots Acne scars Melasma
Uneven tone & texture

DOUBLE CLEANSE PEEL SKIN NEEDLING FINISH PRESCRIBE

CORE STEPS

- 1. Apply Pre Cleansing Oil on dry skin and massage well. Do not remove yet.
 2. Mix Probiotic Powder Cleanser with water in bowl and apply with fan brush. Massage well.
 3. Remove with hot towel.
- 1. With gloved hands, massage 2-3 droppers of Resurfacing Salicylic Oil.
- 2. Neutralize by cleansing with Probiotic Powder Cleanser.
- 3. Remove with aesthetic wipe.
- 1. Pour Pigment Control Powder Serum or selected Powder Peel in medicine cup.
- 2. Add water as indicated and mix with fan brush until dissolved.
- 3. Working in sections, apply to skin and then work over it with your device.
- 4. Repeat for every section.
- 6. Once done, wipe residue with wet aesthetic wipe.
- 1. Apply layer of Pigment Control serum.
- 2. Optional: few drops of Soothing Face Oil.3. Finish with choice of SPF.
- 1. Prescribe 4 must-have products.
- 2. Prescribe series of 3-5 treatments every 3-4 weeks.
- 3. Measure progress by taking pictures.

CUSTOMIZATION OPTIONS

MICRO-NEEDLING (Gliding) Pre Cleansing
 Probiotic
 Powder
 Cleanser



 Resurfacing Salicylic Oil.
 Gently massage for 1 to 5 minutes.



ProductPigment Control Powder Serum





Device

- 1. Depth: For forehead, nose: 0.5mm. Cheeks, chin, temples: 0.75mm
- 2. Speed: Slow
- 3. Technique: 4-6 passes working slowly to help penetration.



1. Probiotic Powder Cleanser 2. Activated Enzyme Scrub 3. Vitamin C Powder Serum 4. Extra Nourishing

Face Oil



MICRO-CHANNELING (Tapping) 1. Pre Cleansing Oil 2. Probiotic Powder Cleanser



 Resurfacing Salicylic Oil.
 Gently massage for 1 to 3 minutes.



ProductSuccinic Azelaic Powder Peel





- Device
 1. Depth: For forehead,
 nose: 0.25mm. Cheeks,
 chin, temples: 0.5mm
 - 2.Technique: Tap 4-6 times to help penetration.
- 1. Pigment Control Powder Serum 2. Soothing Face Oil
- 1. Probiotic Powder Cleanser 2. Activated Enzyme Scrub
- 3. Vitamin C Powder Serum
- 4. Extra Nourishing Face Oil

NANO-INFUSION (Gliding) 1. Pre Cleansing
Oil
2. Probiotic
Powder
Cleanser



 Resurfacing Salicylic Oil.
 Gently massage for 1 to 3 minutes.



ProductLactic Azelaic Powder Peel



Peel



Device

- 1.Depth: Max depth device allows
- 2.Speed: Slow
- 3.Technique: 2-4 passes working slowly to help penetration.



- Probiotic Powder Cleanser
 Activated Enzyme
- Scrub
 3. Vitamin C Powder
 Serum
- 4. Extra Nourishing Face Oil

