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**EN** Physiological Solution,  
cleansing and nourishing

Please read the instructions before use

 **Validity: 5 years**

**INSTRUCTIONS FOR USE**

Cleansing and nourishing physiological solution is a valid help for baby's daily nasal hygiene. Its cleansing and nourishing action helps the elimination of excess mucous. It can also be used for the external hygiene of the eyes, using a compress soaked with physiological solution, and for aerosol therapy.

A natural product that does not contain preservatives. It has no contraindications, even in case of prolonged use. The practical monodose bottles are hygienic and easy to use.

**HOW TO USE THE PRODUCT**

Always clean your hands before opening the bottle.

Nasal Hygiene

1. Open the sterile bottle, turning the butterfly open/close cap (diag. 1).
2. Lay the child on its side, supporting its head with one hand (diag. 2). Gently insert the spout of the bottle into a nostril, and press the bottle to apply the solution. Repeat the operation with the other nostril.
3. Wait a few seconds to let the solution take effect. Raise the child and remove the secretions with a handkerchief or the nasal aspirator (diag. 3).
4. Dispose of the bottle after use.

How to Clean the Eyes

Pour a few drops of physiological solution onto a sterile gauze pad (one for each eye), and gently clean each eyelid externally, avoiding direct contact with the eyes.

**INGREDIENTS**

Sodium chloride 0,9 g, purified water sufficient for 100 ml. STERILE. It does not contain preservatives.

**WARNINGS**

- For external use. Do not inject.
- Store at room temperature. Do not expose to light or heat sources.
- Keep the instructions until the product has been used completely.
- The product comes in mono-dose phials. Each phial should be used only once to preserve the sterile characteristics of the product intact.
- Do not use the bottle after the expiry date.
- Do not dispose of the bottles in the environment.
- Keep out of reach of children

 Disposable device (for single patient use only).

Do not reuse: risk of possible infections and loss of functionality.

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PIKIDARE S.p.A.
Arsiana S.p.A.