



# Virox<sup>TM/MC</sup> 5 RTU

## Ready-to-Use Surface Cleaner & Disinfectant

A one-step disinfectant cleaner based on proprietary Accelerated Hydrogen Peroxide technology to deliver fast and effective cleaning performance.

### Features & Benefits

- Ready-to-use
- Also provides broad spectrum sanitizing on hard surfaces in 30 seconds
- Proven effectiveness against the Polio virus, Sabin strain type 1 (ATCC VR 192) allows for a General Virucide claim against most enveloped and non-enveloped viruses
- Kills Norovirus, MRSA and VRE in 5 minutes
- Designed for cleaning and disinfection of hard, non-porous surfaces
- Environmentally-responsible; no VOC's, no added fragrance, no NPE's
- The active ingredient breaks down to water and oxygen after use

### Applications

- For use as cleaner and disinfectant of hard, nonporous surfaces
- Meets bloodborne pathogen standards for decontaminating blood and body fluids





# Virox<sup>TM/MC</sup> 5 RTU

## Ready-to-Use Surface Cleaner & Disinfectant

### Use instructions

#### Disinfectant Use Directions

- Disinfection of non-critical items, equipment and non-porous hard surfaces coming in contact with intact skin such as the exterior of hemodialysis machines, stethoscopes, tabletops etc.: Apply to surface with cloth or disposable wipe. Ensure surface remains wet for 5 minutes at 20° C. Wipe surface dry or rinse.
- Cleaning and disinfecting surfaces of blood and body fluids:* Gloves should be worn. Remove excess blood and fluid with absorbent materials. *Clean contaminated area:* Apply to surface, soak 30 seconds, wipe dry. *Disinfect contaminated area:* Apply to surface, allow to remain wet for 5 minutes at 20° C. Wipe surface dry or rinse.

#### Broad-Spectrum Sanitizing (99.999%) on Environmental Surfaces

- Apply to surface and allow to remain wet for 30 seconds. Wipe dry. No rinsing is required. Food contact surfaces require rinsing with potable water.

Technical data	Virox <sup>TM/MC</sup> 5 RTU
DIN	02239775
Color/Form	Colorless, liquid
pH	1.75
Scent	Characteristic
Shelf Life	2 years

Product	Pack size	Dilution	Product code	
Virox <sup>TM/MC</sup> 5 RTU	12 x 1 L Spray Bottles	Ready-To-Use	53808	

### Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at [www.diversey.com](http://www.diversey.com) or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate.

Accelerated Hydrogen Peroxide<sup>TM/MC</sup> and AHP<sup>®</sup> logos are trademarks owned by Virox Technologies used under license.