


# DUO-LINK UNIVERSAL™

with UNIVERSAL PRIMER™

DUO-LINK UNIVERSAL™  
with UNIVERSAL PRIMER™



[www.biolinkdubai.com](http://www.biolinkdubai.com)

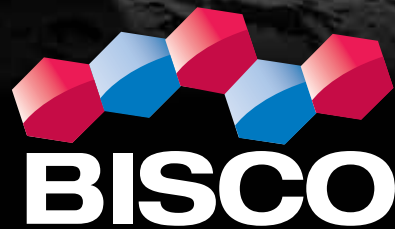

# OUT OF THIS WORLD STRENGTH!

**DUO-LINK UNIVERSAL** adhesive resin cement is specially formulated for cementation of ALL\* indirect restorations. It is available in two esthetic shades, Universal and Milky White. **DUO-LINK UNIVERSAL** is intended for use with adhesives designed for compatibility with all dental materials, including all BISCO adhesives.

## Extremely Easy Clean-Up

Formulated to allow for quick and easy removal of excess cement

## Easy to Use

Auto-mix, dual-syringe provides a consistent mix for immediate delivery

## Highly Radiopaque

Easily identified on radiographs for quick and effective diagnosis

## High Degree of Conversion

High degree of conversion in both light- and self-cured modes ensures a strong, long lasting restoration

## Universal - For all cementation procedures\*

Crowns, bridges (fixed prostheses), inlays, onlays, and posts/dowels, fabricated from metal, composite, porcelain, ceramic, zirconia, alumina, CAD/CAM restorations, etc.

## CAD/CAM restorations


Ideal for all chairside and lab-fabricated restorations

## Esthetic shades


Universal shade has some translucency at low film thickness and Milky White shade is more opaque and can be used to mask darker tooth preps

## UNIVERSAL PRIMER PERFORMS BEST COMPARED TO COMPETITORS\*\*

Shear Bond Strength to Dentin of Manufacturer's Resin Cement System

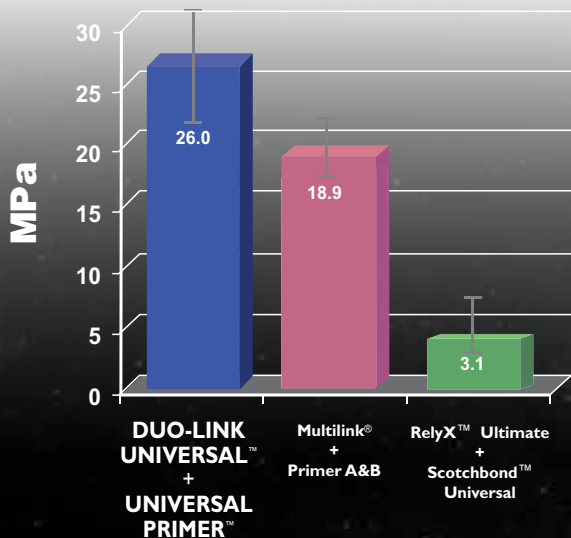

Shear Bond Strength to Dentin Comparison of Manufacturer's Resin Cement System vs. Manufacturer's Cement Paired with BISCO's UNIVERSAL PRIMER

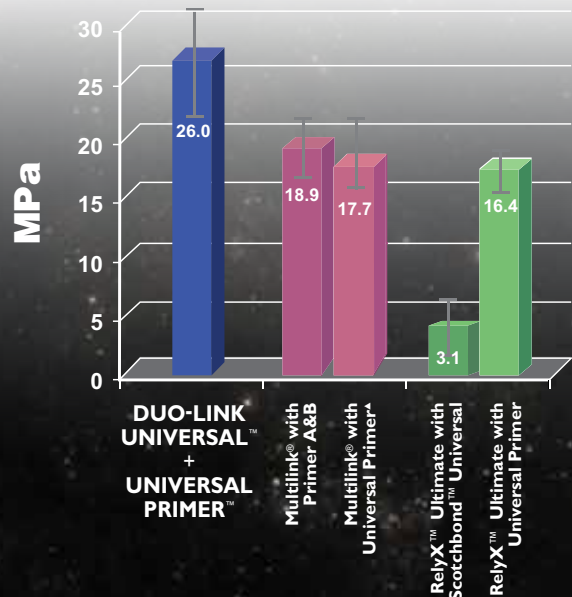

Trademarks are property of their respective owners.

\* BISCO's CHOICE™ 2 is recommended for veneer cementation.

\*\* Both adhesive and cement cured via self-cure mode.

Shear Bond Strength was tested by using the Ultradent jig method. Specimens were aged at 37°C in water for 24 hrs. before sample breaking. Data on file, BISCO, INC.

▲ Per Instructions For Use, Multilink is not recommended for use with other adhesives/primers.



# DUO-LINK UNIVERSAL SYSTEM KIT . . .

## UNIVERSALLY SIMPLE, UNIVERSALLY STRONG

Courtesy of Ross W. Nash, DDS, FAACD



Tooth preparations for a 3-unit fixed partial denture.



Application of dual-curing UNIVERSAL PRIMER.



Simple and easy clean-up of DUO-LINK UNIVERSAL.



Immediate post-operative final restoration.



### DUO-LINK UNIVERSAL with UNIVERSAL PRIMER Kit

is an adhesive cementation system designed to effectively address the cementation of your indirect restorations for optimal performance and esthetics. The combination of award-winning, innovative products provides high retention and durability of indirect substrates in a simple, universal, and easy-to-use kit.

### DUO-LINK UNIVERSAL™

**DUO-LINK UNIVERSAL™** adhesive resin cement is specially formulated for cementation of ALL indirect restorations and provides all of the desired properties for universal cementation: high radiopacity, high mechanical strength, high bond strength, low film thickness and easy clean-up.

### UNIVERSAL PRIMER

**UNIVERSAL PRIMER™** is a low film thickness, dual-cured adhesive/primer designed for the dentist who prefers not to light cure the adhesive layer under indirect restorations.

### Z-PRIME™ plus

**Z-PRIME™ Plus** is a single-component priming agent used to enhance adhesion between indirect restorative materials and resin cements. It is a versatile primer for surface treatment of zirconia and alumina ceramics as well as metals/alloys, composites and endodontic posts.

### PORCELAIN PRIMER

**PORCELAIN PRIMER** is a silane coupling agent used to improve the chemical bond between porcelain restorations and resin cements resulting in increased bond durability over time.



## RESTORATION PREPARATION\*



1. Apply PORCELAIN PRIMER to restoration

OR



1. Apply Z-PRIME Plus to restoration




## TOOTH PREPARATION AND CEMENTATION\*



**1. SELF-ETCH:**  
Prepare Cavity. Wash thoroughly with water spray. Remove excess water (do not desiccate)



**1. TOTAL-ETCH:**  
Prepare cavity. Wash thoroughly with water spray, apply phosphoric acid etchant and rinse thoroughly. Preparation should be visibly moist

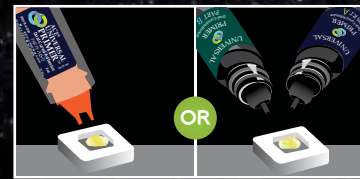

2. Dispense an equal number of drops of UNIVERSAL PRIMER Parts A & B (1:1 ratio) into a clean mixing well. Using a brush, mix for 5 seconds.



3. Apply UNIVERSAL PRIMER



4. Air-dry



5. Apply DUO-LINK UNIVERSAL to restoration



6. Seat the restoration and tack cure for easy removal of excess cement.



7. Optional: Light cure

\* Refer to complete instructions for detailed techniques.

## ORDERING INFORMATION:

<b>DUO-LINK UNIVERSAL WITH UNIVERSAL PRIMER</b> .....	<b>A-19710K</b>
1 Dual-Syringe DUO-LINK UNIVERSAL Shade: Universal (8g), 1 UNIVERSAL PRIMER Part A (6ml), 1 UNIVERSAL PRIMER Part B (6ml), 1 Bottle Z-Prime Plus (2ml), 1 Bottle Porcelain Primer (3ml), Accessories, Instructions	
<b>DUO-LINK UNIVERSAL WITH ACE UNIVERSAL PRIMER &amp; ACE DISPENSER</b> .....	<b>M-16200K</b>
1 Dual-Syringe DUO-LINK UNIVERSAL Shade: Universal (8g), 1 ACE® UNIVERSAL PRIMER Cartridge (2ml), 1 Bottle Z-Prime Plus (2ml), 1 Bottle Porcelain Primer (3ml), 1 ACE Dispenser, Accessories, Instructions	
<b>DUO-LINK UNIVERSAL WITH ACE UNIVERSAL PRIMER</b> .....	<b>A-19730K</b>
1 Dual-Syringe DUO-LINK UNIVERSAL Shade: Universal (8g), 1 ACE® UNIVERSAL PRIMER Cartridge (2ml), 1 Bottle Z-Prime Plus (2ml), 1 Bottle Porcelain Primer (3ml), Accessories, Instructions	
<b>ACE UNIVERSAL PRIMER STARTER KIT</b> .....	<b>M-16300K</b>
2 ACE® UNIVERSAL PRIMER Cartridges (2ml ea.), 1 ACE Dispenser, Accessories, Instructions	
<b>DUO-LINK UNIVERSAL SYRINGE PACKAGE</b>	
1 Dual-Syringe DUO-LINK UNIVERSAL (8g), 15 Mixing Tips and Intra Oral Tips, Instructions	
Universal .....	<b>A-19030P</b>
Milky White .....	<b>A-197MWP</b>
<b>UNIVERSAL PRIMER</b> .....	<b>B-4110P</b>
1 UNIVERSAL PRIMER Part A (6ml), 1 UNIVERSAL PRIMER Part B (6ml), Accessories, Instructions	
<b>ACE UNIVERSAL PRIMER 4 CARTRIDGE PACKAGE</b> .....	<b>B-41240P</b>
4 ACE UNIVERSAL PRIMER Cartridges (2ml ea.), Accessories, Instructions	



Rx Only



www.biolinkdubai.com



MC-3163DUP