

OXYGEN BOTTLE HOLDER AND CASE

FRESH AIR, EVERYWHERE



movinglife[®]

ATTO Accessories are created specifically to enhance, support and work with the ATTO mobility scooters in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

SCAN TO LEARN HOW TO COMBINE OUR ACCESSORIES

SEATING & SUPPORT



ATTO /ATTO SPORT Armrests



Seat cushion



Seat cushion leather



Luxury Swivel Seat Leather



Belt & Support Strap



Luxury Orthopedic Backrest

STORAGE



Essentials Pouch



Front & rear basket



Carryall & Cushion



Carryall & Cushion leather



Pet Carrier & Cushion Black



Flight Kit



Oxygen Tank Holder and Case

TECHNICAL



Cane / Crutches Holders



Mobile Phone Holder



Rear Light Lifting Handle



ATTO Left Hand Throttle Kit



Standard Flight Battery




Launch - H2 2023
Long distance battery - not flight approved



Cable kit


How to Use

OXYGEN TANK HOLDER AND CASE

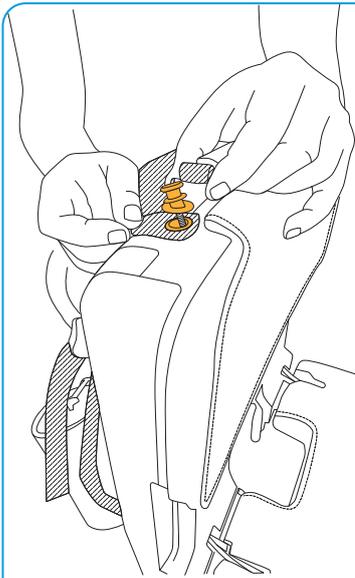


01

Unscrew the plastic knobs attached to the back seat. Insert the loop of the oxygen tank holder rear section under the lift handle, and screw back the knobs

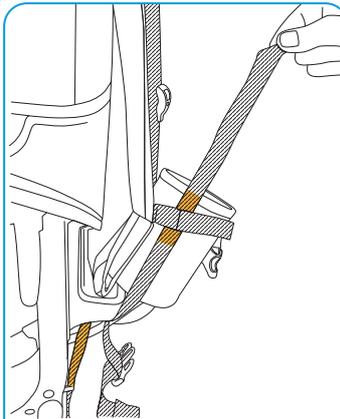


02

Pass the straps through the gap at the back of the chair and then through the straps on the oxygen tank holder

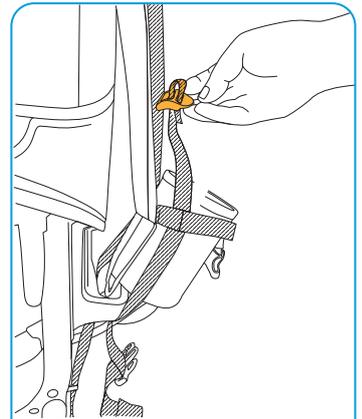


03

Connect the straps together by threading a strap through the loop

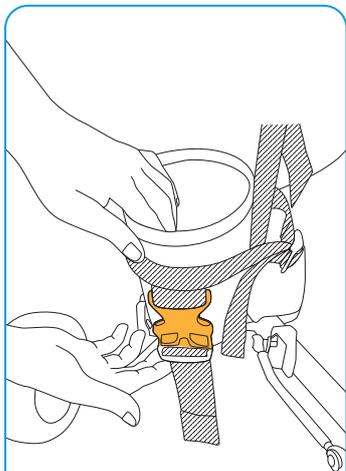


04

Tighten the rear buckle to secure the tank holder

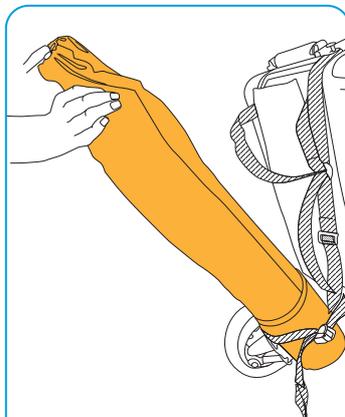


05

Insert the oxygen tank into the bottom of the tank holder

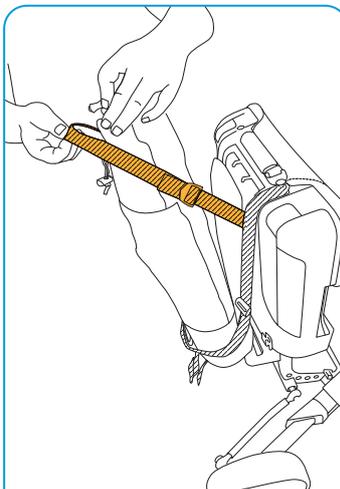

06

Fasten the strap to the top of the container

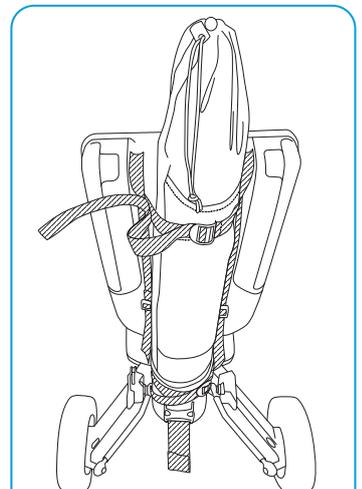

07

The product is ready to use.