

# Product Bulletin

Product Bulletin #2021-034

## Discontinuation of 8900-000260 and 8900-000270 CPR Uni-padz® Electrodes

We would like to inform you of the discontinuation of certain ZOLL CPR Uni-padz electrodes, effective March 31, 2022. This update will allow our customers to streamline their Uni-padz management with one singular product. The discontinued products and their respective alternatives are as follows:

Discontinued Electrode	Part Number	Alternative Electrode	Part Number	Where Used
CPR Uni-padz I	8900-000260	CPR Uni-padz III	8900-000280	For clinical use with all ZOLL AED 3® and ZOLL AED 3® BLS devices.
CPR Uni-padz II	8900-000270	CPR Uni-padz III	8900-000280	For clinical use with all ZOLL AED 3 and ZOLL AED 3 BLS devices.
CPR Uni-padz I Demo/Training Padz	8900-000264	CPR Uni-padz III Demo/Training Padz	8900-000284	Connects to ZOLL AED 3 series defibrillators for training and simulations.
CPR Uni-padz I Training Replacement Pads	8900-000265	CPR Uni-padz III Training Replacement Pads	8900-000285	Contains eight sets of two reusable training pads for use with the 8900-000284.
CPR Uni-padz II Demo/Training Padz	8900-000274	CPR Uni-padz III Demo/Training Padz	8900-000284	Connects to ZOLL AED 3 series defibrillators for training and simulations.
CPR Uni-padz II Training Replacement Pads	8900-000275	CPR Uni-padz III Training Replacement Pads	8900-000285	Contains eight sets of two reusable training pads for use with the 8900-000284.

The 8900-000280 CPR Uni-Padz III electrodes, 8900-000285 CPR Uni-Padz III Training Pads, and 8900-000284 CPR Uni-Padz III Demo/Training Padz are the latest electrode product releases. They are designed for use with ZOLL AED 3® Release 6 Software, although they are also compatible with previous software versions. A software upgrade can be





obtained FREE of charge via electronic delivery or purchased as a pre-loaded USB stick (P/N 7712-000005-01). Email ZOLL Technical Service ([TSservice\\_Master@zoll.com](mailto:TSservice_Master@zoll.com)) to obtain the software via electronic delivery or USB stick.

All ZOLL AED 3® and AED 3 BLS® devices are shipped with Software Release 6, which includes updated graphics and pediatric placement instructions that mirror CPR Uni-padz III. We recommend that all customers using previous software versions on their AEDs update to Software Release 6. The update will allow the graphics and voice prompts on the AED to match the placement recommendations on the electrodes. While all previous electrode versions are clinically compatible with Software Release 6, it is recommended that customers upgrading to Software Release 6 switch to CPR Uni-padz III when replacing their electrodes.

ZOLL has a large electrode portfolio to meet your clinical needs. If you have additional questions about our electrodes, please visit the ZOLL Electrode Finder Website, or contact your local Sales Manager.

This product discontinuation will not affect the following countries: China, El Salvador, Japan, Mexico, Peru, Taiwan.

Please contact customer service at the contact information below to inquire about electrodes orders.

ZOLL Customers Outside the US		1-800-348-9011		<a href="mailto:intlcs@zoll.com">intlcs@zoll.com</a>
ZOLL Customers Within Canada		1-866-442-1011		<a href="mailto:CanadaCustomerService@zoll.com">CanadaCustomerService@zoll.com</a>

Alex Kea  
Global Product Manager, Electrodes  
ZOLL Medical Corporation  
[Alex.Kea@zoll.com](mailto:Alex.Kea@zoll.com)

## FAQs

Q: How do I obtain the latest ZOLL AED 3 device software?

A: There are two options to obtain the Software Release 6 upgrade files:

1. FREE electronic delivery. Email ZOLL Technical Service to request an electronic delivery of the software file (download from secure file server).
2. Purchase a programmed USB stick (P/N 7712-000005-01). You can purchase a USB stick with the Software Release 6 file pre-loaded from ZOLL Technical Service.

Q: Are the 8900-000280 CPR Uni-padz III electrodes compatible with previous ZOLL AED 3 software versions?

A: Yes, the 8900-000280 CPR Uni-padz III electrodes are compatible with all ZOLL AED 3 and ZOLL AED BLS software versions. Software versions prior to Release 6 will instruct to use a different pediatric electrode placement than the CPR Uni-padz III when in pediatric mode.

Q: Are the 8900-000260 CPR Uni-padz I and 8900-000270 CPR Uni-padz II electrodes compatible with ZOLL AED 3 devices operating with Software Release 6?

A: Yes, the 8900-000260 CPR Uni-padz I and 8900-000270 CPR Uni-padz II electrodes are compatible with ZOLL AED 3 devices operating with Software Release 6. Software Release 6 will instruct to use different pediatric electrode placement than CPR Uni-padz I.

Q: Will the ZOLL AED 3 device operation be affected if I place CPR Uni-padz in the original placement recommendations?

A: No, the ZOLL AED 3 will operate with the CPR Uni-padz in either instructed placement. Customers should place electrodes as trained as to not delay treatment.