


User's manual



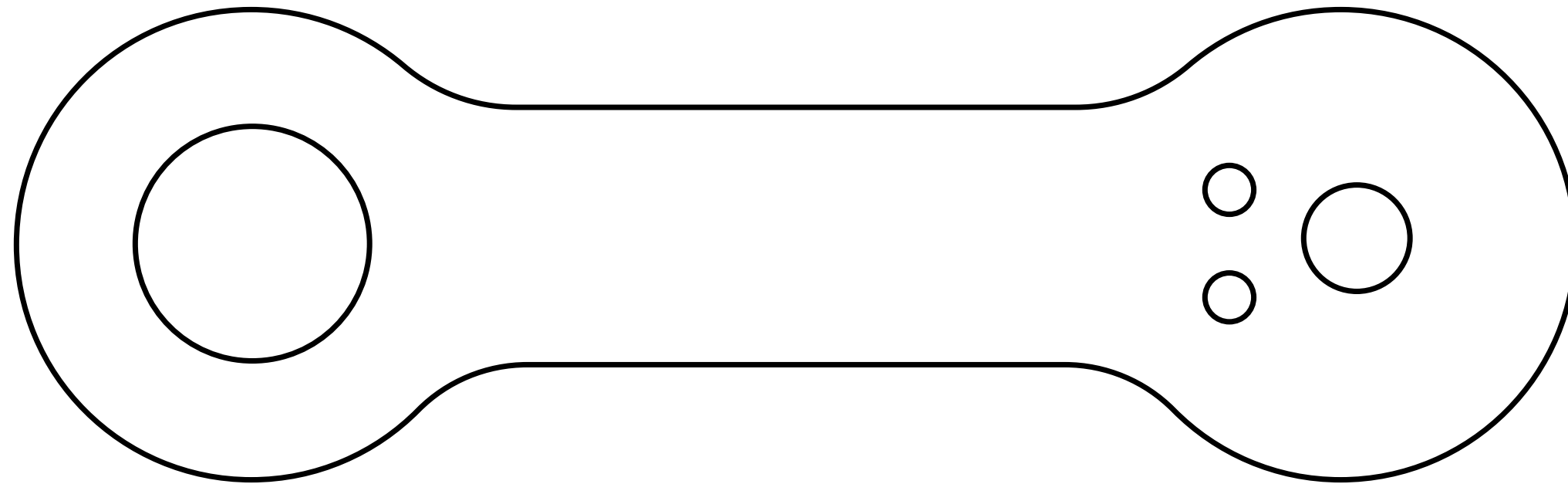

Clip the **CardioPod**
in this direction.
(right hand side)

CardioPod

Clicshirt



2

1 long press
(> 4 secondes)
turns on/off the
CardioPod





5 short and successive presses on the button turn on the CardioPod in configuration mode.

4

1 long press (> 12 seconds) to restart the CardioPod and **resets user settings and internal clock.**

5

In case the battery is discharged (<5%). The CardioPod flashes the **red LED** rapidly for **20 minutes** before going into deep sleep.



6

When the CardioPod is placed on the charger, the red LED starts flashing to indicate charging.

During charging, **the red LED flashes** rapidly (as long as the battery is <5%), then slowly (as soon as >5%), and finally **lights up continuously when the charge is complete** (max. 100% after 3 hours). The CardioPod can then be removed from the charger





When switching on from deep sleep mode or a reset (long press on the CardioPod Clicshirt button > 12 seconds).

The 2 LEDs flash 4 times simultaneously to indicate the restart.

