

# CollaClot

## HOW TO USE COLLA-CLOT™ VETERINARY HAEMOSTATIC SPONGE

Please refer to the instructions below to ensure successful application of CollaClot™ Haemostatic Sponge



1. Identify / Expose the wound.




2. Identify the point of bleeding.




3. Remove CollaClot™ Haemostatic Sponge from packet.



4. Wipe away excess blood where possible.

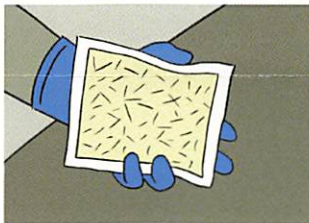

5. If required, cut or tear sponge to size to suit wound. Apply directly to source of bleed with perforated (rough) side of sponge facing outwards on palm, and swabs behind.

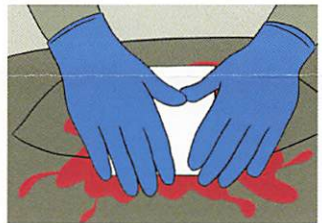

6. Cover the sponge with an appropriate dressing and apply direct pressure until bleeding is controlled / stops.



7. Wrap with a bandage to maintain pressure and seek further veterinary assistance where required.

8. Following use and where possible, excess CollaClot should be removed from the wound site. However, as a naturally occurring product (collagen) it will slowly be absorbed by the body.