EN English

YES® COCO Certified Organic Personal Lubricant

YES® COCO is a luxurious, natural, plant–oil based lubricant with organic coconut and jojoba to nourish and condition intimate tissues, giving lasting lubricity and comfort.

YES® COCO is intended for use by adults and is not intended for use on damaged or broken skin.

Benefits of YES® COCO

YES® COCO assists your lovemaking by providing lubricity to enable penetration and to increase pleasurable glide and sensation. YES® COCO also promotes the day-to-day comfort of intimate tissues its richly nourishing formulation provides long-lasting soothing relief from vaginal dryness and/or discomfort associated with conditions such as menopause, vaginismus, lichen sclerosus/planus and other medical conditions including but not limited to gynaecologic cancer-related treatments.

Application Frequency

YES® COCO can be used freely and frequently according to the body's own needs. To enhance pleasure and ease in love making a 5ml application is typically sufficient, but more may be used if required.

How does YES® COCO work?

Physical protection – The natural plant oils in YES® COCO provide gentle physical protection to sensitive tissues.

Intensely moisturising – Plant oils condition tissues, no need to wash off.

Possible side effects

Side effects related to reinstating vaginal lubrication – you may experience some irritation, itching or burning whilst vaginal lubrication is restored to normal levels.

Rare side effects (affecting less than 1 in 1,000 people) may include discharge or infection. YES® COCO is not made with any known allergens, however there remains a remote possibility (as with anything applied to the body) that sensitisation/allergic reaction may occur.

WARNING: If irritation occurs, discontinue use. If symptoms persist seek medical advice.

Report **serious** side effects to The YES YES Company Ltd and your local health authority.

Interactions with vaginally applied medications/examinations

YES® COCO can be safely used with silicone vaginal dilators / trainers or menstrual cups.

WARNING: Not condom compatible

CAUTION: YES® COCO is not a contraceptive or spermicide.

YES® COCO should not be applied at the same time as other medications such as vaginal HRT, vaginal antifungal or antibacterial medications without consulting your pharmacist / physician.

Not suitable for use with the ThinPrep Pap test.

Storage

Store in a cool dry place (5–30°C). Keep out of direct sunlight. Use before expiry. Once opened, use within 48 hours.

Warning: If the seal on any applicator is damaged or opened DO NOT use, return to place of purchase.

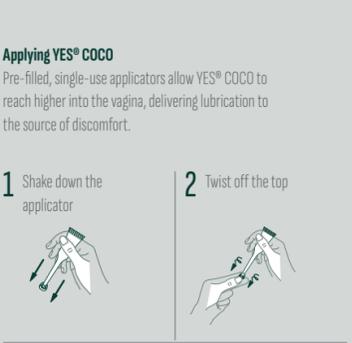
Ingredients

Sunflower seed oil", Coconut oil", Jojoba seed oil", Hydrogenated palm oil", Hydrogenated Rapeseed oil", Grape seed oil", Tocopherol (Vitamin E).

* Organically grown ingredients 99% organic.

† Ingredients responsible for product effectiveness >99%

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Indberet alvorlige bivirkninger til The YES YES Company Ltd og din lokale sundhedsmyndighed.