EN **English**
YES **OB** is a luxurious plant-oil based natural lubricant, whose rich oils and butters nourish and condition intimate tissues, giving lasting lubricity and comfort.

YES[®] OB is intended for use by adults and is not intended for use on damaged or broken skin.

Benefits of YES[®] OB
YES[®] OB assists your love making by providing lubricity to enable penetration and to increase pleasurable glide and sensation. YES[®] OB also promotes the day-to-day comfort of intimate tissues by richly nourishing formation providing soothing relief from vaginal dryness and/or discomfort associated with conditions such as menopause, vaginismus, lichen sclerosus/planus and other medical conditions including but not limited to gynaecological cancer-related treatments.

Applying YES[®] OB
YES[®] OB is a rich cream that liquefies into a smooth oil on skin. Simply squeeze the cream onto your hand, warm it by rubbing between your fingers to ensure a smooth velvety quality, then use your fingertips to apply to either the pre or vaginal tissues and/or labia. (No applicator included). Dual purpose – can also be used as body massage oil.

Application Frequency
YES[®] OB can be used freely and frequently according to the body's own needs. To enhance pleasure and ease in love making a 5ml application is typically sufficient, but more may be used if required.

Pressure tip: Experience the Double Glide Effect – apply a layer of YES[®] WB on top of YES[®] OB for a lasting and new dimension in love making.

How does YES[®] OB work?
YES[®] OB – The mixture of plant oils provide long-lasting lubrication.

Physical protection – The natural butters in YES[®] OB provide gentle cushioning to delicate tissues.

Intensely moisturising – The combination of plant oils and butters condition tissues, no need to wash off.

Care note: YES[®] OB may occasionally present with a slightly grainy texture which is a function of using exclusively natural ingredients that are temperature sensitive. This is easily remedied, simply warm the inverted tube in a mug of hot water, once it is completely liquid, shake to mix, then place in the fridge to cool quickly. This will restore the perfect smooth texture of YES[®] OB. The product is perfectly safe to use.

Possible side effects
Side effects related to reinstating vaginal lubrication – you may experience some irritation, itching or burning whilst vaginal lubrication is restored to normal levels.

Rare side effects (affecting less than 1 in 1,000 people) may include discharge or infection. YES[®] OB is not made with any known allergens, however there remains a remote possibility (as with anything applied to the body) that sensitisation/allergic reaction may occur.

WARNING: If irritation occurs, discontinue use. If symptoms persist seek medical advice.

Report *serious* side effects to The Yes[®] YES Company Ltd [and](#) your local health authority.

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