# Tesmed+

info@tesmed.com

ITALIANO

ENGLISH

FRANÇAIS

DEUTSCH

ESPAÑOL

ELETTROSTIMOLATORE PROFESSIONALE  
PROFESSIONAL ELECTRICAL MUSCLE STIMULATION  
STIMULATEUR ÉLECTRIQUE PROFESSIONNEL  
PROFESSIONELLE ELEKTROSTIMULATIONSGERÄT  
ELECTROESTIMULADOR PROFESIONAL

MANUALE D'USO - INSTRUCTION MANUAL - MODE D'EMPLOI  
GEBRAUCHSANWEISUNGEN - MANUAL DE INSTRUCCIONES

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## contents

> > >	<b>symbols used</b> .....	19
> > >	<b>contraindications</b> .....	20
> > >	<b>warnings</b> .....	21
	Precautions .....	21
	Possible side effects .....	21
> > >	<b>electro stimulation</b> .....	21
	Electrodes .....	22
	Tiredness .....	22
	Advantages of using electro stimulation .....	22
	Results .....	23
> > >	<b>high technology</b> .....	23
> > >	<b>battery charger</b> .....	23
	Replacing the battery .....	24
	Connections.....	24
	Disconnecting the electrodes .....	24
> > >	<b>equipment description</b> .....	25
	LCD display and keys .....	25
> > >	<b>use</b> .....	25
	Switching on.....	25
	Selection of channels and programs.....	25
	Programs 01 to 20.....	25
	Programs 21 to 99 .....	26
	Intensity control .....	26
	Frequency control (for programs 01 to 20) .....	26
	Error warning.....	26
	"Keyboard block" function .....	27
	"ON block" function .....	27
> > >	<b>maintenance</b> .....	27
> > >	<b>disposal</b> .....	28

## contents

> > >	<b>normative references</b> .....	28
	(TESMED MAX 7.8 technical data page 110)	
	(Battery charger technical data page 110)	
	(Guide and manufacturer's statement: electromagnetic emission and immunity page 111)	
> > >	<b>how to choose the right treatment to suit individual needs</b> .....	29
> > >	<b>electrode polarization</b> .....	29
> > >	<b>maintenance</b> .....	29
> > >	<b>positioning of electrodes</b> .....	70
> > >	<b>personalised programs</b> .....	85
	(programs 01 to 20)	
> > >	<b>Tesmed Max 7.8 programs</b> .....	86
	(programs 21 to 99)	
> > >	<b>contacts</b> .....	115
> > >	<b>related products</b> .....	117-118

## symbols used

Important note, if not respected, there may be a risk for the user and/or the equipment may be damaged. We recommend that you read the information in this manual carefully and keep it in a safe place.

Low frequency parts applied.

Internally fed equipment (DC).

Parts that must be disposed of as special waste.

Read the operator's manual

Manufacturer

Double electrical isolation

**IP**

Protection against dust and liquid penetration

## contraindications

### ATTENTION:

#### **Electro stimulation is not recommended in areas with any of the following:**

- inflammation, open wounds, strained or torn muscles, dermatitis, bacterial infections, eritema or skin irritation;
- muscular hernias;
- fractured bones;
- varicose veins or vascular disease in general;
- neoplasia;
- mucous membranes.
- in the thoracic area to people with cardiac arrhythmia.

#### **This device is prohibited in the following cases:**

- pregnant women;
- growing children (risk of juvenile heart attack);
- patients wearing pace-makers (risk of interference with the functioning of the pace-maker);
- implanted defibrillators;
- alteration of the peripheral nerves, muscular disease and serious damage to the joints;
- serious heart problems;
- patients suffering from epilepsy;
- patients with bone tumours;
- patients with serious high blood pressure;
- patients suffering from serious systematic, debilitating or dermatological disease;
- patients undergoing treatment with an HF device, as this could cause burns and damage to the stimulator itself;
- near any short wave or microwave therapy equipment, as this could cause instability in the stimulator output;
- unconscious patients or patients unable to communicate;
- people suffering from psychophysical problems that prohibit their exposure to external factors;
- in the presence of a mix of flammable anaesthetic with air, oxygen or nitrogen protoxide.

**Never place the electrodes on the carotid artery and/or on a spastic muscle.**

Use of the TESMED Max 7.8 for therapy purposes should be supervised and recommended by a doctor. Interrupt application immediately in the event of discomfort and obtain medical advice.

## warnings

- You must take into account the technical characteristics of each program and read the relevant paragraphs in the User Manual.
- The Tesmed Max 7.8 must not be used for purposes other than those indicated.
- You must begin each treatment with the current emission value set at zero (min channel power setting) and then gradually increase and/or reduce the power to suit your tollerability.
- Max power does not necessarily mean better performance or results.
- We recommend using original electrodes only to guarantee top performance, as these have been developed specifically for the Tesmed Max 7.8. Non original electrodes could burn the skin.
- The Tesmed Max 7.8 must be kept out of the reach of children.
- Electrocardiogram monitoring equipment may be disturbed by the current used for the electro stimulator.
- We do not recommend using the Tesmed Max 7.8 on children under the age of 14 or the elderly over 75.
- The electrodes must not be placed less than 10 cm from the eyeballs.
- Do not use the equipment if there are special health problems without first obtaining medical advice from a doctor.
- Should the equipment malfunction, do not try to open or repair the equipment, but send the product in its original packing to **FELDI SRL**, describing the problems found on the inside of the package (see warranty conditions on [www.tesmed.it](http://www.tesmed.it)).
- The bipolar cables not connected to the body for the treatment must be disconnected from the device • Device not suitable for simultaneous use with high frequency surgical devices
- The application of electrodes close to the chest may increase the risk of heart fever
- The electrodes should never be placed on the carotid breast, along the head, directly on the eyes, covering their mouths and / or on a spastic muscle.
- The Tesmed Max 7.8 electrostimulator must be powered only to a very low voltage supplied by the supplied power supply. If the power supply and the cable that connects the unit is damaged, it must be replaced by a power supply and cable of the same model supplied with the product.

## Precautions

- Do not tamper with the equipment or try to open it.
- Keep the electro stimulator away from sources of heat, powder, sprayed water, humid atmospheres and avoid direct exposure to the sun.
- Avoid violent knocks that could damage the equipment.

## Possible side effects

- Isolated cases of skin irritation may occur in people with highly sensitive skin. Should there be an allergic reaction to the electrode gel, stop the treatment and consult a specialist doctor. It is normal for the skin to be slightly red after the treatment in the areas where the electrodes were placed: this will disappear after roughly 30 minutes. We strongly advise against treating any areas of the body with serious skin problems.

## electro stimulation

Electro stimulation is a technique that provokes muscular contractions very similar to voluntary contraction thanks to electrical impulses acting on the motor points of the muscles. As everyone knows, muscles contract when they receive an impulse coming from the nervous system and then return to their normal length once they are no longer stimulated thanks to their elasticity. A motor nerve fibre splits into several ramifications before entering the muscle and ends at the motor end plate. The nerve fibre and the group of

muscular fibres innervated by this make up the motor unit. A muscular fibre responds to the electrical stimulus only when the intensity of the stimulus reaches a threshold value, which is specific to each fibre. The most obvious use of the electro stimulation of innervated muscles is in kinesiotherapy of atrophy, functional reanimation after a long periods of immobility, in vasodilation, pain reducing and in stimulating the treated tissues to combat cellulite (PASSIVE EXERCISE). When treating a patient with electro stimulation, the following are crucial: the correct positioning of the electrodes, the stimulation program and the wave type.

### Electrodes

The positioning of the electrodes is crucial. It must provide accurate stimulus parameters for the zone being treated since the conductivity of body tissues varies according to their structure, aqueous and ionic make-up. The electrodes must be in perfect contact with a thoroughly clean skin (also depilated, if necessary). This ensures better penetrability and permeability of the skin when receiving the stimulus to suit the application to be carried out. **For better electrode conductivity we recommend they be replaced when they loose adherence. They can also be cleaned with running water or wiped with a damp cloth; leave to dry before using them again for a few more applications.** NB: With electro stimulation, the current density diminishes as the tissue consistency increases.

**We recommend using Tesmed electrodes exclusively, as electrodes of different sizes and composition could cause burns.**

### Tiredness

The high percentage of simultaneous contraction/relaxing of the muscles due to electrical stimulus often results in a drop in the relaxation of the neurotransmitters, thus resulting in muscle fatigue. The scientific explanation is as follows: the stimulus tends to provoke the recruitment of large diameter motor neurones (these have a low threshold), thus exciting the more consistent and faster muscle fibres (FIBRES 2 or white fibres). These tire more easily, but they use less effort than FIBRES 1 or red fibres. **In other words, the higher the frequency of stimulus, the more tired the muscle becomes.** For this reason, it is important to include relaxation programs and get medical advice. Having said this however, Tesmed's sports programs already contain a relaxation phase. **During treatments using the electro stimulator, every increase in intensity must be done gradually. If there is EXCESSIVE contraction, you must reduce the intensity.** In fact, stimulation must not provoke muscular tetany (i.e. long continuous contractions). **There are no standard rules for the number of contractions and the duration and frequency of the sessions. These must be set to suit each person and the condition of the tissues to be treated.** When treating areas affected by CELLULITE, you should normally start the treatment by gradually raising the intensity until the patient starts to feel a slight tingling, but without creating excessive contraction of the muscles or an unpleasant sensation of localised heat. **To sum up:** the vasomotor and metabolic effects induced by the passage of an electrical current considerably increase the consistency and quality of tissues. The best level of stimulus is when the patient perceives the current passing through the tissue; this should provoke a light initial contraction of the muscles (a slight trembling), but never too intense.

There are two important advantages here:

- 1) you avoid the risk of muscular over-contraction and the accumulation of lactic acid;
- 2) the frequency of muscular contractions may be much higher than with traditional gym, to get best results faster.

### Advantages of using the electro stimulator

**In beauty:** improvement of capillary micro circulation with more oxygen supplied to tissues, thus producing better fat mobilisation; toning up and firming up of tissues; draining of excess liquids; an increase in metabolism and cellular activities; a reduction of localised fatty tissues and cellulite.

**In sports:** isolation of the muscular group being treated; saving of physical and psychological energy; recruitment of a larger number of muscular fibres than is the case with voluntary contraction; training of under-used muscular fibres during normal work-out sessions; shorter recovery periods; protection of the body's system of tendons and muscles; improved blood circulation. Muscular damage due to electro stimulation is most unlikely, as long as the intensity is increased gradually. Any interference in the menstrual cycle (e.g. early, late, heavy or light periods) are extremely subjective and variable. We recommend, therefore, to avoid treatments in the abdominal area during the actual period and the days immediately before and after this. To date, no side effects have been noted concerning the use of electro stimulation during lactation. However, we recommend not treating the upper body area during this time.

**We strongly advise against treating any parts of the body with serious skin problems.**

### Results

**All results in the aesthetics field are naturally subjective.** We can, in any case, sustain that in the case of TONE UP, good results can be seen after just 15 days with 5-7 regular and constant weekly sessions; cellulite treatment will require 30-40 days. Better and quicker results may be obtained if treatments are added to suitable physical activity and a correct lifestyle.

## high technology

Tesmed Max 7.8 is designed and built with cutting-edge electronic components, a high storage microcontroller with great memory, processing and storage capacity. This means that the Tesmed Max 7.8 is an extremely professional product with superior performance, yet compact size. The software and the hardware in the Tesmed Max 7.8 guarantee a special self-calibration function for the output parameters to suit the different characteristics of every human body.

The Tesmed Max 7.8 programs include a sequential pulsed treatment.

This stimulation is an exclusive feature of the Tesmed Max 7.8 that, thanks to this draining, helps blood circulation remarkably, since the extracellular liquids and built-up fats are moved and can thus be removed by the body. This type of massage leaves your legs and arms feeling lighter and gives the skin an even look, at the same time considerably reducing the "orange peel" effect. All areas treated with the WAIMSSYSTEM® gain better tone and are firmer.

## Battery recharge

The battery is already installed in the device.

Charge the battery when the flashing battery symbol  appears, or the device switches off suddenly, even in the event that it switches off when the intensity is increased, and at the discretion of the user, after a certain number of treatments.

Battery life varies depending on the treatments performed, the number of channels used and the intensity of work.

To recharge the Tesmed, disconnect all the cables from the appliance, insert the battery charger into the electrical socket and connect the low voltage connector to the stimulator (Fig. 2 page 66).

When the Tesmed is connected to the charger, the screen will light up for a few seconds and the flashing symbol  of the battery will appear. Keep the device in charge for at least 10-12 hours to have an excellent battery life, regardless of the presence of the battery symbol  on the display, as the adapter continues to charge even in the case of a fixed or no symbol  .

The duration for a full battery recharge can therefore be 10-12 hours, depending on various factors such as: the external temperature, the battery life, as it was used etc. The charger can still remain connected to the Tesmed even after the charging has been completed, without causing any damage to the device.

**Battery life depends on use and retention mode. The battery is as a spare part subject to wear**

**Never recharge the equipment when the cables and electrodes are connected to a person and never recharge the battery by using a different battery charger: always use the one supplied with the equipment. Do not short-circuit the battery.**

### Battery replacement

Tesmed Max 7.8 works thanks to a rechargeable battery located at the back of the device. The useful life of the battery depends on several factors such as: type of use, number of refills made, period of inactivity, temperature, etc. The battery can be purchased as a spare part.

If the device is left unused for a long time, it may be necessary to replace the battery.

- To access the battery compartment, simply remove the removable belt clip and slide the cover in the direction of the arrow (Fig. 3 page 66).
- The battery is connected to the appliance via a connector; to replace it, remove it from its housing and remove the connector from the appliance (Fig. 4 page 66).
- When inserting the new battery pay particular attention to the connection: the connector has on its side a flat surface which must be inserted facing upwards (Fig. 5 page 66).

**NB: Used batteries must be disposed of properly to protect the environment.**

### Connections

The electrical pulses generated by the Tesmed Max 7.8 are transmitted to the muscles by means of self-adhesive electrodes. Before switching the Tesmed Max 7.8 on and before using the intensity command, always apply the electrodes to the body part to be treated and then connect them via the cables to the Tesmed Max 7.8: insert the pins on the bipolar cables in every pair of electrodes used for the stimulation and then connect them to the equipment by inserting them in the relevant colour coded seat on the rear of the Tesmed Max 7.8 (fig. 6 pag. 66). If this procedure is not followed, there is the risk that the equipment will exceed a set threshold and so produce an error message. Thanks to an in-depth study and the TMS "Tesmed Multiplexer used System" technology used in the Tesmed Max 7.8, the equipment is designed to work correctly without having to apply a specific polarity to the electrodes, thus making it easier to apply, connect and use.

### Disconnecting the electrodes

After treatment, disconnect the pins on the bipolar cables as shown figure 7 pag. 66. Take care not to pull the connection cables from the electrodes.

## equipment description (fig. 8 page 67)

### LCD display and keys

1	<b>Tesmed logo</b>	11	<b>ON/OFF and select channels double function key</b>
2	<b>Program number</b>	12	<b>Intensity symbol</b>
3	<b>Program symbol</b>	13	<b>Keyboard block and ON block symbol</b>
4	<b>Low battery symbol</b>	14	<b>Timer symbol</b>
5	<b>Frequency symbol</b>	15	<b>Treatment time and remaining time</b>
6	<b>Selected output channels (4)</b>	16	<b>Clock symbol</b>
7	<b>Relative intensity for each 4 output</b>	17	<b>Output channels</b>
8	<b>Function keys (MODE)</b>	18	<b>Battery recharger socket</b>
9	<b>Decrease key (  )</b>	19	<b>Battery door</b>
10	<b>Increase key (  )</b>	20	<b>Removable belt hook</b>

## use

The Tesmed Max 7.8 has **99** programs (01 to 99).

Programs **01** to **20** can be personalised and you can change the frequency and treatment time values.

Programs **21** to **99** are pre-set with a fixed duration and cannot, therefore, be changed.

Before using the Tesmed Max 7.8, you must choose the treatment type you want, using the personalised or set treatment tables, find the relevant program and then apply the electrodes to the body part you want to treat and connect them to the equipment.

### Switching on

To switch the Tesmed Max 7.8 on, press key  for approximately 2 seconds: the system runs a self-test procedure, during which the display shows all the available functions (fig. 9 page 68); then appears on the display what shown in fig. 10 page 68.

#### - "Display lighting" function:

*The display automatically switches off after about 20 seconds when the equipment is not in use. The display is lit again when you press any key.*

#### - "Self switching-off" function

*The Tesmed Max 7.8 automatically switches off is not used for more 5 minutes in order to save the batter*

### Selection of channels and programs

Press key  repeatedly to select the number of channels you want to use to suit the treatment and then key  to select the program (fig. 11 page 68).

### Programs 01 to 20

Programs 01 to 20 let you personalise the treatment time, once selected. Press key  and the relevant symbol **TIME** starts flashing on the display (fig. 12 page 68).

Use keys   to adjust the treatment time: min 1 minute to max 60 minutes. Press key  again to control the intensity and the symbol  will appear.

## Programs 21 to 99

After having selected a program from 21 to 99, press key to go straight to the intensity control.

### Intensity control

By "intensity" we mean the value of current emitted to the body part and thus the degree of stimulation causing the muscle to contract. Symbol appears on the display (fig. 13 page 68): you can now adjust the intensity using keys and (1-99). The intensity rises or falls on all the selected channels at the same time. When you set the intensity, the clock begins to flash indicating that the equipment is providing energy (fig. 14 page 68). You can, however, access each channel during use to control the intensity independently: simply press key to select the channel whose intensity you want to change and then use keys and to increase or decrease the value. Fig. 15 page 68 shows that the intensity values for the 4 channels can be different.

The display constantly updates the treatment time until completion, as well as showing all the set parameters, meaning that you can always control the treatment.

At the end of the treatment, the Tesmed Max 7.8 clears the intensity setting for all output channels. You can now switch the equipment off by pressing key for 2 seconds or set new parameters to begin a new treatment. Use keys and , to increase or decrease the value of any parameter (in steps of one unit). Keep pressed for faster increases/decreases.

**NB:** To perceive a change in intensity in a single channel or all channels, changes are only made when the muscle is contracting. This avoids the risk of uncontrolled or sudden changes in intensity. During the treatment you can quickly clear the intensity setting by pressing key briefly. All the default parameter settings are restored and so you can start increasing/decreasing the intensity again to resume the treatment.

### Frequency control (for programs 01 to 20)

Programs 01 to 20 let you correct the pulse frequency value at will. The "frequency" is the number of pulses per second. When you change the frequency, you feel an increase or decrease in the speed of stimulation. These programs are identified by a special symbol on the display (fig. 16 page 68).

After starting the treatment, press key for 3 seconds to enter the frequency control mode. Press keys and repeatedly to increase or to decrease the frequency. The symbol flashes (at a varying speed) to indicate frequency control mode. Press key once to quit frequency control mode.

**NB:** if in frequency control mode during a treatment and you want to clear the intensity, briefly press key twice. All the default parameter settings are restored and so you can start increasing/decreasing the intensity again to resume the treatment.

### Error warning

The Tesmed Max 7.8 automatically goes into error mode if an electrode comes loose. In this case, the equipment display will show "er" (flashing for about 5 seconds) against the channel with the current distribution problem (fig. 17 page 68). All pulses are stopped immediately and "00" appears as the intensity value on the display. The error message can mean that:

- There are no electrodes connected to the channel in error mode.
- The electrodes are not properly adhered to the skin: reposition the electrodes or replace if too old.
- The cable used to connect the electrodes is faulty: check by inverting two cables.

The system automatically resets itself once the intensity value increases. Make sure the electrode is connected properly and then use key  to select the channel in error mode and adjust the intensity. If the problem persists, the equipment repeats the error warning: check the state of the cables and that the electrodes adhere properly to the body. The error warning may be delayed when using low intensity values, or even not detected, as it is influenced by the difference in each person's resistance.

### "Keyboard block" function

Press key , and, at the same time, press key  and release. Symbol  appears on the display (fig. 18 page 68). Follow the same procedure to unblock the keyboard.

This function, when enabled, disables all the keys and so avoids the risk of accidental changes to the set parameters during treatment.

### "ON block" function

Press key , and, at the same time, press key , for 3 seconds. Symbol  and the Tesmed logo appear on the display (fig. 19 page 68). Follow the same procedure to switch the equipment on.

We recommend using this function when storing the equipment in its case or bag, thus avoiding the risk of accidentally switching it on.

## maintenance

The Tesmed Max 7.8 is a highly sophisticated product and must be handled with care. The maintenance tips below will help you ensure its long life.

- Do not allow it to get wet. Rain, humidity and liquids or condensation generally contain corrosive mineral substances that can damage the electronic circuits.
- Keep the Tesmed Max 7.8 out of the reach of children.
- Do not leave the Tesmed Max 7.8 in particularly hot rooms. Excessive temperatures can affect the life of the electronic circuits, damage the batteries and deform the plastic parts.
- Do not leave the Tesmed Max 7.8 in particularly cold rooms, as condensation could form when it heats up during use and so damage the electrical circuits.
- Do not open the Tesmed Max 7.8. Any tampering with the equipment by inexperienced people could damage it.
- Do not allow it to fall, be knocked or shaken, as the electronic circuits inside could be irreparably damaged.
- Do not use the battery for purposes other than those intended.
- Never use the battery charger or batteries if damaged.
- Do not short circuit the battery.
- Use only the battery charger provided. Contact Customer Service in the event of a malfunction.
- Never leave the battery in hot or cold places, such as a closed car in the height of Summer or during the Winter, as this will reduce its capacity and life.
- Never throw used batteries in the fire!
- Follow local regulations to dispose of used batteries, e.g. recycle them. Never throw them away with household waste.
- To clean the Tesmed Max 7.8 (with the equipment switched off), use a soft cloth and alcohol, but no solvents, as these could ruin the plastic parts.

Use only original spare-parts and products for maintenance.

Follow these maintenance times to keep the Tesmed Max 7.8 in top working condition:

- A)** Check the state of the electrode cables and check for signs of oxidation (monthly if used regularly; fortnightly if used intensely).
- B)** Replace the electricity conducting electrode as soon as these show signs of wear. The manufacturer's warranty is invalidated if the above maintenance is not done. The manufacturer guarantees a 5 year working life of the equipment provided it is used and maintained correctly. The equipment, batteries and its accessories must be disposed of properly (treat as special waste) to protect the environment.

## disposal

The equipment, batteries and its accessories must be disposed of properly (treat as special waste) to protect the environment.

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## how to choose the right treatment to suit individual needs

- You can choose the most suitable treatment from those in the table of treatments, consulting the list of photos for details of how to position the electrodes on the muscles in question.
- The photo number is shown next to each treatment, in the "Photo" column. If more than one photo is indicated for the same treatment (e.g. 1, 37, 40-44), select photo 1, 37, 40, 41, 42, 43 or 44 as required.
- We recommend that you do not treat a group of muscles with more than one session a day.
- We recommend that you do not exceed 2 hours of electro stimulation per day, even if involving different groups of muscles.
- **The treatment times shown in the tables is not binding: actual treatment times may vary considerably from one person to another.**
- If in doubt about which treatment to use, we advise you consult a doctor or a trainer.
- During the treatment, the output intensity is controlled automatically at each change in phase for greater safety. You can then increase/decrease the intensity at will.
- To ensure your safety, you cannot increase the intensity during the pause between one pulse and the next.

**NB:** The programs in the **Tesmed Max 7.8** are modulates to provide various stages\* of massage (1 to max 5). Some programs have one or more Waims-system<sup>®</sup> stages. During treatment (and especially with the Waims-system<sup>®</sup>) we always recommend that you adapt the intensity of each channel, as your sensitivity will vary various from one group of muscles to another.

\*The number of stages in a program corresponds to the number of wave types used in it.

## electrode polarization

Most commonly available electro stimulators adopt a common electrode standard: they either contain continuous current (DC) or generate modified waves (different positive/negative values, with more of one or the other). Hence the need to specify the polarity. After careful study and with the use of TMS "Tesmed Multiplexer used System" technology in the Tesmed Max 7.8 means that this equipment can work correctly without having to give a specific polarity to the the electrodes, thus making them easier for you to apply, connect and use.

### positioning of electrodes

OUT 1  
Blue

OUT 2  
Orange

OUT 3  
Green

OUT 4  
Yellow

## maintenance

Follow the same program as for the treatment, but only 2-3 times a week and not on two consecutive days, unless otherwise indicated by your doctor or trainer

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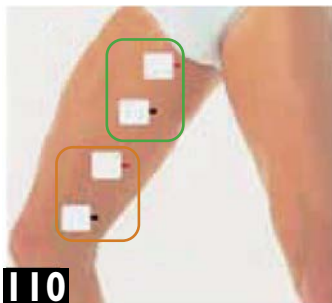
**-The larger electrodes are positioned in the same way as those shown.**

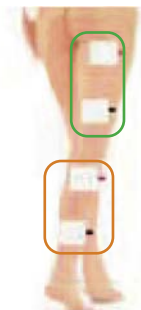
**Posizionamento elettrodi - Positioning of electrodes**  
**Positionierung der Elektroden - Positionnement électrodes - Colocación de los electrodos**

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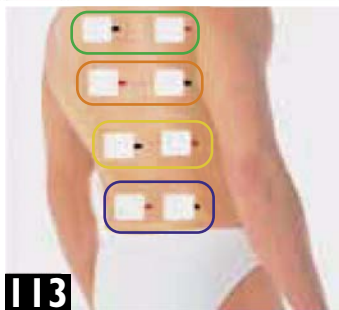
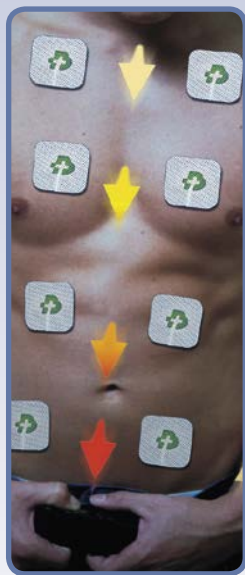
You can change the time and frequency of the programs in the table below at your discretion.

For programs from 1 to 5, we suggest following pictures of the electrode placement 10,13,14,33,34,56,75,76 (see page 70 and following)

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Prog.	Onda	Parametri	Indicazione Trattamenti	Treatment Indication	Indication Traitements	Angabe Behandlung	Indicación Tratamientos
1	Waïms-System®	(pause 1)	Linfodrenante, Rilassante, Recupero attivo, Defaticante	Lymph draining, Relaxing, Active recovery, Tiredness	Lymphodrainage, Relaxant, Récupération active, Défatigant	Lymphknotendrainage Entspannung Aktive Kräfterückgewinnung Erholung	Linfodrenante, Relajante, Recuperación activa, Desfatigante
2	Waïms-System®	(pause 2)					
3	Waïms-System®	(pause 3)					
4	Waïms-System®	(pause 1)					
5	Waïms-System®	(pause 2)					
6	Tapping 1	Tpulse=100µs Freq.=1÷100Hz Time=1'÷60'	Decontratturante, Rilattivante, Cellulite *	Decontracting massage, Reactivating, Cellulite *	Décontracturant, Réactivant, Cellulite *	Lösen von Verspannungen Reaktivierung Zellulitis *	Descontracturante, Reactivante, Cellulitis *
7	Tapping 2	Tpulse=200µs Freq.=1÷100Hz Time=1'÷60'					
8	Tapping 3	Tpulse=300µs Freq.=1÷100Hz Time=1'÷60'					
9	Tapping 4	Tpulse=400µs Freq.=1÷100Hz Time=1'÷60'					
10	Press 1	Tpulse=120µs Freq.=1÷150Hz Time=1'÷60'	Forza resistente, Forza esplosiva, Sviluppo forza	Stamina, Explosive force, Development force	Force résistante, Force explosive, Développement de la force	Widerstandskraft Explosive Kraft Kraftentwicklung	Fuerza resistente, Fuerza explosiva, Desarrollo fuerza
11	Press 2	Tpulse=240µs Freq.=1÷150Hz Time=1'÷60'					
12	Press 3	Tpulse=300µs Freq.=1÷150Hz Time=1'÷60'					
13	Kneading 1	Tpulse=40µs Freq.=1÷150Hz Time=1'÷60'	Tonificazione, Modellamento, Cellulite *, Rassaldamento	Tone-up, Modelling, Cellulite *, Firm-up	Tonification, Modelage, Cellulite *, Raffermissement	Stärkung Modellierung Zellulitis *, Kräftigung	Tonificación, Modelamiento, Cellulitis *, Rearfirmación
14	Kneading 2	Tpulse=160µs Freq.=1÷150Hz Time=1'÷60'					
15	Kneading 3	Tpulse=200µs Freq.=1÷150Hz Time=1'÷60'					
16	Kneading 4	Tpulse=340µs Freq.=1÷150Hz Time=1'÷60'					
17	Modulation 1	Tpulse=80µs Freq.=1÷150Hz Time=1'÷60'	Drenante, Stretching	Draining, Stretching	Drainant, Stretching	Drainage Stretching	Drenante, Stretching
18	Modulation 2	Tpulse=140µs Freq.=1÷150Hz Time=1'÷60'					
19	Burst 1	Tpulse=180µs Freq.=1÷150Hz Time=1'÷60'	Aerobico, Ipertonia	Aerobics, Winding-down	Aérobique, Hypertonie	Aerobik Hypertonie	Aeróbico, Hipertonía
20	Burst 2	Tpulse=260µs Freq.=1÷150Hz Time=1'÷60'					

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\* Before carrying out the treatments for cellulite, we suggest completing a cycle of toning treatments

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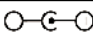
## TESMED MAX 7.8 TECHNICAL DATA

Power supply	Nickel-metal-hydrate (NI-MH) rechargeable battery 4.8V = 750 mA/h
Casing	Polycarbonate
Pulse repetition frequency	1Hz - 150Hz
Max energy per pulse	E= 3200 $\mu$ J
Max peak power / mean power	10 W per pulse / 5 mW
Mean current at max frequency Mean current at min frequency	2 mA 0,2 mA
Max peak current	120mA
Pulse shape	Rectangular, biphase (positive and negative), no continuous component
Pulse duration (Positive/Negative)	da 40 $\mu$ s a 400 $\mu$ s
Pre-set programs	99
Number of channels	4
Dimensions (mm)	128 x 58 x 18
Weight (gr)	145
Applied part: Low Voltage	BF
Conditions of use	Continuous use
<b>All electrical data are provided for a charge of 1000 ohm per channel</b>	

Equipment not suitable for use in the presence of inflammable anaesthetic mixtures with air or oxygen or nitrous oxide.

## BATTERY CHARGER TECHNICAL DATA

Electrical input	100-240V $\sim$ , 50/60Hz, 180mA
Electrical output	6V $\text{---}$ , 60mA
Polarity, external $\emptyset$ , internal $\emptyset$	 3mm, 1.1mm
Compliance with technical standards	EN 60601-1, 3a ed.

### Guide and manufacturer's statement - electromagnetic immunity

The Tesmed Max 7.8 electro stimulator is designed to work in the electromagnetic environment described below. The customer or user of the Tesmed Max 7.8 electro stimulator must guarantee that it used in such an environment.

Immunity test	IEC 60601 test level	Conformity level	Electromagnetic environment - guide
Electrostatic discharge (ESD) IEC 61000-4-2	± 8kV contact ± 15kV in air	± 8kV contact ± 15kV in air	Floors must be wooden, concrete or tiled. If floors are covered with a synthetic material, the relative humidity should be at least 30%.
Transiting / fast electric trains IEC 61000-4-4	± 2kV for power lines ±1kV for input/output lines	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery
Overvoltage IEC 61000-4-5	±1kV differential mode ±2kV common mode	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery
Voltage drops, brief cuts and variations in voltage on supply input lines IEC 61000-4-11	5% $U_T$ (>95% drop in $U_T$ ) for 0,5 cycles 40% $U_T$ (60% drop in $U_T$ ) for 5 cycles 70% $U_T$ (30% drop in $U_T$ ) for 25 cycles <5% $U_T$ (>95% drop in $U_T$ ) for 5 seconds	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery
Magnetic field at mains frequency (50/60 Hz) IEC 61000-4-8	30 A/m	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery

NOTE:  $U_T$  is the AC mains voltage rate prior to application of the test level.

### Recommended separation distances between portable and mobile radio frequency equipment and the Tesmed Max 7.8 electro stimulator

The Tesmed Max 7.8 electro stimulator is designed to work in an electromagnetic environment where irradiated RF interference levels are kept under control. The customer or the user of the Tesmed Max 7.8 electro stimulator may help avoiding electromagnetic interference by guaranteeing a min distance between mobile and portable RF communication devices (transmitters) and the Tesmed Max 7.8 electro stimulator, as recommended below and to suit the max output power of the radio communication devices.

Max output power of the specified transmitter (W)	Separation distance at the transmitter frequency (m)		
	150kHz - 80MHz	80MHz - 800MHz	800MHz - 2,5GHz
	$1,17 \sqrt{P}$	$1,17 \sqrt{P}$	$1,17 \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,17	1,17	2,33
10	3,7	3,7	7,37
100	11,7	11,7	23,3

Transmitters with a max output power not indicated above: the recommended separation distance in metres (m) can be calculated using the applicable equation for the transmitter frequency, where P is the max nominal output power of the transmitter in Watts (W) as indicated by the manufacturer of the transmitter and d is the recommended separation distance in metres (m).

**NOTE 1:** At 80 MHz and 800 MHz, the highest frequency interval is applied.

**NOTE 2:** These guidelines may not apply to all situations. Electromagnetic propagation is influenced by the absorption and reflection of structures, objects and people.

### Guide and manufacturer's statement - electromagnetic immunity

The Tesmed Max 7.8 electro stimulator is designed to work in the electromagnetic environment described below. The customer or user of the Tesmed Max 7.8 electro stimulator must guarantee that it used in such an environment.

Immunity test	IEC 60601 test level	Conformity level	Electromagnetic environment - guide
RF carried IEC 61000-4-6	3Vrms da 150kHz a 80MHz	Not applicable	Portable and mobile RF communication devices should not be used any closer to any part of the Tesmed max 7.8 electro stimulator, including the cables, than the recommended separation distance calculated using the applicable equation for the transmitter frequency. Recommended separation distance: $d = 1,17\sqrt{P}$ $d = 1,17\sqrt{P}$ 80MHz - 800MHz $d = 2,33\sqrt{P}$ 800MHz - 2,5GHz
RF irradiated IEC 61000-4-3	3V/m da 80MHz a 2,7GHz	3V/m 80MHz - 2,7GHz	where P is the max nominal output power of the transmitter in Watts (W) as indicated by the manufacturer of the transmitter and d is the recommended separation distance in metres (m). The intensity of fixed RF transmitters (as established by an electromagnetic survey of the site) could be less than the conformity level in each frequency interval. There may be interference near devices marked: 

**NOTA 1:** At 80 MHz and 800 MHz, the highest frequency interval is applied.

**NOTA 2:** These guidelines may not apply to all situations. Electromagnetic propagation is influenced by the absorption and reflection of structures, objects and people.

**A)** a. Field intensity for fixed transmitters, such as radio phone stations ( mobile phones and cordless phones) and walkie-talkies, radio ham equipment, AM and FM radio equipment and TV transmitters cannot be foreseen in theory or with precision. To assess an electromagnetic field caused by fixed RF transmitters, you need to carry out an electromagnetic survey of the site. If the field intensity found in the place where the Tesmed Max 7.8 electro stimulator is to be used exceed the applicable conformity level above, you will need to observe the normal working of the Tesmed Max 7.8 electro stimulator. If there are anomalies, extra measures will be needed, such as a different position or direction of the Tesmed Max 7.8 electro stimulator.

**B)** b. The field intensity in the frequency range 150 kHz to 80 MHz should be less than 3V/m.

### Guide and manufacturer's statement - electromagnetic emissions

The Tesmed Max 7.8 electro stimulator is designed to work in the electromagnetic environment described below. The customer or user of the Tesmed Max 7.8 electro stimulator must guarantee that it used in such an environment.

Emission test	Conformity level	Electromagnetic environment - guide
RF emissions CISPR 11	Group 1	The Tesmed Max 7.8 electro stimulator only uses RF energy for its internal functioning. Therefore, its RF emission is very low and is unlikely to cause any interference with any electronic equipment nearby.
RF emissions CISPR 11	Class B	—
Harmonic emissions IEC 61000-3-2	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery
Voltage fluctuation emissions / flicker IEC 61000-3-3	Not applicable	The Tesmed Max 7.8 electro stimulator is a device with DC power supply from an incorporated battery

Tesmed suggests: electrodes 50x100, the large size makes them ideal for big muscles.

Tesmed suggests:  
Tesmed Sport. You can use it with one channel.  
Maximum power for fast results.

Tesmed suggests: Tesmed Abs Pad.  
Use it with one channel.  
Try it with program 16 at the maximum  
level of power you can tolerate.

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### Information on use and technical assistance:

**info@tesmed.com**

(please specify the date of purchase)

**+ 39 035 662660**

every working day from 8.30 am to 12.30 am and from 1.00 p.m. to 5.00 pm

**WhatsApp Tesmed Fast Check Video**

**+ 39 345 6979093**

If appropriate, it is possible to send a short video concerning the device, to demonstrate any operating problems - the message must be accompanied by a brief written explanation, with a screen of the order number and the date of purchase. Service active on working days and hours.

After your purchase, to activate the warranty, write to: **warranty@tesmed.com** indicating in the subject: the order number - the date of purchase - Max 830

A copy of the manual in electronic format can be obtained on the **tesmed.com** website at the INFORMATION AND MANUALS page by selecting Manuals

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**Feldi S.r.l. - Italy**