# Termö

*Please read the instruction manual carefully*

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### Description

The Termö has been developed using the latest infrared technology allowing temperature to be taken at a distance of about 1 to 2 inches (3 to 5 cm) away from the body (forehead, ears and armpit). It also allows precise, instantaneous, non-contact and safe temperature measurement of the various surfaces such as bottle milk, food, bath water, etc. A tool that is both practical and essential in your care kit for the whole family.

**The non-contact infrared thermometer is not recommended for use with children under 2 years old.**

### Package Content

- 1 Non-Contact Infrared Thermometer
- 2 x AA batteries
- 1 carry case
- 1 instruction manual

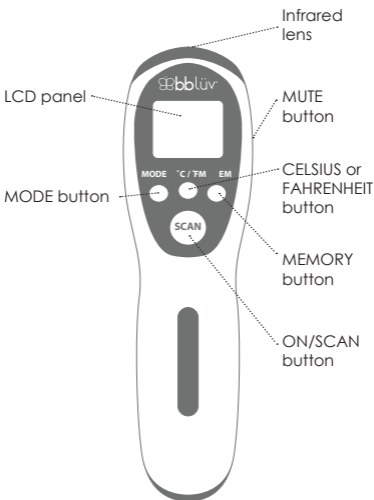
### Guidelines

This device complies with the ISO 80601-2-56 and European Standard EN60601-1-2:2007 and is subject to particular precautions with regards to electromagnetic compatibility.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. For detailed description of EMC requirements, please contact your authorized local Service Center.

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

## Description of components

## Important information

Please note that the temperature of the human body varies throughout the day. It can also be influenced by numerous external factors: age, sex, type and thickness of skin.

The temperature should be taken in a neutral environment.

The person must not have been undertaken vigorous physical activity prior to taking his/her temperature and the room temperature must be moderate. Also note that in sitting, temperature is lower by about 0.5 °F to 0.7 °F (0.3 °C to 0.4 °C) than when standing.

## Before use

### 1. Installing the batteries

- Install batteries in the battery compartment by pushing on the battery compartment door and sliding it towards

the direction indicated by the arrow.

Insert 2 AA batteries (included) in the compartment making sure to respect the polarity and put back the door by sliding it back in place.

- Follow this step whenever the LCD screen displays the flashing low battery symbol which means batteries need to be replaced.

## **2. In order to obtain reliable and stable results:**

- For the first use or each time there is a significant change in the ambient temperature, please wait from 10 to 15 minutes for the Termö to acclimate to the new environment.
- Allow 3 to 5 seconds interval between two measurements.
- Push back hair from the forehead and wipe away any perspiration.
- Avoid any drafts (e.g. from air conditioning, etc.).

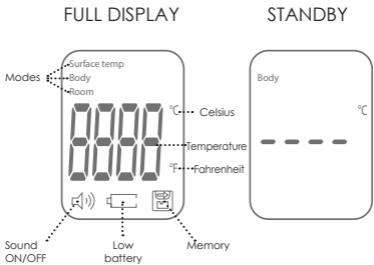
**The surface temperature differs from the internal body temperature. To obtain the internal temperature, always use the BODY mode. Please make sure to select SURFACE TEMP mode for an external area reading.**

## **Getting started with the Termö**

Your Termö is designed to let you measure different types of temperature by letting you choose the proper mode beforehand.

1. Press the ON / SCAN button for one

second, after that, the screen panel is in full display for one second and then it goes on Standby showing your selected options.

2. Press the °C/°F Button to select the desired temperature unit.
3. Press the MODE Button to change the type of temperature you want to measure.
  - SURFACE TEMP mode: aim at any surface (liquid or solid);
  - BODY mode: aim at any part of the body, usually forehead, ears and armpit (never aim at the eyes!);
  - ROOM mode: just press on the SCAN button and the room temperature will be displayed.
  - Aim the lens towards the chosen surface or body part to read at a distance of 1 to 2 inches (3 to 5 cm) and press the ON/SCAN button to take a reading within 1 second.
4. In order to save the batteries, the Termö will shut off automatically after 30 seconds.
5. Press the MEM Button to recall saved measured value. The Termö can store up to 32 measurements.

### Instructions for recalibration of the Termö

When there is a difference between Termö and a calibrated mercury thermometer, you can use the recalibration function to adjust

the Termö to make it the same as the calibrated mercury thermometer. To do so, please follow these simple steps.

1. While the device is on, press the MODE and °C/ °F buttons simultaneously for 3 seconds.
2. The screen will then display: F-4.
3. Press the MODE button to increase by 0.1 °F (0.1 °C),
4. Press the °C/ °F button to reduce 0.1 °F (0.1 °C).
5. Press the MEM button to save the setting.

## **Maintenance**

- The protective glass over the lens is the most important and fragile part of the Termö, please take great care of it. Clean the glass with a cotton ball, wet with 95% alcohol when needed.
- Only use AA batteries.
- Remove the batteries when thermometer is not used for an extended period of time.

## **WARNINGS**

- Follow the maintenance advice found in this instruction manual to insure reliable stable results.
- The Termö may be used for professional purposes or for personal use.
- The Termö must be used for the purposes described in this instruction manual, in an ambient temperature ranging from 50 °F (10 °C) to 104 °F (40 °C).
- The Termö must always be kept in a clean, dry area.
- Do not expose the Termö to electric shocks or to extreme temperatures over 131 °F (55 °C) or under -4 °F (-20 °C).
- Do not use the Termö when relative humidity is higher than 85%.

- Do not touch the glass of the infrared lens with your fingers or point directly in the eyes.
- Do not expose the Termö to sunlight or water.
- Never drop the Termö, any impact may damage the product.
- If a problem occurs with your Termö, contact your retailer. Do not attempt to repair it yourself.
- The device should not be used shortly after exercise, bathing or coming indoors.
- The device should not be used to manage important health concerns.
- Parents should not rely only on temperature readings; if they have concerns they should seek medical advice.

## Troubleshooting

If you have problems while using your Termö, please refer to this guide to help resolve the problem. If the problem persists, please contact our customer service at (+1) 855-808-2636 or [info@bbluvgroup.com](mailto:info@bbluvgroup.com)

1. *The screen displays temperature higher than 109.2 °F (42.9 °C).*  
The temperature is in Fahrenheit. Change the measurement to Celsius by pressing the °C/ °F button.
2. *The screen displays temperature lower than 89.6 °F (32 °C)*  
If the Termö is in SURFACE TEMP mode, the temperature displayed is showing the external temperature of your body, rather than the internal. To take the internal temperature, press the MODE button and set to the reading BODY mode.
3. *The screen displays the message HI*  
When using the Termö, the message HI can show on the screen. In this case, the temperature is above the measurement range selected, for example above 109.2 °F (42.9 °C) in BODY mode.

#### 4. The screen displays the message LO

When using the Termö, the message LO can show on the screen. In this case, the temperature analyzed is under the measuring range selected, for example less than 89.6 °F (32 °C) in BODY mode. This message can also be displayed for various reasons. Please find below a list of the possible issues:

<b>Reasons for LO message display</b>	<b>Advice</b>
Temperature reading hampered by hair or perspiration.	Make sure there is no obstruction or dampness prior to taking temperature.
Temperature hampered by an air draft or dramatic change in ambient temperature.	Make sure there is no air blowing in the area of use; this could affect the infrared reading.
Temperature readings are too close together, and the thermometer did not had time to reboot.	Pause of 3 to 5 seconds minimum between readings; a 15 second pause is recommended.
The measuring distance is too far.	Take measurements at the recommended distance: approx. 1.2 to 2 in (3 to 5 cm).

### **Technical Specifications**

#### NORMAL USE CONDITIONS

- Ambient temperature: 50 °F to 104 °F (10 °C to 40 °C)
- Relative humidity: ≤ 85%

#### BATTERIES

- 2 x AA batteries (Included)

#### UNIT SIZE

- 6.9 in x 1.9 in x 2 in (17.5 cm x 2.5 cm x 5 cm)

#### UNIT WEIGHT (including batteries)

- 0.32 lbs (0.15 kg)

#### TEMPERATURE DISPLAY ACCURACY

- 0.1 °F (0.1°C)

## MEASURING RANGE

- In BODY mode: 89.6 °F to 109.2 °F (32 °C to 42.9 °C), there are three color-identifiers for temperatures:
  - GREEN color backlit:  $\leq 99.1$  °F (37.3 °C), means normal temperature;
  - ORANGE color backlit: 99.3 °F to 100.2 °F (37.4 °C to 37.9 °C), means low fever;
  - RED color backlit:  $\geq 100.4$  °F (38.0 °C), means high fever.

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