## Theramics® Hot & Cold Therapy System

**⚠ CAUTION:** Follow these instructions carefully. Misuse can cause burns. Overheating the pad in the microwave can damage or destroy product. If any swelling of the pad is observed while heating, turn off the microwave immediately and wait for swelling to go down before opening oven.

Before heating, the pad should be at room temperature. If you store the pad in the freezer follow instructions for HEATING FROM FREEZER below.

Fold the pad in the cloth cover. Place in the center of the microwave at HIGH power setting for the times indicated in chart:

### 6" x 12" TherMiPaq

600 Watt Microwave	800 Watt Microwave	1000 Watt Microwave	1200 Watt Microwave
90 seconds	75 seconds	60 seconds	45 seconds

If microwave **DOES NOT** have a turntable, rotate the pad 90 degrees halfway through the heat time. These settings will produce a "Low" heat setting. Test pad prior to use. If it is too hot, allow to cool to desired temperature.

**⚠ CAUTION:** Maximum heat of the pad is reached approximately 3 minutes after removal from microwave. If hotter pad is desired after this time, heat in microwave for 15 seconds at High power setting and test again. Repeat this process until desired temperature is reached.

**REHEATING:** If the pad is to be reheated after 30 minutes of use, follow above instructions, but reduce microwave heat time by half.

**HEATING FROM FREEZER:** Place the pad in the microwave at defrost or 30% Power Level for double the amount of time in chart above. Allow to stand 5 minutes, then follow normal heating instructions.

**FOR COLD THERAPY:** Place the pad in your freezer. The pad will reach the temperature of your freezer after one hour; however, it will remain flexible and form to your body. Comfort level could last up to one hour. Apply the pad to the area you wish to cool. Should there be discomfort, discontinue use and consult your physician.

Manufactured by Compass Health Brands, Middleburg Heights, OH 44130 USA.  
www.compasshealthbrands.com • (800) 526-8051 Made in China

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