
KAHUNA
MASSAGE CHAIR®
HANI
Massage Chair Manual



KAHUNA  CHAIR

CONTENTS

Please read and follow all safety and operating instructions listed in this user manual prior to use. Basic precautions should always be used when operating an electrical appliance.

Safety Instructions	2
Product Specifications	4
Product Components	5
Chair Installation	8
Operation Guide	10
Trouble Shooting	13
Maintenance	14

SAFETY INSTRUCTION

Warning

The plug must be fully inserted into an appropriate outlet. Improper connection of the equipment-grounding conductor can result in the risk of electric shock. Check with a qualified electrician or service technician, if you are in doubt as to whether the product is properly grounded. Do not modify the plug provided with the product. If the outlet is not suitable. Please ensure that a proper outlet is installed by a qualified electrician. When using the massage chair, basic precautions should always be followed to reduce the risk of electric shock, burns, fire, or injury to person.

1. Please ensure that you plug the massage chair into a proper electrical outlet.
2. Make sure the power plug is properly inserted into the proper electrical outlet to reduce the risk of short-circuit and fire.
3. Always unplug the massage chair from the electrical outlet immediately after use, before cleaning, etc.
4. Do not use the massage chair in places where the humidity is high.
5. Do not use the massage chair outdoors and avoid place with direct sunlight.
6. If the power turns off during use make sure to switch the power OFF position and pull the power plug out from the socket to prevent damage from occurring when the power is restored.
7. Do not operate the massage chair if the power cord is wet or damaged.
8. Do not plug/unplug the power cord from the socket with wet hands.
9. Supervision is necessary when the massage chair is being used by or near the children or disabled persons.
10. Use the massage chair only for its intended use as described in this user manual.
11. Do not use any parts not from factory.
12. Do not stand, sit or jump in the calf rest, back rest and armrest when the massage chair is in use to prevent any injury.
13. Do not attempt to stand on the backrest and armrest to prevent any injury to person.
14. Be carefully not to fall asleep during the massage session
15. Always keep the massage chair clean and never insert any object into the gaps on and around the massage chair
16. Do not move the massage chair by pulling the power supply cord.
17. Always check whether the surface of the massage chair is torn or ripped before usage, if you find any unfamiliar rips in any area of the cloth in the backrest, do not use the massage chair, kindly contact the service centre immediately for repairs.
18. This appliance is designed for household use only.
19. For those suffering from illnesses, please consult your doctor for advice prior to use.

SAFETY INSTRUCTION

20. The following persons are requested to consult a doctor prior to use in order to reduce the risk of user feeling sick or suffering from injury.
 - Those who have malignant tumor
 - Those who suffer from heart disease
 - Those who are experiencing pregnancy, menstruation and puerperal fever
 - Those who require complete rest
 - Those who are under medical treatment
 - Those who suffer from back problem or whose backbone is abnormal or bent
21. If you experience pain during usage, stop using the massage chair immediately, if any injuries occur, seek medical treatment immediately.
22. Never massage the swollen or inflamed parts of the body.
23. This product is non-professional appliance designed to provide a comfortable massage. do not use this product as substitute for medical treatment.
24. Each massage session should not exceed 30 minutes. massage the targeted area only for 15mins, over-massaging can lead to overheating, thus giving the appliance a shorter lifespan.
25. Do not operate where aerosol(spray) product are used or where oxygen is being administered
26. The massage chair is not suitable for children aged 12 and below

NOTE


- This appliance is not intended for use by persons(including children) with reduced physical sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- If the supply cord is damaged, it must be replaced by manufacturer, its service agent or similarly qualified persons in order to avoid a hazard.
- The appliance heats up during operation. persons sensitive to heat must take extra caution when using the appliance.

PRODUCT SPECIFICATIONS

Description	Specifications
Product Name	Kahuna Massage Chair
Model	HANI
Rated Voltage	110-120V~
Rated Frequency	50/60Hz
Rated Power Consumption	50 W
Dimension (LxWxH) Inches	38.6 x 26.8 x 37.4 inches
Weight (LBS)	Net Weight: 70.5 lbs Gross Weight :94.8 lbs


PRODUCT COMPONENTS

Main Body Diagram (Front)

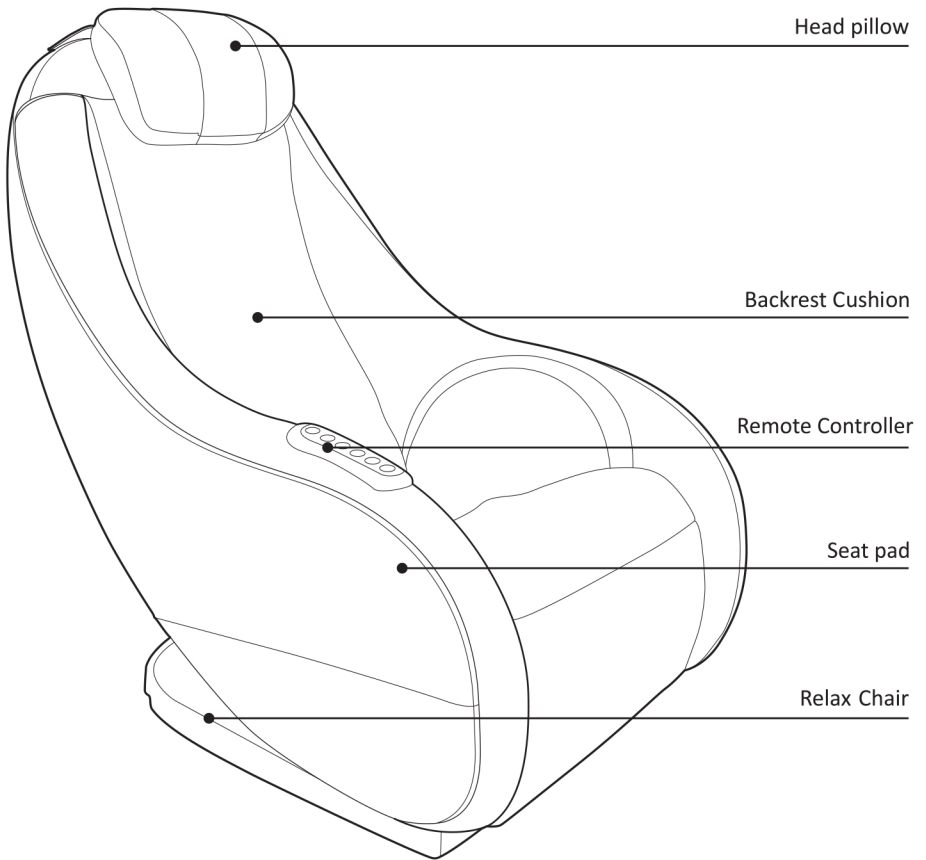


PRODUCT COMPONENTS

Main Body Diagram (Rear)

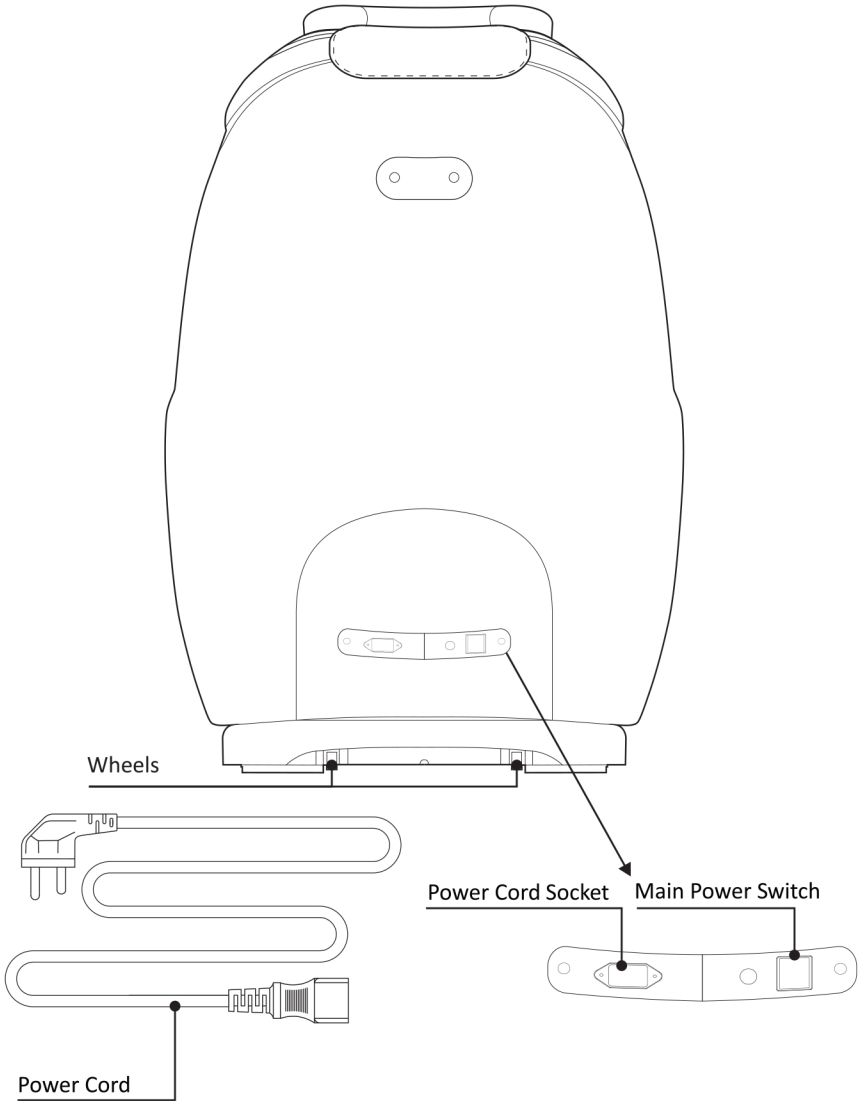

PRODUCT COMPONENTS


Part of the chair

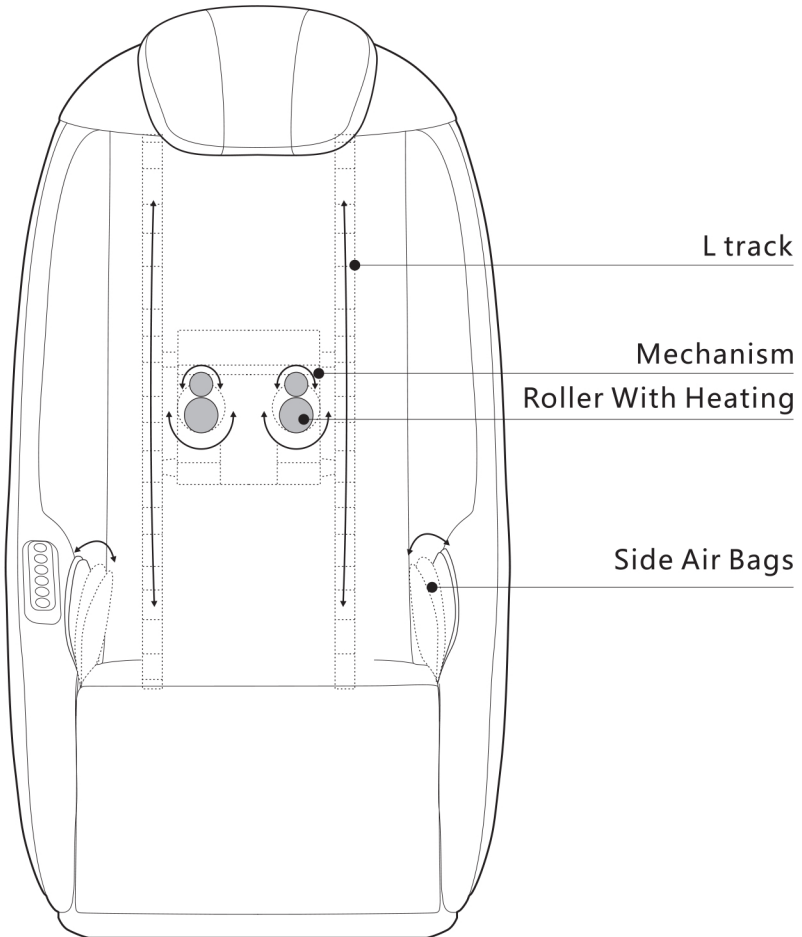

*Massage chair have heating function, not sensitive person must be used with attention.


CHAIR INSTALLATION

1. Remove the package and get out the spare parts

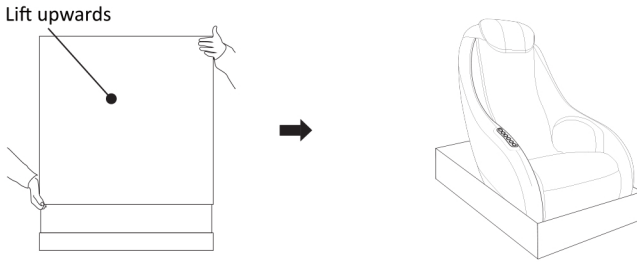


- 1) Remove the carton; Take out the package.
- 2) Install the pillow, backrest cushion, cushion and spare parts.

2. Place the massage chair in an open area

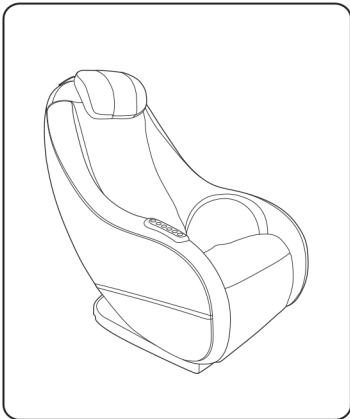


- Allow sufficient space around the chair for recline purposes.
- *To avoid the damage of the floor and to reduce the noise, please put area rug under the massage chair.
- *Make sure that there are no barriers at the front and the back of the massage chair.
- *Please do not put it in wet or uneven floor.
- *To avoid signal interference, please place it 4 feet away from AV, TV and radio.

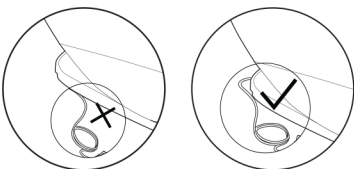

Warning
Do not put the power cord under of HANI chair.

CHAIR INSTALLATION

3. Double check the installation steps, smooth out any creases in the cushions.
4. Moving the massage chair.

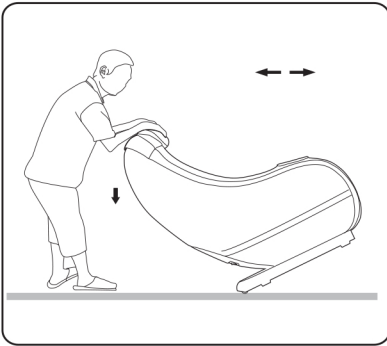


1) As illustrated, push the backrest downwards and tilt chair on to its wheels at a 45° angle. Slowly push and maneuver the massage chair to desired location. Lower massage chair slowly and smoothly until it is fully flat on the ground.

CAUTION

- * Do not move massage chair when occupied or in use.
- * Moving the massage chair may cause damage to the floors. Floor protection is recommended (e.g. rug or carpet).
- * Do not drop or suddenly let go of the chair when moving as it may damage the chair.
- * Two people are recommended when moving and extra care when handling the massage chair.

OPERATION GUIDE

1 Remote Controller

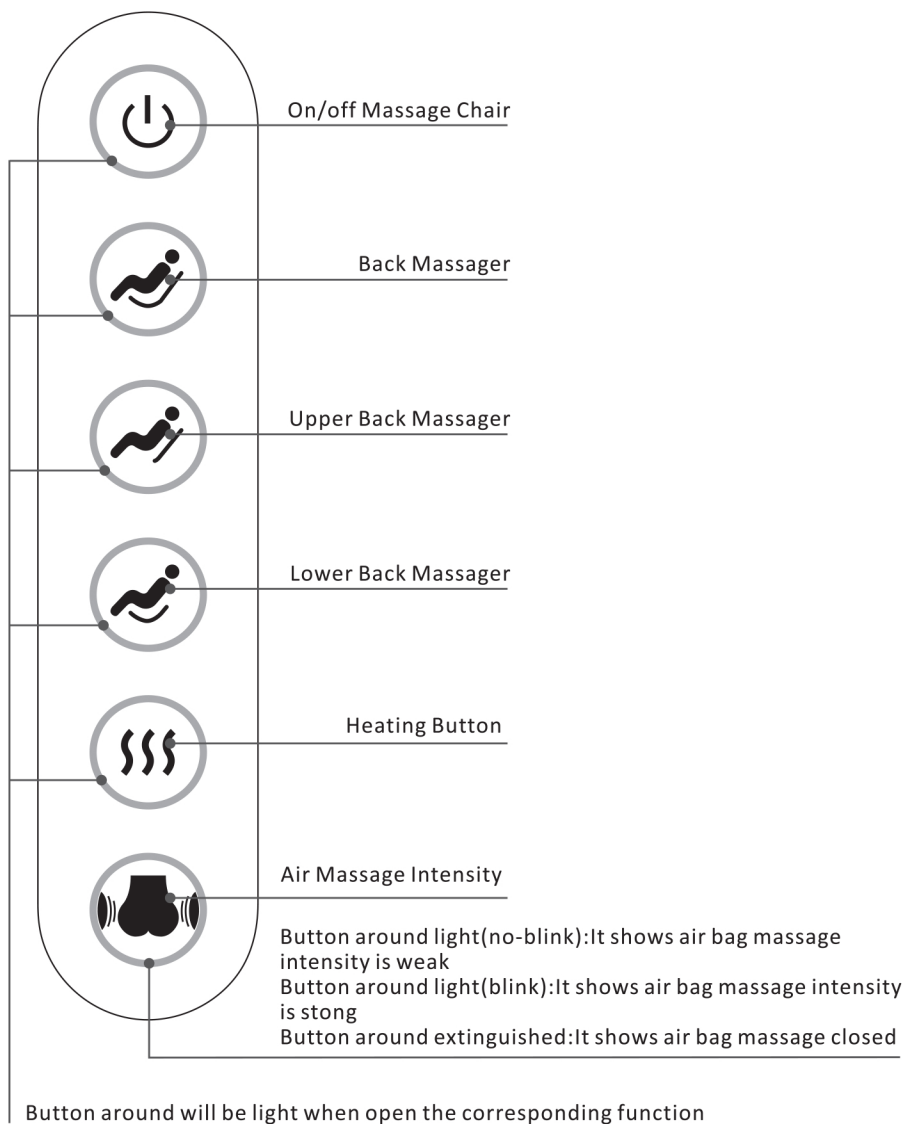

OPERATION GUIDE

1 Function description of Controller buttons



Press the power button to turn on the massage chair.
Press again to power off, massage chair will then return to default upright position.



Press the button to switch on/off the Back Massager mode.



Press the button to switch on/off the upper Back mode.



Press the button to switch on/off the lower Back mode.




Switch on/off for heating function.




"Intensity" Air Intensity: Press to adjust intensity of air massage, with 2 intensity levels to select from.

OPERATION GUIDE

2 After use

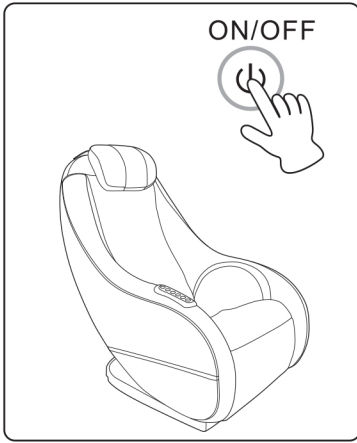

1. Press the power button to power off the massage chair.

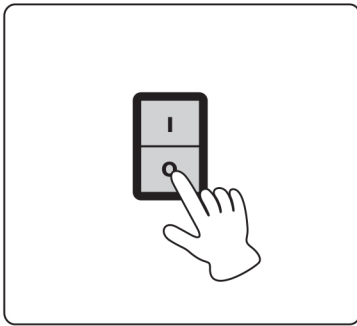

2. Turn off the main power switch.

3. Remove plug from power outlet.

* Please do not pull the power cord to remove the plug.

* Please do not unplug with wet hands.


TROUBLE SHOOTING

If the sounds or movements are not normal, please turn off the massage chair and remove the plug from the electric outlet. Please contact the customer service department for further assistance.

855.646.9500 / 1.714.388.2625, service@kahunachair.com


Problem	Solution
The massage chair is connected to power but is not functioning.	Check to see if the power plug is inserted or if the power outlet is functioning properly.
The remote controller is responding, but the massage chair is not functioning.	The massage chair may have automatically shut down due to overheating protection devices. Please turn off the power and allow the chair to cool off for 30 to 50 minutes.
Both the remote controller and massage chair are not responding.	The massage chair automatically turns into "full stop" mode when a massage program is completed. Please press the power button again to turn off the chair.

MAINTENANCE

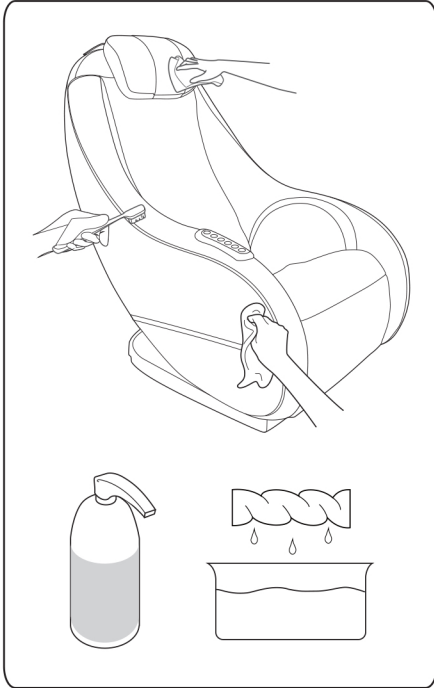

Use dry cloth to clean

Turn off and pull out the plug before cleaning the massage chair, do not touch the plug with wet hands.

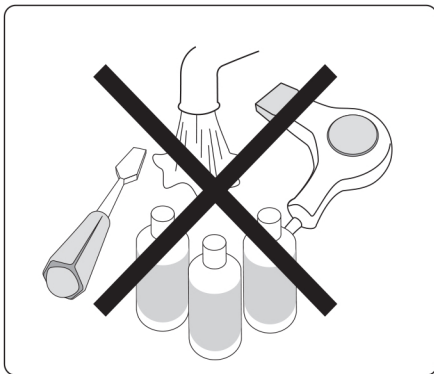

Prohibit using liquid cleaning products like water, alcohol, or other cleaning product to maintain a good quality.

HANI

Product Massage Chair

Model HANI


Developed by AJX Distribution LLC

7001 Village Dr #250

Buena Park, CA 90621

Tel 1-855-646-9500

Email INFO@KAHUNACHAIR.COM

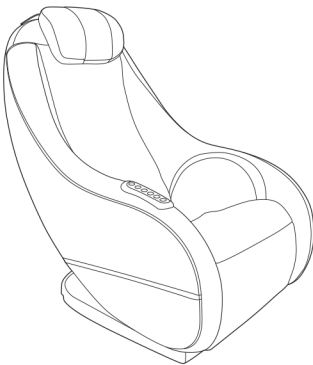

INSTRUCTION MANUAL
MODEL:HANI

Plaease read this user manual thoroughly prior to operating the massage chair. Store this booklet for future reference.