



RXCRUSH® PILL CRUSHER CLEANING INSTRUCTIONS

**RXCRUSH PILL CRUSHER DEVICE IS COMPATIBLE WITH
MOST HOSPITAL CLEANERS AND DISINFECTANTS.**

Clean the Pill Crusher frequently to keep pill residue from accumulating on the Crusher. Wipes are used to clean the surfaces and sprays clean the joints of the crusher. Crusher can be cleaned in a dish washer.

Approved cleaning agents contain one of the following:

- Common detergents and soap-based cleaners
- Mildly acidic cleaners
- Isopropyl Alcohol solutions > 60% alcohol
- Ammonia based cleaners
- Solutions with Hydrogen Peroxide (less than 5% HPO)
- Sodium hypochlorite based solutions (Clorox etc.) – Chlorides may cause discoloration of logo/text

Commercially available cleaning agents, (non-exhaustive list):

- PDI – Super-Sani Cloth® Germicidal Disposable Wipes
- Clorox® Healthcare Hydrogen Peroxide Cleaner Disinfectant Wipes
- PeridoxRTU® Sporidical Disinfectant and Cleaner
- Sporidican® Disinfectant FG Clean Wipes and High Level Disinfecting Solution
- CaviWipes1TM
- PREemptTM RTU Disinfectant Solution and Wipes
- Metres 13-5024 CaviCide Surface Disinfectant Cleaner
- Sani-Cloth®
- Oxivir® TB Wipes

The following chemicals are not recommended for use with RxCrush Pill Crusher:

- Acetone
- Phenol (phenolic disinfectants)
- Iodine solutions

We're Here to Help

To reorder or for assistance, visit us at www.RxCrush.com or call +01.877.343.2783.

