

DERMASTIR Post-Op Hyaluronic Face Mask

DERMASTIR Hyaluronic Invisible Face Mask is an aesthetic device used post-op by dermatologists and plastic surgeons. The unique formula produces a protective barrier and collagen production is stimulated.

The mask is hydrophilic and it diminishes the TEWL since it can retain moisture levels up to 100 times its own dry weight. The formula was designed to inhibit enzymes that break down hyaluronic, collagen and elastin.

Take care of the skin in post-op phase is essential since it lets a practitioner close properly the work and soothe the skin after invasive or non-invasive treatment.




DERMASTIR®

WWW.DERMASTIR.COM



MONODOSE
MASK

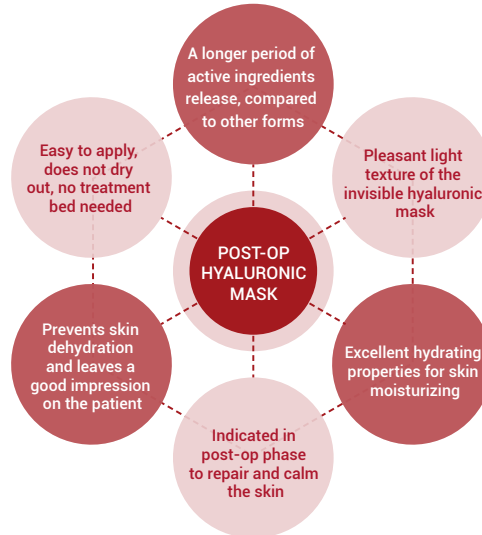

Quantity: 1 mask x 25 ml

LABORATOIRES
ALTA
CARE

6, Piazza di Spagna - 00187 Roma, Italia.
www.altacare.com

Dermastir Post-Op Hyaluronic Face Mask



POST PEEL



UNEVEN SKIN
SURFACE



DRY SKIN



ANTI WRINKLE



POST LASER / POST FILLER




POST BOTOX



Skin Mechanism



← A lack of hyaluronic acid leads to less elasticity of your skin, which increases the likelihood of cartilage destruction and saggy, wrinkled skin.

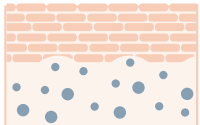

← Stimulation of collagen synthesis.

← The increase of HA content in the dermis leads to a plumping effect to fill in wrinkles.

Hyaluronic Acid

The hydration provided with the HA is extremely important, since the lack of moisture in the skin may lead to its drying out and cause premature signs of ageing.

Soluble Collagen

Reduced signs of ageing, it's smooths expression lines and fine wrinkles.

Aloe Vera Extract

Known for hydrating, emollient, calming properties.

Tocopherol (Vitamin E)

Radical scavenger. Known for its strong antioxidant properties.

DIRECTIONS FOR USE



Apply once or twice a week or post-op for 15-20 minutes.

1. Cleanse your face to remove any make-up, dirt or oils. Rinse well and pat dry with a towel.
2. Remove the outer layer of the hyaluronic invisible mask's packaging to reveal the invisible hyaluronic mask inside.
3. Place the hyaluronic mask on the face. Pat it gently to ensure it adheres to the skin.
4. After 15-20 minutes, remove the mask. Massage the rest of the serum on the skin. Massage the rest of the serum on the skin.
5. Proceed with the normal skin care routine without washing off the serum.

