



THE SILVER LIFE MASK

CARE SHEET

THE SILVER LIFE MASK is triple-layered as recommended by the World Health Organisation, (WHO). It provides personal protection for individuals against viruses, (including human coronavirus), bacteria and fungi. Up to 99.9% removal of these microbes occurs. Particulates are removed at the same rate down to a size of 3 microns. The material used in the mask is silver, a very well known anti-microbial element. The mask will remain odour free and may be washed for re-use up to 100 times. In 2021 the Silver Life Masks gained CE accreditation as a Type II PPE.

HOW TO USE:

PUTTING ON THE SILVER LIFE MASK:

1. Please ensure you wash your hands with soap and water or use a hand sanitiser prior to handling the mask.
2. Place the elasticated loops over your ears and adjust for comfort.
3. Ensure the mask is covering your nose and chin.
4. Try to avoid touching the mask once it's in position.

REMOVING THE MASK:

1. After use, DO NOT touch the front of the mask.
2. Remove the mask from your face using the ear loops, alone.
3. After removal, ensure you wash your hands thoroughly.

CLEANING YOUR MASK:

1. It is recommended that you wash your SILVER LIFE MASK at least once per week.
2. Hand wash in warm soapy water or in your washing machine on a full cycle at 40°C.
3. The SILVER LIFE MASK will retain its anti-microbial ability for up to 100 washes.
4. DO NOT TUMBLE DRY.
5. DO NOT BLEACH.



Wash at
40°C



Iron
Friendly



DO NOT
Tumble Dry



DO NOT
Bleach

DISCLAIMER:

For Customer Support, please visit:
www.spscience.co.uk

1. This product is a 3D knitted face-mask made from a silver cotton-polyester textile in which the polyester has been 100% recycled.
2. This face-mask **IS** classed as a Type II PPE under the CE accreditation system.
3. The textile has been successfully tested and proven to kill up to 99.9% of viruses, (incl human coronavirus), bacteria and fungi and is recognized by the UK H&SE as both a chemical and physical barrier.
4. Each mask is intended to be used individually and should not be shared.
5. Please follow instructions for use and cleaning carefully.