Size Adjusters
S M L
S M L
L R
Z Zones
Applicator

Read Instructions Carefully Before Use
The Z Zones must be fitted correctly for them to work effectively. Inserting anything into your ear must be done slowly and with care. By fitting the different sized Size Adjusters into the Z Zone silicone ear plugs, one can ensure they fit most ear canals perfectly and this process only needs to be done once.
How To Fit
1) Take the Z Zone for the right ear denoted by the letter "R" printed on the inside of the ear plug.
2) Gently push the small Size Adjuster into the Applicator (See Fig. 1) and insert into the Z Zone (See Fig. 2).
3) Ensure each Size Adjuster is pushed as deep into the Z Zone as possible before fitting them into the ear (See Fig. 3).
4) Double checking the orientation is correct (See Fig. 4). Take the Applicator with the Z Zone attached and carefully insert into the ear canal. (A gentle pull of the ear with the opposite hand opens the ear canal making insertion easier). Carefully pull back the Applicator to leave the Z Zone perfectly fitted inside the ear (See Fig. 5)
5) Repeat steps 1-4 with the left Z Zone denoted by the letter "L".
6) Once fitted, brush your palm over your ears. If you feel the Z Zone's edges, it is possible the orientation is not correct. Check and re-insert if necessary.

Enlarging The Z Zone To Improve Fitting
If the Z Zone feels too loose in the ear canal, the small Size Adjuster may be too small and a larger size may be preferred. Follow the steps below to change the Size Adjuster to your preferred size.
1) Pull the small Size Adjusters with fingers from both Z Zone to remove. Note: Z Zones are made from highly durable silicone and are extremely elastic. They will stretch significantly when removing the Size Adjusters.
2) Follow steps 1-5 of HOW TO FIT to insert the medium or large Size Adjuster until the fit is comfortable.

Care Instructions
• These Z Zone are made from durable silicone and therefore should be hand washed in mild detergent liquid to remove any residue and disinfect.
• This product may be adversely affected by certain chemical substances. Further information should be sought from the manufacturer.
Recommendations
• Ensure that the Z Zone are fitted, adjusted and maintained in accordance with the manufacturer's instructions.
• For temporary use only.
• The Z Zone can be worn at all times in noisy surroundings.
• The Z Zone should be checked on a regular basis for any signs of damage. If damaged the Z Zone should not be inserted into the ear.

Storage
• The Z Zone should be cleaned before and after use and kept in the carry case supplied.
Attention Table

Frequency (Hz)	63	125	250	500	1000	2000	4000	8000
Measured Attenuation (dB)	16.8	18.7	18.4	18.8	21.3	24.6	29.5	31.0
Minimum Permitted Level (dB)	-	5	8	10	12	12	12	12
Mean Attenuation (dB)	19.6	21.3	21.8	21.4	24.3	27.1	32.5	35.3
Standard Deviation (dB)	2.8	2.6	3.4	2.6	3.0	2.5	3.0	4.3
Assumed Protection (dB)	16.8	18.7	18.4	18.8	21.3	24.6	29.5	31.0

H = 26 SNR = 25 L = 20

Important Warnings:
• If the recommendations are not followed the quality and protection offered by the product may be severely impaired.
• Do not push the Z Zone into your ear canal incorrectly, (by hand or otherwise). Never insert Size Adjuster directly into the ear (See Fig. 5).
• Keep out of reach of children.
• Children should be supervised by an adult when fitting and done so in strict accordance with these instructions.
EN352-2 : 2002 Design Go Ltd, Unit 1 Mill Hill Ind. Est. Flower Lane, Mill Hill, London NW7 2HU.

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