

PRODUCT SIZING

ModelNo. 1XY-06ZZ

Select desired packaging, style, and length

Packaging	X	Style	Y	06	Width	ZZ	Length
Bags Only	3	Bags Only	0	06	06" 15cm	07	06" 15cm
DFP	4	2-Slot DFP	2	06	06" 15cm	10	10" 25cm
		4-Slot DFP	4	06	06" 15cm	12	12" 30cm

Note: All options are available in 2 packs and bulk quantities of 24 and 48

FIRST ICE® FEATURES:

- Maintains ideal on-patient temperature for up to 5 hours
- Patented technology that lasts 5x longer than gel
- Malleability to conform to the targeted area
- Reusable for up to 50 freeze-thaw cycles
- Snow-like material eliminates the thermal shock of gel
- Helps to mitigate the need for opioid-centric treatment
- No pumps or hoses, allowing for flexibility and mobility in use
- Safe, Non-Toxic, and repeatably effective

DIRECT STICK™ FEATURES:

- Patent-pending technology allows application anywhere on our device
- Soft thermal barrier for comfortable direct skin contact
- Intuitive design for quick and straightforward application
- Eliminates messy and bacteria-prone pockets

FIRST ICE® DIRECT STICK™ PACKS

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Inflammation
- Pain following Injury or Surgical Procedure

WARNING: This product is intended for single patient use at the direction of a medical professional and does not prevent or reduce the risk of injury. This product is not intended for internal use. If ingested, seek medical attention and treat symptomatically. This product is not microwave safe. Do not directly heat. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CAUTION: This product is non-toxic and is intended for external use only. Do not apply plastic barriers directly to the skin. For use on intact skin only. Each pack will last for approximately 40 freeze-thaw cycles and will incrementally become less soft with each application. If this is the case, mix contents prior to freezing and allow brief thawing before use. If a pack begins to freeze hard, dispose and replace with a new pack. DISCARD IF PUNCTURED OR TORN.

STORAGE: This product is sensitive to direct light and heat. For best results freeze pack(s) at temperature(s) between 0°F and -10°F (-23°C to -28°C). When not in use, store unfrozen in a cool, dry, climate-controlled place. Use within 2 years of purchase date.

CARE: Hand-wash using standard cleaning protocols of diluted mild detergents and rinse thoroughly to avoid skin irritation. Reshape flat and allow to air/towel dry only. Do not twist. Do not machine wash.

© 2023 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *287852*




REV A 11/23


54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



1.

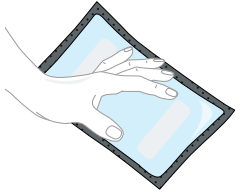

2.

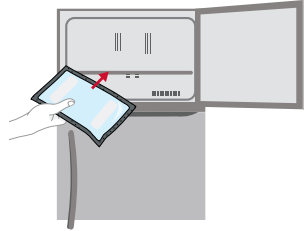


1. Ensure contents are evenly distributed throughout pack(s)

2. Place in freezer at least 4 hours before application


3. Remove pack(s) from the freezer

4. Prepare pack(s) by massaging to desired softness

4.

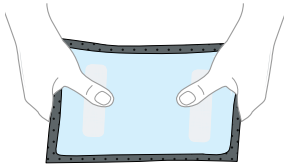


5.

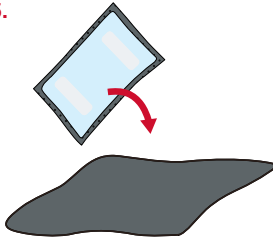


5. Place anywhere on the wrap or device with soft black barrier facing the skin

6. Replace pack(s) after 4+ hours for continuous therapeutic cold

7.

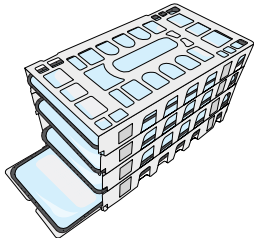

8.

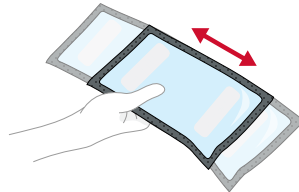

Optional:

7. Place bags in Direct Freeze Package (DFP) to accelerate freeze time and optimize storage

8. Shake pack as needed before freezing to maintain softness and freeze pack flat

Note: Follow the instructions and regimen of your healthcare provider

Scan for application and accessories.

