

PRODUCT SIZING

ModelNo. 2XY-06ZZ

Select desired packaging, style, and length

Packaging	X	Style	Y	06	Width	ZZ	Length
Bags Only	3	Bags Only	0	06	06" 15cm	07 06" 15cm	10 10" 25cm
				06	06" 15cm	10 10" 25cm	
				06	06" 15cm	12 12" 30cm	

Note: All options are available in 2 packs and bulk quantities of 24 and 48

FIRST GEL® FEATURES:

- Maintains ideal on-patient temperature for up to 2 hours
- Can be used for hot or cold therapy
- Ultra-soft malleability to conform to the targeted area
- Helps to mitigate the need for opioid-centric treatment
- No pumps or hoses, allowing for flexibility and mobility in use
- Safe, Non-Toxic, and repeatably effective

DIRECT STICK™ FEATURES:

- Patent-pending technology allows application anywhere on our device
- Soft thermal barrier for comfortable direct skin contact
- Intuitive design for quick and straightforward application
- Eliminates messy and bacteria-prone pockets

FIRST GEL® DIRECT STICK™ PACKS

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Inflammation
- Pain following Injury or Surgical Procedure

WARNING: This product is intended for single patient use at the direction of a medical professional and does not prevent or reduce the risk of injury. This product is not intended for internal use. If ingested, seek medical attention and treat symptomatically. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CAUTION: This product is non-toxic and is intended for external use only. Do not apply plastic barriers directly to the skin. For use on intact skin only. DISCARD IF PUNCTURED OR TORN. DO NOT OVERHEAT. If deformation occurs while heating, wait until the pack is cooled before handling.

STORAGE: This product is sensitive to direct light and heat. For best results freeze pack(s) at temperature(s) between 0°F and -10°F (-23°C to -28°C). When not in use, store unfrozen in a cool, dry, climate-controlled place. Use within 2 years of purchase date.

CARE: Hand-wash using standard cleaning protocols of diluted mild detergents and rinse thoroughly to avoid skin irritation. Reshape flat and allow to air/towel dry only. Do not twist. Do not machine wash.

© 2023 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *287853*



REV A 11/23



54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



INSTRUCTIONS FOR USE



US and Foreign Patents Pending

ModelNo. 2XY-06ZZ

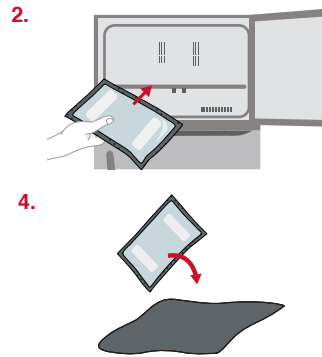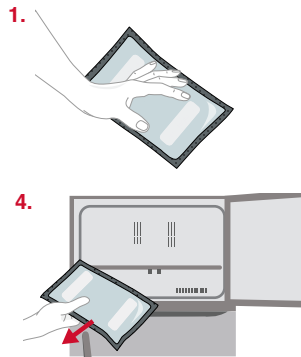

INSTRUCTIONS - COLD THERAPY

1. Ensure contents are evenly distributed throughout pack(s)
2. Place in freezer at least 2 hours before application
3. Remove pack(s) from the freezer and massage to desired softness
4. Place pack(s) anywhere on wrap or device
5. Replace pack(s) after 2+ hours for continuous therapeutic cold

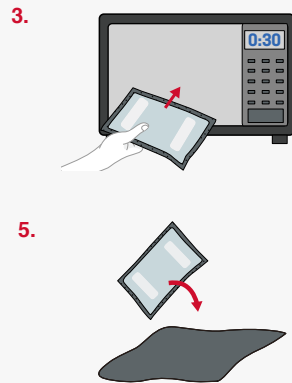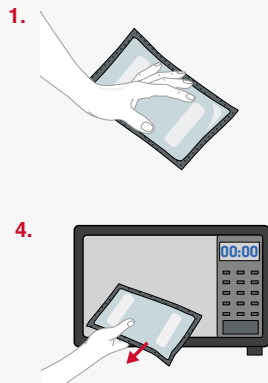

INSTRUCTIONS - HOT THERAPY

1. Ensure contents are evenly distributed throughout pack(s)
2. Allow pack to reach room temperature
3. Heat in microwave at full power for 30 seconds
4. Remove from microwave and knead pack(s)
 - a. If pack(s) is too hot, allow cooling
 - b. If pack(s) is too cool, heat in increments of 10 seconds
5. Place anywhere on the wrap or device

Note: Follow the instructions and regimen of your healthcare provider