

PRODUCT INFO

Product Name	Size	Ref No.
Platform Hand Orthosis	Universal	1105-00

OPTIONAL COLD THERAPY ACCESSORIES

Product Name	Size	Ref No.
FirstIce® Direct Stick™ Cold Therapy Pack	6 x 7"	100-0607

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

PLATFORM HAND ORTHOSIS

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Carpal Tunnel Syndrome
- Finger Contractures
- Wrist Sprains / Strains

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2022 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *282466*



REV A 8/22







54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



INSTRUCTIONS FOR USE

US and Foreign Patents Pending

Model No. 1105

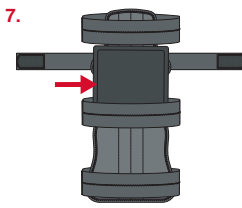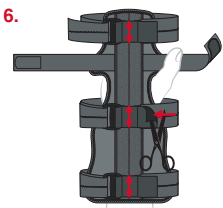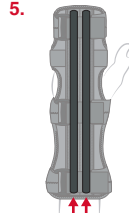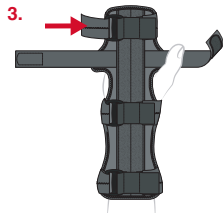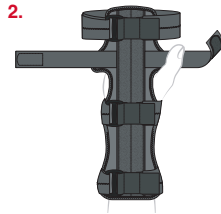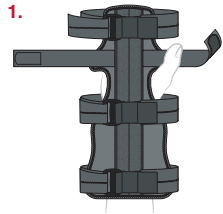

1. Open and loosen all straps then press palm firmly into palmar pocket.
2. Align orthosis to center of wrist, position and secure wrist then forearm straps.
3. Position finger strap and tighten to appropriate pressure.
4. Secure palmar straps to the back of the hand.

Customization

5. Adjust malleable stays as necessary.
6. All moveable straps are adjustable and trimmable.

Optional Cold Therapy Accessory

7. Apply Direct Stick™ Cold Therapy to affected area and adjust straps as required.

*Scan for application
and accessories.*

