

PRODUCT SIZING

Ref No. 1142 & 1143-XX

Measure the circumference of wrist at narrowest

Size	XX	Min	Max	Min	Max
UNI	00	6.0"	11.0"	15cm	28cm

Sizing Note: For Y specify L for left or R for right

UNIVERSAL WRIST ORTHOSIS

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Carpal Tunnel Syndrome
- Wrist Sprains and Strains
- Wrist Immobilization
- Control of Post-op Effusion

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2022 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *285391*



REV A 2/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



INSTRUCTIONS FOR USE

Model No. 1142/3


1.




2.




3.

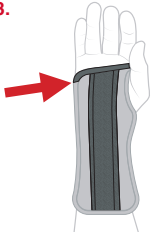

4.

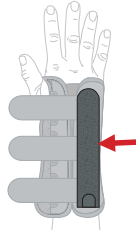

5.

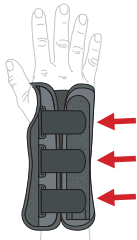

1. Loosen straps and place hand in brace.
2. Align with the center of the wrist.
3. Position stay at the palmar crease.
4. Position removable stay on top of hand.
5. Secure straps.

*Scan for application
and accessories.*

