PRODUCT SIZING				Ref No. 1147 & 1148-XXY	
Measure the circumference of wrist at narrowest					
Size	XX	Min	Max	Min	Max
UNI	00	6.0"	11.0"	15cm	28cm

Sizing Note: For Y specify L for left or R for right

UNIVERSAL THUMB ORTHOSIS

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Thumb Injuries
- CMC Injuries
- Metacarpal Injuries

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.


CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

54 Hugh Adams Rd., Defuniak Springs, FL 32435 National: (800) 234-9004 International: (850) 892-5731 Fax: (800) 274-9006





INSTRUCTIONS FOR USE

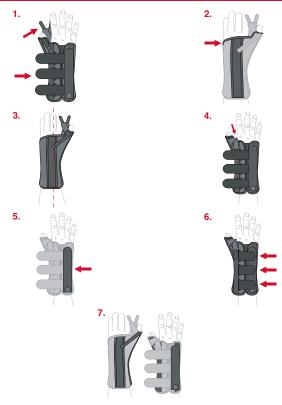

- 1. Loosen all straps and place hand into brace.
- 2. Position stay at the palmar crease.
- 3. Align with the center of the wrist.
- 4. Secure thumb straps starting closest to the hand.
- 5. Position removable stay on top of hand.
- 6. Secure wrist straps.

Customization (as required) 7. Remove, stack or shape stays for desired function.

Scan for application and accessories.

