

PRODUCT SIZING

Ref No. 1258/9-XXYY

Measure leg circumference 6" above mid patella

Length	YY	Max Universal	Max X-Circum
12" 30cm	12	20" 51cm	27" 86cm
14" 36cm	14	24" 61cm	31" 78cm
16" 41cm	16	28" 71cm	35" 89cm
18" 46cm	18	28" 71cm	36" 91cm
20" 51cm	20	28" 71cm	36" 91cm
22" 56cm	22	28" 71cm	36" 91cm
24" 61cm	24	28" 71cm	36" 91cm
26" 66cm	26	28" 71cm	36" 91cm
28" 71cm	28	28" 71cm	36" 91cm

Sizing Note: Specify XX as 00 for Universal or 50 for Extra Large Circumference.

OPTIONAL COLD THERAPY ACCESSORIES

Product Name	Size	Ref No.
Firstice® Direct Stick™ Cold Therapy Pack	06 x 07"	100-0607
	06 x 10"	100-0610
	06 x 12"	100-0612
Patella Strap	Universal	120-00

Note: Foam model is not cold therapy ready.

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

© 2022 Professional Products, Inc.

www.ezywrap.com

Made in the USA

R&ID# *282596*



REV A 8/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435

National: (800) 234-9004

Fax: (800) 274-9006



COMPRESSION KNEE IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-operative rehabilitation
- Knee ligament injuries

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.






CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.



INSTRUCTIONS FOR USE

US and Foreign Patents Pending

ModelNo. 1258 & 1259

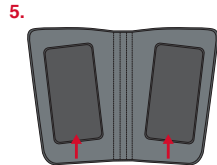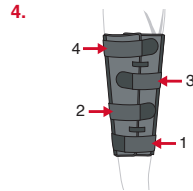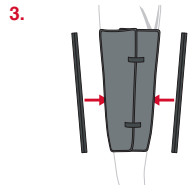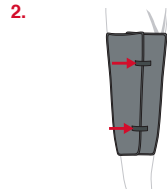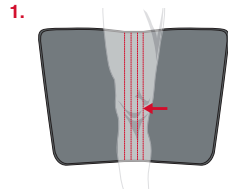

1. Position device with contoured stays centered behind affected knee.
2. Close base with hook-and-loop assist straps.
3. Place medial / lateral stays behind patella. Shape as required.
4. Alternate securing straps from ankle to thigh.

Optional Cold Therapy Accessory

5. Apply optional Direct Stick™ Cold Therapy on compatible area.

Note: Foam model is not cold therapy ready.

*Scan for application
and accessories.*

