

PRODUCT SIZING

Ref No. 1278 & 1279-00YY

Measure leg circumference 6" above mid patella

Length	YY	Min	Max
12" 30cm	12	15" 43cm	20" 51cm
14" 36cm	14	17" 43cm	24" 66cm
16" 41cm	16	18" 46cm	28" 71cm
18" 46cm	18	18" 46cm	28" 71cm
20" 51cm	20	19" 48cm	28" 71cm
22" 56cm	22	19" 48cm	28" 71cm
24" 61cm	24	20" 51cm	28" 71cm
26" 66cm	26	20" 51cm	28" 71cm

OPTIONAL COLD THERAPY ACCESSORIES

Product Name	Size	Ref No.
Firstlice® Direct Stick™ Cold Therapy Pack	06 x 07"	100-0607
	06 x 10"	100-0610
	06 x 12"	100-0612
Patella Strap	Universal	120-00

Note: Foam model is not cold therapy ready.

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

© 2021 Professional Products, Inc.
www.ezywrap.com
Made in the USA

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
Fax: (800) 274-9006

R&ID# *282618*



REV A 9/21



TRI-PANEL KNEE IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-operative rehabilitation
- Knee ligament injuries

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.




INSTRUCTIONS FOR USE

US and Foreign Patents Pending

Model No. 1278/9


1.

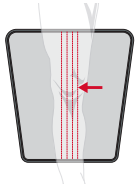


2.




3.

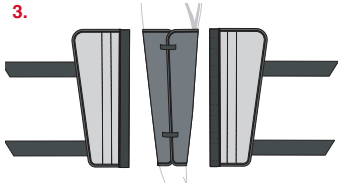


4.

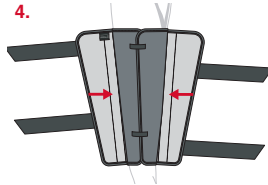

5.

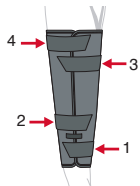

6.

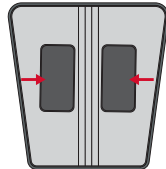

1. Position orthosis with stays centered behind affected knee.
2. Close base with quick connect-assist straps.
3. Angle lateral panels for custom sizing.
4. Shape medial / lateral stays as required.
5. Alternate securing straps from ankle to thigh.

Optional Cold Therapy Accessory

6. Apply optional Direct Stick™ Cold Therapy on compatible area.
Note: Foam model is not cold therapy ready.

*Scan for application
and accessories.*

