

PRODUCT SIZING

Ref No. 1288 & 1289-XXYY

Measure leg circumference 6" above mid patella

Size	XX	Min		Max		YY	Length	
PED	05	12"	30cm	14"	36cm	10*	10"	25cm
XS	10	16"	41cm	18"	46cm	12*	12"	30cm
S	20	18"	46cm	20"	51cm	14	14"	36cm
M	30	20"	51cm	22"	56cm	16	16"	41cm
L	40	22"	56cm	23"	58cm	18	18"	46cm
XL	50	23"	58cm	26"	66cm	20	20"	51cm
XXL	60	26"	66cm	28"	71cm	22	22"	56cm
XXXL	70	28"	71cm	30"	76cm	24	24"	61cm
						26	26"	66cm

Sizing Note: Specify XX for size and YY for length; Only in Pediatric*

OPTIONAL COLD THERAPY ACCESSORIES

Product Name	Size	Ref No.
Firstice® Direct Stick™ Cold Therapy Pack	06 x 07"	100-0607
	06 x 10"	100-0610
	06 x 12"	100-0612
Patella Strap	Universal	120-00

Note: Foam model is not cold therapy ready.

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

© 2022 Professional Products, Inc.

www.ezywrap.com

Made in the USA

R&ID# *282622*



REV A 9/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435

National: (800) 234-9004

International: (850) 892-5731

Fax: (800) 274-9006



SIZED KNEE IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications


- Post-operative rehabilitation
- Knee ligament injuries

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.


CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.




1.

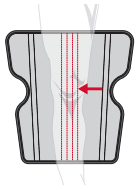


2.

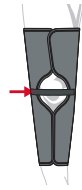

3.

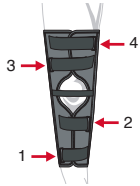

4.

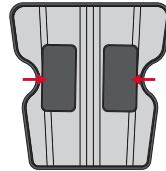

1. Position orthosis with stays centered behind affected knee.
2. Close base with quick connect-assist strap.
3. Alternate securing straps from ankle to thigh.

Optional Cold Therapy Accessory

4. Apply optional Direct Stick™ Cold Therapy on compatible area.
Note: Foam model is not cold therapy ready.

*Scan for application
and accessories.*

