

PRODUCT SIZING

Ref No. 1299/8 - 00YY

Measure leg circumference 6" above mid patella

| Length | | YY | Min | | Max | |
|--------|------|----|-----|------|-----|------|
| 14" | 36cm | 14 | 17" | 43cm | 24" | 61cm |
| 16" | 41cm | 16 | 18" | 46cm | 28" | 71cm |
| 18" | 46cm | 18 | 18" | 46cm | 28" | 71cm |
| 20" | 51cm | 20 | 19" | 48cm | 28" | 71cm |
| 22" | 56cm | 22 | 19" | 48cm | 28" | 71cm |
| 24" | 61cm | 24 | 20" | 51cm | 28" | 71cm |
| 26" | 66cm | 26 | 20" | 51cm | 28" | 71cm |

OPTIONAL COLD THERAPY ACCESSORIES

| Product Name | Size | Ref No. |
|---|-----------|----------|
| FirstIce® Direct Stick™ Cold Therapy Pack | 06 x 07" | 100-0607 |
| | 06 x 10" | 100-0610 |
| | 06 x 12" | 100-0612 |
| Patella Strap | Universal | 120-00 |

Note: Foam model is not cold therapy ready.

FIRSTICE® FEATURES:

- Patented technology that lasts five times longer than gel.
- Direct Stick™ packs eliminate the need for pockets.
- Non-toxic, soft and malleable to conform to the body.

© 2022 Professional Products, Inc.

www.ezywrap.com

Made in the USA

R&ID# *283248*



REV A 9/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435

National: (800) 234-9004

International: (850) 892-5731

Fax: (800) 274-9006



UNIVERSAL KNEE IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-operative rehabilitation
- Knee ligament injuries

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.


CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.



INSTRUCTIONS FOR USE

US and Foreign Patents Pending

ModelNo. 1298 & 1299

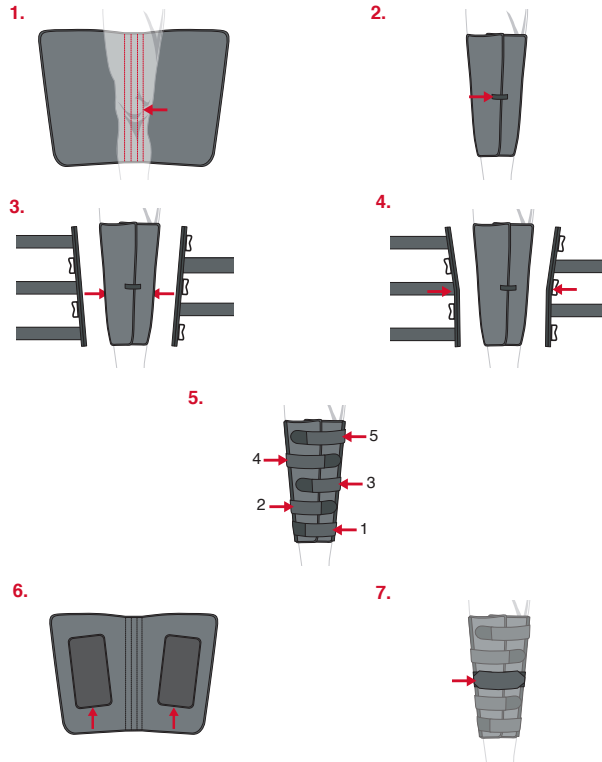

1. Position orthosis with stays centered behind affected knee.
2. Close base with quick connect-assist strap.
3. Angle lateral panels for custom sizing.
4. Shape medial / lateral stays as required.
5. Alternate securing straps from ankle to thigh (see diagram).

Optional Accessories

6. Apply optional Direct Stick™ Cold Therapy on compatible area.
Note: Foam model is not cold therapy ready.
7. Apply optional patella strap for stabilization.

*Scan for application
and accessories.*

