

PRODUCT SIZING

Ref No. 2306-XX

Measure length of lateral epicondyle to knuckle.

Size	XX	Min	Max	Min	Max
XS	10	8"	10"	20cm	25cm
S	20	10"	12"	25cm	30cm
M	30	12"	14"	30cm	35cm
L	40	14"	16"	35cm	40cm
XL	50	16"	18"	40cm	46cm

ACCOMMODATOR ARM SLING

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post operative support and positioning
- Injuries requiring immobilization

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2021 Professional Products, Inc.
www.ezywrap.com
Made in the USA




R&ID# *282548*



REV A 9/21

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



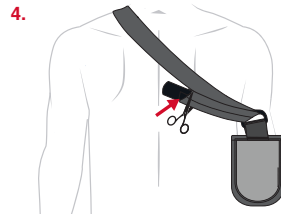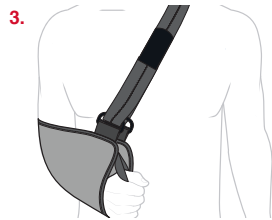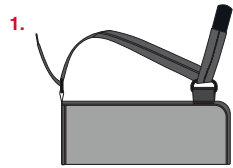

1. Loosen shoulder strap before application.
2. Place elbow in the back of envelope and thumb in thumb loop.
3. Position shoulder straps for ease of removal.
4. Adjust excess shoulder strap lengths from back, and trim as necessary.

*Scan for application
and accessories.*

