

PRODUCT SIZING

Ref No. 2351-XX

Measure length of lateral epicondyle to knuckle.

| Size | XX | Min | Max | Min | Max |
|------|----|-----|-----|------|------|
| XS | 10 | 8" | 10" | 20cm | 25cm |
| S | 20 | 10" | 12" | 25cm | 30cm |
| M | 30 | 12" | 14" | 30cm | 35cm |
| L | 40 | 14" | 16" | 35cm | 40cm |
| XL | 50 | 16" | 18" | 40cm | 46cm |

TAPERED SHOULDER IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-operative rehabilitation of the shoulder
- Injuries requiring immobilization

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2022 Professional Products, Inc.
www.ezywrap.com
Made in the USA





R&ID# *282626*



REV A 9/22

54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



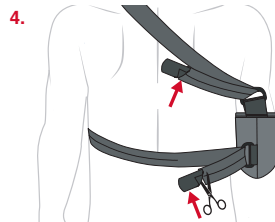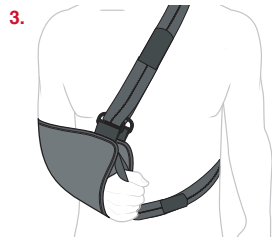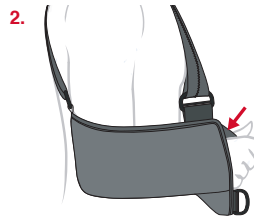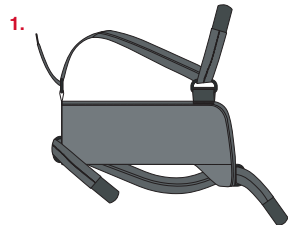

1. Loosen straps before application.
2. Place elbow in the back of envelope and thumb in thumb loop.
3. Position waist and shoulder straps for ease of removal.
4. Adjust excess waist and shoulder strap lengths from back; trim as necessary.

***Scan for application
and accessories.***

