

PRODUCT SIZING

Ref No. 2384-XX

Measure circumference of chest including affected arm

Size	XX		Max
PED	04	32"	81cm
UNI	00	53"	134cm

SLING AND SWATHE SHOULDER IMMOBILIZER

INDICATIONS:

Patient Population: Adults and Children with identified indications

- Post-operative support and positioning
- Injuries requiring immobilization

WARNING: This product is intended for single patient use at the direction of a medical professional and will not prevent or reduce the risk of injury. Do Not Over Tighten. If you experience a sudden increase in pain, numbness, or unusual activity with this device, discontinue use immediately and consult your healthcare provider.

CARE: Hand wash using a mild detergent. Rinse thoroughly. Air Dry only. Do not tumble dry.

© 2022 Professional Products, Inc.
www.ezywrap.com
Made in the USA

R&ID# *282645*



REV A 9/22



54 Hugh Adams Rd., Defuniak Springs, FL 32435
National: (800) 234-9004
International: (850) 892-5731
Fax: (800) 274-9006



INSTRUCTIONS FOR USE

US and Foreign Patents Pending

Model No. 2384

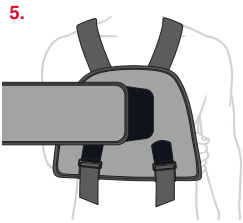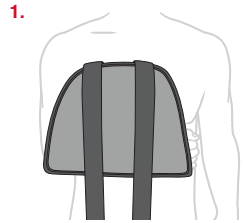

1. Insert arm into sling envelope.
2. Criss-cross figure 8 straps.
3. Bring straps under arms and attach to envelope.
4. Adjust buckles as necessary.
5. Fix end of swathe to envelope.
6. Wrap around affected arm and fix end to envelope.

*Scan for application
and accessories.*

